



THUASNE

ORTHOPAEDICS
2022-2023



THUASNE GROUP

is over 170 years old

From its Saint-Etienne base, THUASNE Group has been drawing on its know-how for more than 170 years to conceive, develop and manufacture medical devices enabling all patients and consumers to become active players in their own health.

Derived from the shared history of 6 generations of entrepreneurs, THUASNE offers concrete, simple, adapted and innovative health solutions for improved mobility, in more than 85 countries. Our expertise is aimed at patients in the fields of orthopaedics (belts, braces), medical compression (phlebology, lymphology, wound healing), women's health (external breast forms, lingerie) and homecare.



With its operations spanning the worlds of medicine, materials and digital technology, Thuasne creates and distributes medical devices enabling all patients and consumers to become active players in their own health. Year after year, our joint commitment enables better management of numerous chronic conditions (back problems, musculoskeletal disorders, venous insufficiency, lymphoedema, etc.) using non-invasive therapeutic solutions

Elizabeth Ducottet
Chairman and CEO

A handwritten signature in black ink that reads "Elizabeth Ducottet". The signature is written in a cursive style and is positioned above a thin horizontal line.

THUASNE

today is...



2 200
employees around
the globe



A presence in
85 countries



14
industrial sites in Europe
and the United-States



22 000
professional customers



30 to 40
new product references
every year



One Thuasne brace sold every
2 seconds
somewhere in the world



15 millions
kilometres of knitted thread



5,000
km of fabric produced/year

The «Made in Thuasne» label



The capacity to fully control every step in the value chain

Because THUASNE is committed to offering patients and health professionals effective, high-quality products, it can guarantee that all its products are designed, made and distributed using technologies mastered by the Group following extensive experience accumulated over the years.


Conceive Create Manufacture Control Supply Deliver.

Knitting expertise for the benefit of patients

Because THUASNE is committed to patients, its knitted braces are designed using exclusive expertise to promote efficacy, comfort and compliance.

They are **effective** thanks to their compressive properties, restoring proprioception and improving the blood circulation.


They are **comfortable** thanks to their breathability and specific knit (different techniques and comfort zones).

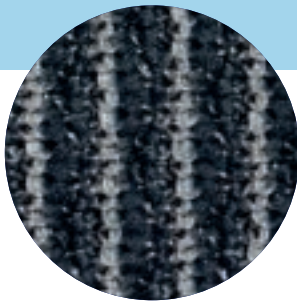

Integrated silicone knit
for good support



Ventilated knit
for better breathability

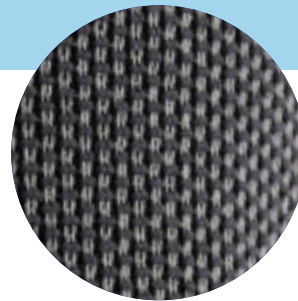


Tight, dense knit
for a compressive effect
and improved resistance



Non-compressive tab
for easy fitting



Less compressive "accordion" knit for freedom of movement

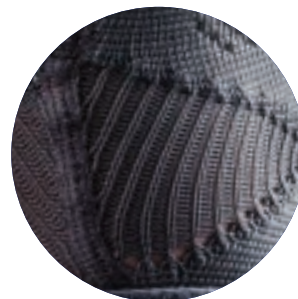

Lighter, compression-free knit,
for enhanced comfort behind the
knee, at the elbow crease, etc...


CONTENTS

Measurements	07
Back	08
Neck	24
Shoulder, Elbow	30
Wrist, Thumb	36
Knee	42
Ankle	60
Foot	70
Size 0	80
Thuasne Sport	86
Other Ranges & Order Forms	91
Product Codes	96


All the medical devices mentioned in this document are CE marked according to the Regulation 2017/745 on medical devices, unless specifically identified as "NOT CE MARKED". Products mentioned in this document are CE class I.
Please contact Thuasne should you need any additional information on devices classification. Please carefully read the instructions for use, indications and contraindications of the product. The medical indications for the products are available on certain product pages or summarised on the following product pages: pages 10, 26, 44, 62, 70. The contraindications or precautions for use, where relevant, are also specified for each medical device.

MEASUREMENTS

BACK

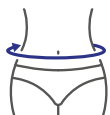

Measure the waist circumference at the level of the navel

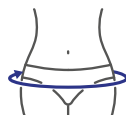

Measure the hip circumference

NECK




Measure the neck circumference



Measure the neck height, head upright, from the trachea to the chin

SHOULDER - ELBOW

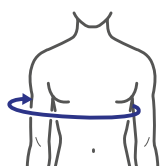

Measure the circumference under the chest with the injured arm.



Measure the arm circumference at the biceps level.



Measure the X cm circumference under the elbow

WRIST - FINGERS




Measure the circumference of the wrist



Measure the width of the hand at the level of the metacarpals



Measure the circumference of the index finger at the base of the nail

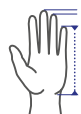

Measure the distance between the tip of the injured finger and the wrist


KNEE - CALF



Measure the circumference of the knee in the middle of the patella, knee in slight flexion.




Measure the circumference of the calf 25 mm below the fibular head

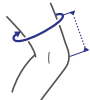


Measure the circumference of the thigh at the highest point or X cm above the knee.




Measure the width of the knee

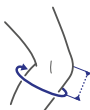

Measure calf circumference Y cm below the knee.

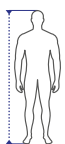

Measure the size

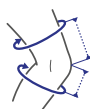

Measure the circumference of the thigh X cm above the knee and the calf circumference Y cm below the knee.



Measure the floor-to-groin height

ANKLE - FOOT



Measure the circumference of the ankle above the malleoli



Measure the short heel circumference



Foot length




Shoe size (UK)




BACK

LUMBAR BELTS: KEY BENEFITS

1 Relieve the lumbar zone Increase in intra-abdominal pressure

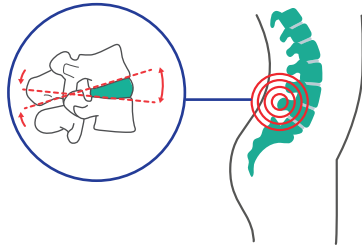


Pain-relieving effect due to compression

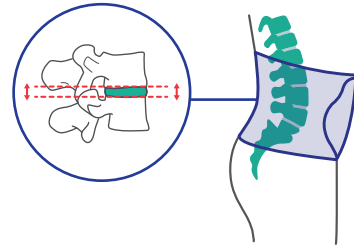

Better distribution of load on intervertebral discs

2 Promote a pain-relieving position (posture correction) and limit extreme, painful movements

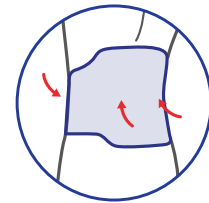

3 Aid early remobilisation and reduce apprehension about everyday activities

Did you know?

1,500 is the number of times our backs have to work every day.¹

80% of the population will suffer from low back pain once in their lifetime.²


Staying Active is the recommended treatment by the NHS for lower back pain 3

1: The median number of spinal movements performed within 24 h was approximately 4,400 . source :Rohlmann A, Consmüller T, Dreischarf M, Bashkuev M, Disch A, Pries E, Duda GN, Schmidt H. Measurement of the number of lumbar spinal movements in the sagittal plane in a 24-hour period. Eur Spine J. 2014 Nov;23(11):2375-84. doi: 10.1007/s00586-014-3588-0. Epub 2014 Sep 20. PMID: 25238799.

2: Source: www.nhs.uk/conditions/back-pain/treatment/

3: National Institute for Health and Care Excellence. (2016) Low back pain and sciatica in over 16s: assessment and management [NICE Guideline No. 59] <https://www.nice.org.uk/guidance/ng59>

BACK | Which solution for which condition?

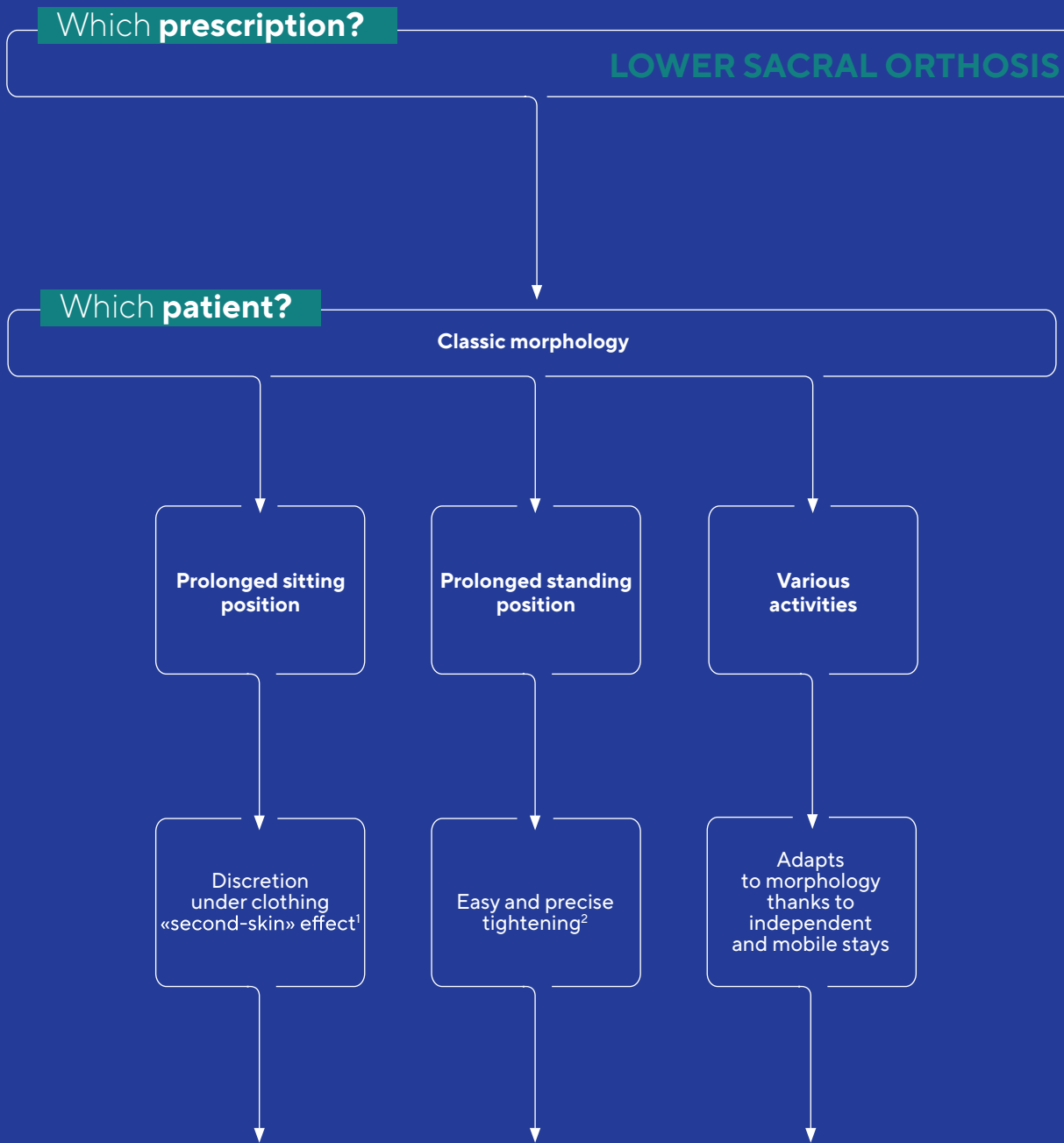

LombaSkin



page
16

LombaStab




page
18

Lombatech®



page
16

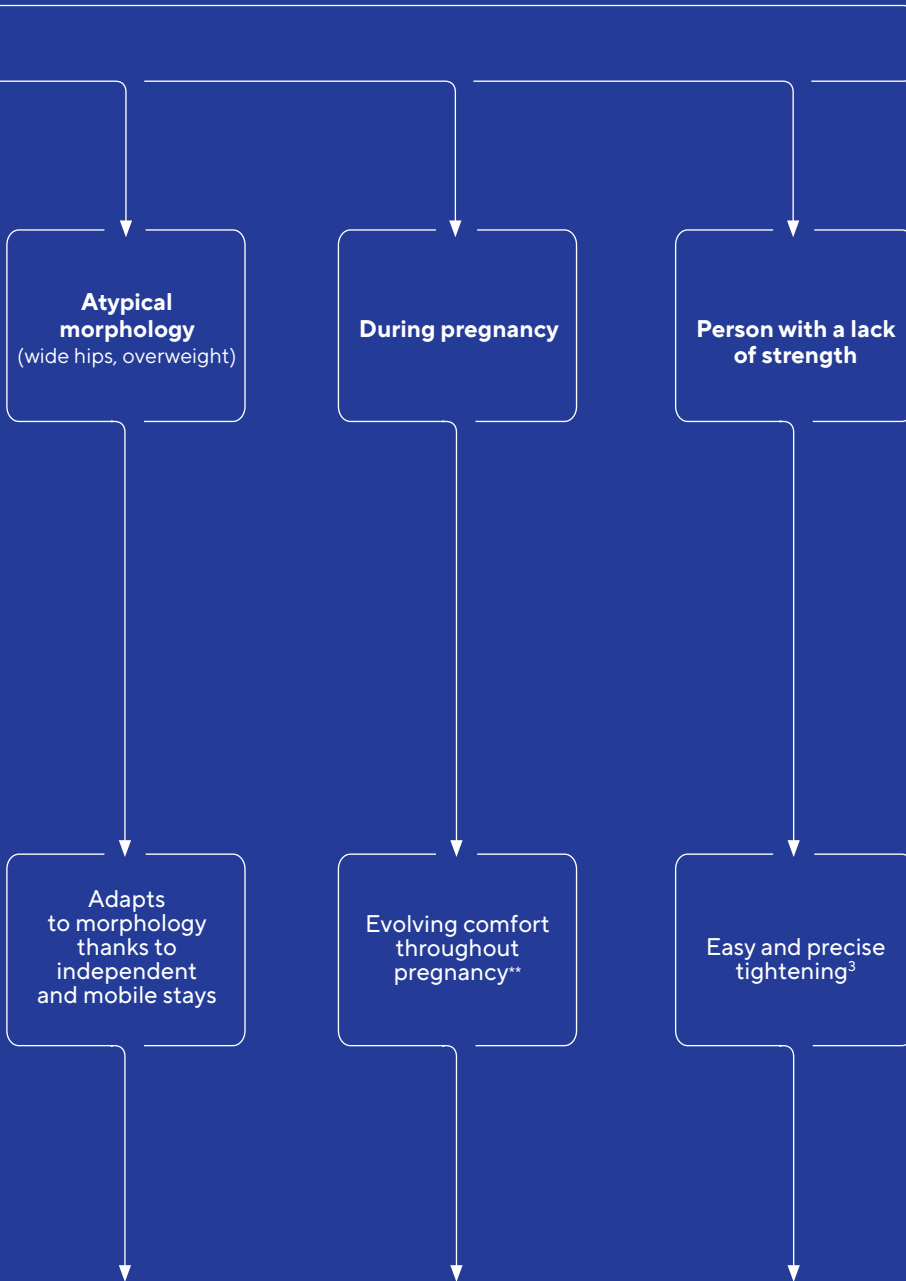

Lombatech®



page
16

Lombamum®



page
17

LombaStab



page
18

(1) OC1806 In-house study report, 2019. (2) Study conducted in-house on a panel of 13 people, July 2019
(3) Study conducted in-house on a panel of 13 people, July 2019.
* additional reinforced strap. ** evolving belt



SPINAL ORTHOSIS

SOFT CERVICAL COLLARS C1 - C7

SUPPORT



Ortel C1 anatomic
Analgesic support for cervical spine



Ortel C2 plus
Analgesic support for cervical spine (reinforced)

C1 - C7 RIGID CERVICAL COLLARS

IMMOBILISATION/STABILISATION



Ortel T-Neck
Stable, non-dislocated vertebral body fractures (without neurological symptoms)

DORSO-LUMBAR ORTHOSIS T7 - S1

COMPRESSION/SUPPORT



LombaStab immo
Post-operative stabilisation
Spondylolysis
Spinal Stenosis
Height: 35 cm



LombaStab high
LBP*
Spinal Stenosis
Spondylolysis
Available in two heights:
- 35 cm
- 40 cm

T3 - S1 THORACO-LUMBO-SACRAL ORTHOSIS

STABILISATION



LombaStab dorso
Stable traumatic and osteoporotic fractures. Scheuermann's kyphosis



Sleeq® Flex
Osteoporotic compression fractures. Degenerative postural presentations

IMMOBILISATION



Sleeq® MAX
Post operative vertebral stabilisation
Stable vertebral fractures
Complex spinal pathologies, see IFU

LUMBAR ORTHOSIS L1 - S1

COMPRESSION/SUPPORT



LombaStab
LBP*
Spinal Stenosis
Available in two heights:
- 21 cm
- 26 cm



Lombatech®
LBP*
Height: 21 to 27 cm, 26 to 32 cm



LombaFirst
LBP*
Spinal stenosis
Available in two heights:
- 21 cm
- 26 cm



LombaStart
LBP*
Spinal stenosis
Additional strap for increased compression
Available in two heights:
- 21 cm, - 26 cm

L1 - S1 SACRO-LUMBAR ORTHOSIS

COMPRESSION/SUPPORT



Lombamum®
Maternity belt
Completely safe for use with LBP*, pelvis and sacroiliac pain during pregnancy



DorsoRigid 35*
Immobilising spinal hyperextension orthosis

SACROILIAC ORTHOSIS S1 - S5



Ortel P
Sacroiliac instability pain
Symphyseal instability pain

*Low back pain.
Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region. All the medical devices mentioned are CE marked according to the European council directive 93/42/EEC and its relatives or the Regulation 2017/745 on medical devices. Products mentioned are CE class I. Please read carefully the instructions for use, indications and contraindications of the product.
Last revision date: 06/2021
Ref: 2105180

BELTS LUMBAR SPINE
Offering reinforced support



LombaFirst

Lumbar support belt

0876 02 (21 cm) - 0875 02 (26 cm)



Indicated for

Non-specific Low Back Pain (acute, subacute, chronic phases).
Lumbar stenosis.

POINTS TO REMEMBER

Discreet and lightweight back brace thanks to its thin fabric and seamless sewing technology


Strong support for an efficient analgesic effect* thanks to the conformable spring-loaded steel stays

Additional strap to reach a higher compression

Finger loops for an easy fitting

Designed angles⁽¹⁾ for more comfort while wearing the belt

Waist circumference

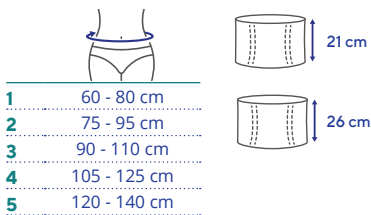

Colour: Black ●

LombaStart

Lumbar support belt

0874 02 (21 cm) - 0873 02 (26 cm)



Indicated for

Non-specific Low Back Pain (acute, subacute, and chronic phases).
Spinal Stenosis

POINTS TO REMEMBER

Discreet and lightweight back brace thanks to its thin fabric and seamless sewing technology


Strong support for an efficient analgesic effect* thanks to the conformable spring-loaded steel stays

Additional strap to reach a higher compression

Finger loops for an easy fitting

Designed angles at hips and ribs area for more comfort while wearing the belt.

Waist circumference

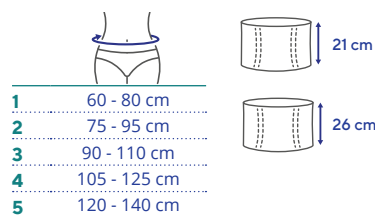

Colour: Black ●



Presentation video
available online

⁽¹⁾ Less material at hips and ribs area
* internal data

* internal data
⁽¹⁾ The additional strap is doubled to provide more compression than a single strap.
⁽²⁾ OC1806 In-house study report, 2019.

LombaSkin

"Second skin" compact lumbar support belt

0871 03 (21 cm) - 0870 03 (26 cm)



Indicated for

Non-specific Low Back Pain (acute, subacute, chronic phases).

Lumbar stenosis

Target patient

Low back pain sufferer with moderate physical activity e.g. *sedentary office worker*

POINTS TO REMEMBER

Comfortable to wear thanks to the "second skin effect" especially at the rib and hips area

Discreet and lightweight back brace thanks to its thin fabric and seamless sewing technology

Strong support for an efficient analgesic effect* thanks to the conformable spring-loaded steel stays

Specific elastic fabric ensures comfortable feel

Designed angles² for more comfort while wearing the belt.

Easy care: machine wash at 30°C



Waist circumference

1	60 - 80 cm	
2	75 - 95 cm	
3	90 - 110 cm	
4	105 - 125 cm	
5	120 - 140 cm	

Colour: Black

LombaTech®

Adaptable, reinforced lumbar support belt

0852 02 (21 cm) - 0851 03 (26 cm)



Indicated for

Non-specific Low Back Pain (acute, subacute, and chronic phases)

Spinal Stenosis

Target patient

Patient with varied activities or an atypical body shape

POINTS TO REMEMBER

1 - Adaptation to the body's movements: removable additional strap

2 - Precise targeting of painful area: additional strap

3 - Breathability: moisture wicking thanks to Combitec fabric¹ and Coolmax lining

4 - Coverage of a broad range of sizes: to fit as many patients as possible (very slim, large hips, round stomach, etc.)

5 - Easy Care: machine wash at 30 °C



Waist circumference

1	60 - 80 cm	
2	75 - 95 cm	
3	90 - 110 cm	
4	105 - 125 cm	
5	120 - 140 cm	

Colour: Grey



Presentation video
available online

(1) OC1806 In-house study report, 2019
(2) Less material at hips and ribs area
* internal data

(1) OC1806 In-house study report, 2019.

LombaMum®

Posture-correcting medical maternity belt

0805 01



indicated for


Pregnancy-related low back pain.
Pregnancy-related girdle pain (posterior pelvic pain,
sacro-iliac pain, symphyseal pain)

POINTS TO REMEMBER


- 1 - Adjustable throughout the pregnancy:** 2 abdominal support bands
- 2 - Adjustable support:** adjustable additional straps
- 3 - Comfort:** bamboo fibre lining
- 4 - Easy care:** Machine wash at 30 °C



JANUS 2014
DE LA SANTE

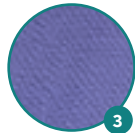

Waist circumference

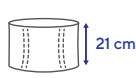

1 82 - 125 cm

Ortel P

Pelvic belt

2740 01



Indicated for

Pregnancy-related pain girdle (posterior pelvic pain,
sacro-iliac pain, symphyseal pain).


POINTS TO REMEMBER

Adjustable extra-strong support: elastic fabric and additional strap

Comfort: soft terry lining

Easy care: machine wash at 30 °C

Waist circumference

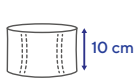

- 1 83 - 89 cm
- 2 90 - 96 cm
- 3 97 - 105 cm
- 4 106 - 114 cm
- 5 115 - 125 cm
- 6 126 - 136 cm

Colour: White



LombaStab

Lumbar support belt with an easy and precise tightening system

0854 02 (21 cm) - 0853 02 (26 cm)



Indicated for

Non-specific low back pain (acute, subacute and chronic phases)

Target patient

Persons looking for an easy and precise tightening to adapt it according to their pain. Persons in prolonged standing position. People lacking strength

POINTS TO REMEMBER

- 1 - **Easy and precise tightening**¹ thanks to the innovative lacing system
- 2 - **Strong support for an efficient analgesic effect*** thanks to the back-plastic plates and the adaptable spring steel stays
Adaptable to the anatomy in A or V-Shape thanks to the innovative quick lacing system and the smart designed angles at hips and ribs area
- 3 - **Symmetrical hand loops** to ensure the belt is properly centered on the spine
- 4 - **Washable at 30°C**



Waist circumference

	21 cm
	26 cm
1	60 - 80 cm
2	75 - 95 cm
3	90 - 110 cm
4	105 - 125 cm

Colour: Black/Dark Blue

* internal data
(1) Study conducted in-house on a panel of 13 people, July 2019

LombaStab high

Dorso-lumbar support belt with an easy and precise tightening system

0855 01 (35 cm) / 0856 01 (40 cm)



Indicated for

Non-specific low back pain (acute, subacute and chronic phases)
Lumbar-spinal stenosis
Spondylololysis.

Target patient

Persons looking for an easy and precise tightening to adapt it according to their pain. Persons in prolonged standing position. People lacking strength.

POINTS TO REMEMBER

- Easy and precise tightening**¹ thanks to the innovative lacing system
- Strong support for an efficient analgesic effect*** thanks to the back-plastic plates and the adaptable spring steel stays
- Specific elastic fabric** to reach the desired level of compression and to ensure a good breathability
- Adaptable to the anatomy in A or V-Shape** thanks to the innovative quick lacing system and the smart designed angles at hips and ribs area
- Symmetrical hand loops** to ensure the belt is properly centered on the spine
- Washable at 30°C**

Waist circumference

	35 cm
	40 cm
1	60 - 80 cm
2	75 - 95 cm
3	90 - 110 cm
4	105 - 125 cm
5	120 - 140 cm

Colour: Black/Dark Blue

* internal data
(1) Study conducted in-house on a panel of 13 people, July 2019

LombaStab immo

Dorso-lumbar support belt with corset and an easy and precise tightening system

0859 01

New



Indicated for

Non-specific low back pain (acute, subacute and chronic phases);
Spinal stenosis
Spondylolysis
Postoperative immobilisation
MODIC type 1

POINTS TO REMEMBER

Easy and precise tightening¹ thanks to the innovative lacing system

Strong support for an efficient analgesic effect* thanks to the back-plastic plates and adaptable back spring steel stays

Specific elastic fabric to reach the desired level of compression

Smart designed angles at hips and ribs area to ensure sitting and standing position comfort while wearing the belt.

Symmetrical hand loops to ensure the belt is properly centered on the spine

Thermoformable corset

Washable at 30°C

Waist circumference



35 cm

1	60 - 80 cm
2	75 - 95 cm
3	90 - 110 cm
4	105 - 125 cm
5	120 - 140 cm

Colour: Dark Blue

LombaStab dorso

Thoraco-lumbar support belt with an easy and precise tightening system

0847 01

New



Indicated for

Non-specific low back pain (acute, subacute and chronic phases)
Static disorders (kyphosis, lordosis)
Stable vertebral fracture (traumatic, osteoporotic)
Scheuermann's disease

Target patient

Patients with a waist size of between 60 and 140 cm

POINTS TO REMEMBER

3 flexion points thanks to the intra-abdominal pressure and the sternal support maintaining the body in a hyperextension

Easy and precise tightening¹ thanks to the innovative lacing system

Adaptability to the curvature of the spine thanks to the conformable and height-adjustable back splint.

Comfortable S-Shaped and padded shoulder strap preventing armpit friction.

Easy backpack design to facilitate application and tightening of the shoulder straps.

Specific elastic fabric to reach the desired level of compression and to ensure a good breathability

Waist circumference



35 cm

1	60 - 80 cm
2	75 - 95 cm
3	90 - 110 cm
4	105 - 125 cm
5	120 - 140 cm

Colour: Black/Dark Blue

* internal data
(1) Study conducted in-house on a panel of 13 people, July 2019

(1) Study conducted in-house on a panel of 13 people, July 2019

Sleeq® Flex

Universal TLSO

U50401004001



Indicated for

Mild to moderate osteoporosis / compression fractures. Postural issues and kyphosis.
Chronic or acute low/mid back pain / osteoarthritis / degenerative disc disease / bulging or herniated disc / low-mid back sprains and strains / lumbar and trunk muscle weakness.

POINTS TO REMEMBER

One size fits all - Easy universal sizing technology fits waist sizes 26 – 57”.

Semi-rigid comfort enhancing design.

Highest quality breathable materials.

Ventilated self-moulding support panels.

Wings can be angled and optimized to fit all body types.

Easy to Apply & Wear.

Available extension panel accommodates waist sizes 57 – 65” code UU50401001001.

Sleeq® MAX

Premium Post-Op/Post-Injury TLSO

U50401007001



Indicated for

Postoperative vertebral stabilisation/ fusion.
Stable vertebral fractures.
Mechanical or discogenic thoracic/lumbar & extremity pain. Degenerative disc disease. Post-injury to facilitate healing. Spondylolysis, spinal stenosis. Spondylolisthesis, spondylosis, laminectomy, discectomy, facet syndrome, spinal muscle weakness, arthritis, low back sprains and strains, prophylactic back support.

POINTS TO REMEMBER

One size fits all - Easy universal sizing technology fits waist sizes 26 – 57”.

Rigid Tri-Planar Control: sagittal/coronal/transverse plane motion restriction.

Increased intracavitary pressure to reduce spine & disc loading.

Easy to Apply & Wear.

Steel sternal bar & **reinforced** panels.

Wings can be angled and optimized to fit all body types.

Available extension panel accommodates waist sizes 57 – 65” code UU50401007001.

BOA Dual TLSO

U50301



Indicated for

Sagittal, coronal, transverse plane motion restriction / increased intracavitary pressure to reduce spine and disc loading / mechanical or discogenic thoracic / lumbar and extremity pain / post-operative spinal stabilization / fusion / degenerative disc disease / post-injury to facilitate healing / spondylolysis / spinal stenosis / spondylolisthesis / spondylosis laminectomy / discectomy / facet syndrome / spinal muscle weakness / arthritis / low back sprains, strains / prophylactic back support.

POINTS TO REMEMBER

Full-circumferential rigid support for the thoracolumbar spine.

Patentend Lacer Design Compression System.

Simple pull straps for easy tightening and adjustment.

Waist circumference




SM	25 - 30"
MD	30 - 35"
LG	35 - 42"
XL	42 - 48"
2X	48 - 57"

Legal manufacturer: Thuasne Deutschland GmbH

DorsoRigid 35

Immobilising spinal
hyper-extension brace

0890 02

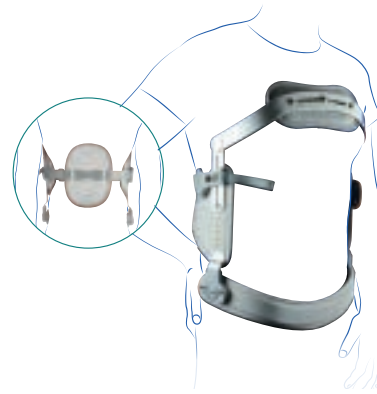

Indicated for

Post-operative immobilisation after spinal surgery. Reduction and stabilisation of vertebral fractures with preservation of the posterior wall (T10-L1/L2). Conservative treatment for stable vertebral fractures of the spine (T10-L1/L2). Osteoporotic or metastatic-related compression fractures of the thoracic spine and the dorsolumbar transition (T10-L1/L2). Osteomalacia with fractures. Dorsolumbar osteoarthritis. Osteoporosis and degenerative metastases of vertebral bodies


POINTS TO REMEMBER

Resting and static posture correction of the spine thanks to a stabilising and immobilising action, based on three pressure points

Rust-proof and adjustable

Water-resistant, can be worn in the swimming pool for physiotherapy sessions, for example

Waist circumference

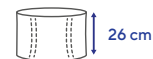

1	60 - 80 cm
2	75 - 95 cm
3	90 - 110 cm
4	105 - 125 cm
5	120 - 140 cm

Colour: Grey ●

Legal manufacturer: Orthoservice Ag.

Cemen

Thoracic and abdominal belt-bandage

2800 02




Indicated for

Thoracic and abdominal compression (rib fractures, post-operative complications).

POINTS TO REMEMBER

Strong support: elastic fabric

Easy to put on: self-adhesive fastening

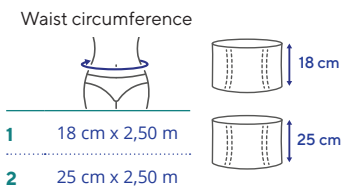

Colour: White

Cemen

Thoracic belt

2900 01



Indicated for


Thoracic compression: rib fractures, surgery, respiratory rehabilitation.

POINTS TO REMEMBER

Strong support thanks to hard-wearing elastic fabric

Easy to put on thanks to self-adhesive fastening

Machine wash at 30 °C

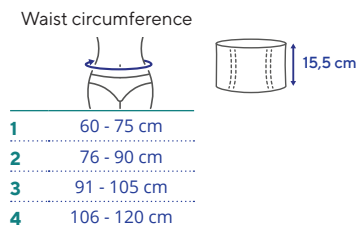

Colour: White

Stomex

Abdominal support belt for stoma patients

2700 02




Indicated for

Post-operative abdominal compression.

POINTS TO REMEMBER

Easy to use: front panel in ladder proof fabric, for passing through drains and colostomy bags

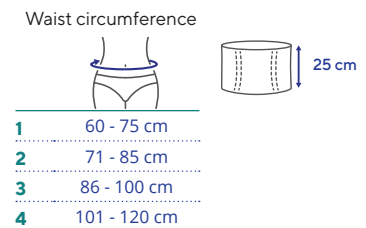

Dynabelt

Aerated abdominal support belt

7010 07 (16 cm) - 7010 08 (26 cm)

7010 09 (33 cm)



Indicated for


Support for minor upper and lower back pain
Support of the abdominal wall.

POINTS TO REMEMBER

Comfort and breathability: moisture wicking thanks to Combitex fabric

Easy to use as required: removable support stays

Can be sterilised in an autoclave

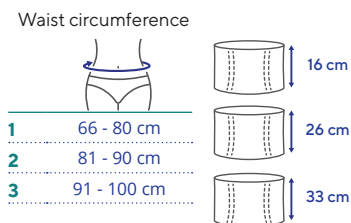

Ortel

Hernia truss

2730 01: unilateral (adaptable for right or left)

/ 2720 01: bilateral



Indicated for


Right or left inguinal hernia
Bilateral inguinal hernias
Pre- or post-operative wear
Permanent wear.

POINTS TO REMEMBER

Adaptable for right or left: exclusive buckle system

Comfort and efficacy: removable and repositionable silicone pads

Easy care: machine wash at 30 °C

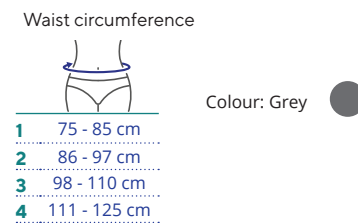



NECK

CERVICAL COLLAR: KEY BENEFITS

Cervical collars immobilise the cervical spine, limiting flexion, extension and rotation movements.

They act on several levels:

- 1 Pain-relieving effect**
By limiting movements and painful mechanical stresses
- 2 Relaxing effect**
By increasing local heat, thereby promoting muscle relaxation
- 3 Limitation/immobilisation of the cervical spine**
Varies depending on the degree of rigidity
- 4 Stimulation of proprioception**
By reinforcing control sensations from the spinal column

Did you know?

10 to 20%
of adults



suffer from neck pain¹⁻²
The neck is one of the areas of the body most affected by repetitive strain injuries in the office³

1/4 of all people with
an office job

complain of neck pain⁴

People who play **contact sports** (football, boxing, hockey, etc.) are also commonly affected⁵

(1-2) American College of Rheumatology. Patients and Public, Patient Education – Neck Pain, American College of Rheumatology. [Consulted on 24 February 2009]. www.rheumatology.org

(3) Côté P, Cassidy JD, et al. The annual incidence and course of neck pain in the general population: a population-based cohort study. *Pain*. 2004 Dec;112(3):267-73.

(4) Léna, P. (2013). Le cou, première victime de la vie au bureau. *Le figaro santé*.

(5) Passeport santé. (s.d.). Les symptômes, les personnes à risque et les facteurs de risque des troubles musculosquelettiques du cou (entorse cervicale, torticolis). Consulted on Passeport santé: <https://www.passeportsante.net/fr/Maux/Problemes/Fiche.aspx?doc=troubles-musculosquelettiques-cou-pm-symptomes-des-douleurs-au-cou>

NECK | Which collar for which condition?

Which prescription?

Which indication?

Torticollis, mild whiplash - mild sprain, cervical osteoarthritis, dropped head syndrome, post-surgery - post-operative, cervical syndrome, rehabilitation after wearing a rigid cervical collar.

Torticollis, mild whiplash - mild sprain, cervical osteoarthritis, dropped head syndrome, cervicobrachial neuralgia, cervical syndrome, rehabilitation after wearing a rigid cervical collar.

Flexible structure Foam

Semi-rigid structure Foam + plastic

Light support
●○○○

Moderate support
●●○○

Ortel C1 classic

Ortel C1 anatomic

Ortel C2 plus




page
28

page
28

page
28

PAIN-RELIEVING AND RELAXING EFFECT

COLLAR

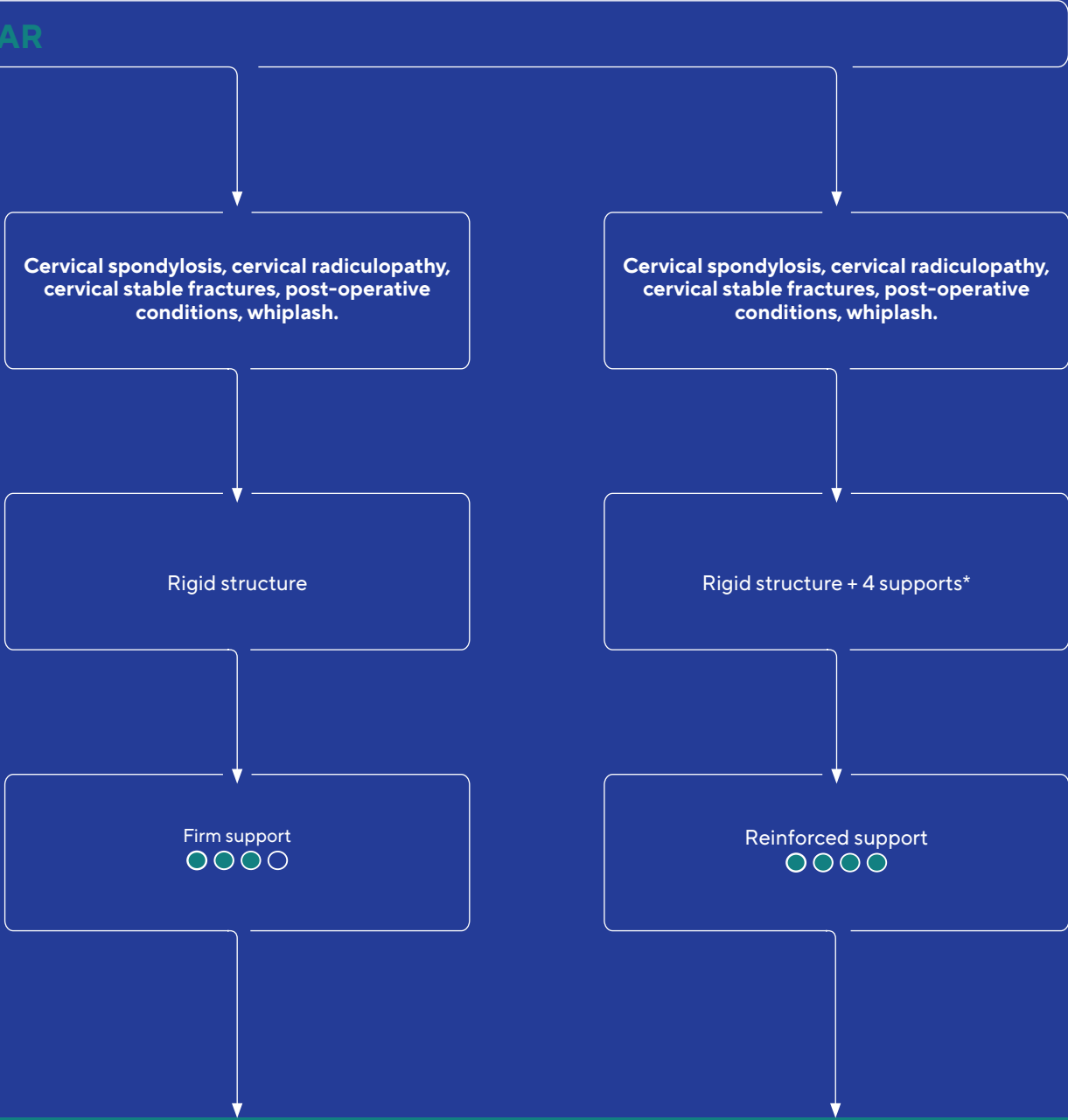

Ortel C3



page
29

LIMITATION OF FLEXION

Ortel T-Neck



page
29

COMPLETE RESTRICTION OF MOBILITY

Ortel C1 classic

Light cervical collar

2393 02



Indicated for

Torticollis, mild whiplash / mild sprain, cervical osteoarthritis, dropped head syndrome, post-surgery / post-operative, cervical syndrome, rehabilitation after wearing a rigid cervical collar.

Ortel C1 anatomic

Light anatomical cervical collar

2394 02



Indicated for

Analgesic support for the cervical spine (torticollis, benign sprains, cervical arthritis)

Ortel C2 plus

Semi-rigid cervical collar

2392 04



Indicated for

Reinforced and analgesic support for the cervical spine (benign sprains, cervical arthritis, torticollis, cervico-brachial neuralgia)

POINTS TO REMEMBER

Flexible support: high-quality foam

Comfort: removable and washable comfortable cover

Hold: precise self-fastening adjustment

Average height of 8 to 10 cm

POINTS TO REMEMBER

Flexible support: anatomically-shape compact support foam

Comfort: Removable and washable comfortable cover

Hold: precise self-fastening adjustment

Average height of 7.5, 9 or 11 cm

POINTS TO REMEMBER




Semi-rigid support: semi-rigid internal structure covered in compact support foam

Comfort: removable and washable comfortable cover

Hold: precise self-fastening adjustment

Anatomical shape

Average height of 7.5, 9 or 11 cm

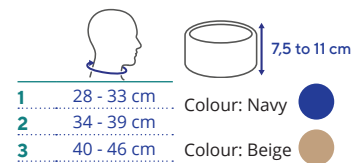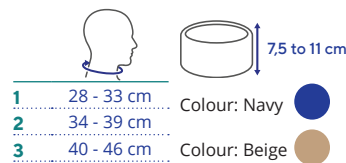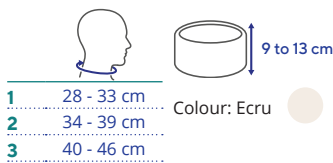

Ortel C3


Stiff cervical collar
239102



Indicated for
Compression of the cervical spine for a strengthening and analgesic result (moderate and benign sprains, cervicobra-

POINTS TO REMEMBER

- Reinforced support:** rigid perforated polyethylene structure
- Comfort:** padding of the edges in contact with the skin, perforations for better ventilation, chin support
- Hold:** self-fastening adjustment
- Anatomical fit:** height-adjustable from 9 to 13 cm

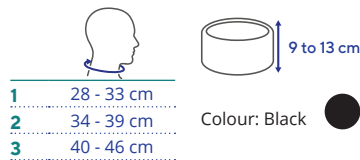

Ortel T-Neck

Collar brace for reinforced immobilisation
T49100



Indicated for
Radicular pain in the cervical spinal region
Severe whiplash injury
Moderate to severe cervical syndrome
Inflammations
Stable, non-dislocated vertebral body fractures (without neurological symptoms).

POINTS TO REMEMBER

- One hand adjustment system** for enhanced ease of use for the patient
- Anatomically shaped** with rigid frame to provide strong support and reinforced immobilisation of the cervical vertebra
- Five-steps chin** height for precise adjustments
- Light weight and comfortable foam pads** to deliver an increased comfort
- Modern and sober** design

Single size
Colour: ●

Ortel T-Neckplus

Collar brace for reinforced immobilisation with occipital support
T49105



Indicated for
Preoperative / Postoperative / Post-traumatic
Inflammatory disease
Degeneration
Severe instabilities of the cervical spine
Stable, non-dislocated vertebral body fractures (without neurological symptoms)

POINTS TO REMEMBER

- Sternal extension** for reinforced occipital immobilisation and stabilization
- One hand adjustment** system for enhanced ease of use for the patient
- Anatomically shaped brace** with rigid frame to provide strong support and reinforced immobilisation of the cervical vertebra
- Five-steps chin** height for precise adjustments
- Light weight and comfortable foam pads** to deliver an increased comfort
- Modern and sober** design

Single size
Colour: ●



SHOULDER - ELBOW

Did you know?



14 to 20%
of the population

say they are suffering from
shoulder pain on any given day¹

The shoulder is one of the areas of the body most affected by repetitive strain injuries (RSI), particularly rotator cuff tendinitis.

Collar bone fracture:

Children are the most affected

(almost half of all collar bone fractures), followed by men under the age of 30, due to sports injuries²

SHOULDER IMMOBILISERS & ARM SLINGS: KEY BENEFITS

Thuasne shoulder braces can be modular, depending on the models, and accompany patients from the post-traumatic or post-operative immobilisation phase right through to rehabilitation and recovery. There are two different types of immobilisation:

- 1 Immobilisation in neutral position ("Elbow-to-body")** following trauma or surgery
- 2 Functional immobilisation** for rehabilitation and gradual resumption of shoulder movement and/or function

The physician decides on the immobilisation duration on the basis of the severity of the trauma, its nature and the type of associated damage.

Prolonged strict immobilisation can have significant consequences, such as stiffness. It is for this reason that the initiation of a gradual, adapted rehabilitation programme designed to restore pain-free, unrestricted function is recommended*.

(1) T.Marc, T.Gaudin, J.Teissier, & F.Bonnel. (s.d.). "Pathologie de la coiffe des rotateurs et TMS. Le muscle Nouveaux concepts. Anatomie - Biomécanique - Chirurgie - Rééducation".
(2) Fracture de la clavicule. (s.d.). Site web : <http://www.physiotherapiepourous.com/fracture/la-fracture-de-la-clavicule/>

*HAS, (French National Authority for Health), professional guidelines - Criteria for rehabilitation and referral follow-up in the outpatient or follow-on or rehabilitative care setting after rotator cuff rupture surgery and shoulder joint replacements - 2008.

ImmoClassic

Scapulo-humeral immobilisation

2445 01



Indicated for

Shoulder and/or elbow injuries or inflammatory conditions.
Conservative treatment for non-surgical proximal humeral fractures.
Shoulder dislocation.
Post-operative immobilisation (such as after rotator cuff surgery or dislocation reduction...),
Shoulder subluxation or dislocation prevention (joint instability).
Support for paralytic upper limb.

POINTS TO REMEMBER

Scapulo-humeral immobilisation: sleeve and additional strap to support the forearm and shoulder

Adaptability: adjustable upper strap

Comfort: protective foam on the shoulder and neck

	HEIGHT
1	50 - 71 cm
2	72 - 94 cm
3	95 - 117 cm
4	118 - 140 cm

Colour: Black



ImmoVest

Scapulo-humeral immobilisation vest

2441 02



Indicated for

Uncomplicated shoulder injuries:
- anterior dislocation of the shoulder,
- non-displaced, non-articular fracture of the scapula,
- uncomplicated acromioclavicular dislocation ("separated shoulder").
Immobilisation of the elbow following dislocation.
Fractures of the greater tuberosity of the humerus, humeral head or lesser tuberosity of the humerus (non-surgical indications).
Immobilisation following surgery to correct instability.
Rotator cuff rupture in the acute phase.
Acute scapulo-humeral peri-arthritis ("frozen shoulder").
Paralysis of the brachial plexus.
Re-education of the scapulo-humeral joint.
Pain-relieving immobilisation of the shoulder after the inflammatory phase (acute rheumatism, rheumatoid arthritis).

POINTS TO REMEMBER

Immobilisation of the shoulder and elbow: in neutral position thanks to the 3 straps

Reinforced protection: high coverage vest

Comfort: anatomically shaped breathable fabric and comfort pad on the shoulder

Easy to fit: numbered and coloured marking system

HEIGHT	CORPULENCE		
< of 1.70 m	Size 1		Size 2
1.70 - 1.80 m	Size 1		Size 2
> of 1.80 m	Size 1		Size 2

Colour: Black




Fitting video available online

Immo

Immobilisation vest with forearm and hand bands

2440 01

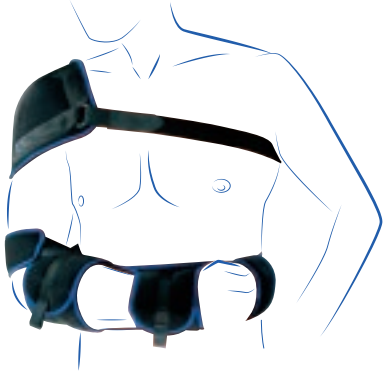

Indicated for

Analgesic immobilisation of the shoulder, arm and forearm following:
 Non-surgical shoulder and elbow injuries (dislocations or sprains).
 Arm muscle injuries.
 Non-surgical shoulder and elbow bone injuries.
 Shoulder tendon injuries.

POINTS TO REMEMBER

Immobilisation of the elbow against the torso through its reinforced belt and support bands

Better comfort thanks to the belt that supports the arm, **no pressure on the shoulder and neck**

Easy for the patient to put on with its **self-fastening** attachment



1	50 - 71 cm
2	72 - 94 cm
3	95 - 117 cm
4	118 - 140 cm


Colour: Black



Ligaflex® clavicular straps

Clavicular immobilisation strap

2450 02

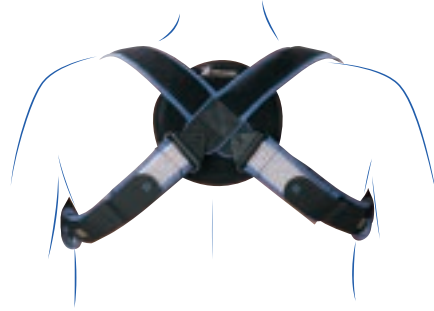

Indicated for

Displaced or non-displaced clavicle fractures.

POINTS TO REMEMBER

Adaptability: one size fits all with trimmable straps to fit all morphologies

Comfort: dorsal foam anchoring pad and quilted straps

Easy to fit: buckle closure

Single size / Colour: Black





Silistab® Epi

Anti-epicondylitis brace

2305 03

New design



Indicated for

Insertion epicondylitis (tennis elbow) and epitrochleitis (functional treatment).

Prevents against future recurrence and gives active support for sports activities.

Ideal for manual work situations.

POINTS TO REMEMBER

Proprioception: anatomical compressive knit and adjustable pressure thanks to the additional removable strap

Comfort: silicone inserts and anatomical knit with comfort areas on the epicondyles

Easy to fit: visual markers at the hollow and tip of the elbow for precise positioning.

Practical: pull tabs for ease of application



8 cm



Universal

1	19 - 22 cm
2	22 - 25 cm
3	25 - 28 cm
4	28 - 31 cm
5	31 - 34 cm
6	34 - 37 cm

Colour: Grey/Dark Blue



Fitting video available online

Epi-Med

Rigid armband

T480013



Indicated for
 Inflammatory pathologies
 Epicondylitis
 Epitrochleitis

POINTS TO REMEMBER

Comfort: felt lining for targeted pressure around the epicondyles
Easy adjustment: pressure adjustment thanks to a self-fastening closure



5 cm



Universal

XS 22 - 23 cm
S 24 - 25 cm
M 26 - 27 cm
L 28 - 30 cm
XL 31 - 34 cm

Colour: Grey



Epi-Med PROMASTER

Preventive rigid armband with silicone pads

T35087



Indicated for
 Inflammatory pathologies (Prevention of recurrence):
 Epicondylitis
 Epitrochleitis

POINTS TO REMEMBER

Comfort: silicone inserts for targeted double pressure around the epicondyles and additional inserts included (Flexi Pads) to adjust the pressure
Easy adjustment: pressure adjustment thanks to a self-fastening closure system with immobilisation indication area



8 cm

XS 22 - 23 cm
S 24 - 25 cm
M 26 - 27 cm
L 28 - 30 cm
XL 31 - 34 cm

Version : Universal
 Colour: Grey





WRIST-THUMB

FUNCTION OF SPLINTS



WRIST SPLINT

Varying degrees of immobilisation of the carpometacarpal joints. Used mainly for sprains, wrist instabilities, tendinitis of the wrist, carpal tunnel syndrome that has not undergone surgery and during the post-operative phase.



WRIST/THUMB SPLINT

Varying degrees of immobilisation of the carpometacarpal joints and the thumb. Used in the event of trauma (sprains and fractures), wrist and thumb instabilities, tendinitis of the wrist and De Quervain's tenosynovitis, basal joint arthritis of the thumb and during the post-operative phase.



THUMB SPLINT

Stabilisation of the thumb in the required therapeutic position. Mainly used in basal joint arthritis of the thumb or thumb sprains.



WRIST-FINGER SPLINT

Varying degrees of immobilisation of the metacarpophalangeal and interphalangeal joints. Used in the event of sprains, dislocations, non-displaced fractures or in the post-operative phase.

Did you know?

Carpal tunnel syndrome, due to median nerve compression, represents almost all repetitive strain injuries of the wrist⁽¹⁾. It is caused by repeated flexion/extension movements, tight gripping, pressure on the palm and vibrations (vibrating tools), etc...



1/4 pregnant women

are affected by carpal tunnel syndrome. In this case, the trigger is fluid retention, causing the tissues to swell and compress the median nerve.



130,000

is the number of wrist fractures every year, often due to a fall⁽²⁾.



Sport is also one of the biggest causes of injury. For example, 3/4 of all handball injuries affect the fingers⁽³⁾.

(1) Dr Guéniot, C. (2009, février). Le poignet victime de la crise. Consulted on Doctissimo: http://www.doctissimo.fr/html/sante/mag_2002/sem01/mag0308/dossier/sa_5162_canal_carpien_poignet.htm.

(2) Allodocteurs. (2016, octobre). Chutes : attention à la fracture du poignet. Consulted on Allodocteurs: https://www.allodocteurs.fr/maladies/os-et-articulations/fractures/chutes-attention-a-la-fracture-du-poignet_114.html

(3) Knobloch K, Rossner D, Jagodzinski M, Zeichen J, Gössling T, Martin-Schmitt S et al. Prevention of school sport injuries-an analysis of ballsports with 2234 injuries. Sportverletz Sportschaden 2005; 19:82-88

Ligaflex® Action

Flexible wrist stabilising brace with removable palmar stay

2436 02



Indicated for

Wrist sprain.
Post-operative, post-traumatic residual pain and/or oedema.
Wrist tendinopathies.
Wrist osteoarthritis.

POINTS TO REMEMBER

Modular support: conformable and removable palmar stay

Proprioception: Compressive knit and additional removable strap to increase the pressure in the pain area

Comfort: silicone insert on the ulnar condyle. Anatomical and elastic knitting



1	14 - 15 cm
2	15 - 16 cm
3	16 - 17 cm
4	17 - 18 cm
5	18 - 19 cm
6	19 - 20 cm

Colour: Grey ●

Ligaflex® Classic

Wrist immobilisation splint

2435 02



Indicated for

Wrist sprain.
Non-surgical wrist fracture.
Post-operative or post-traumatic wrist immobilisation.
Wrist tendinopathies.
Conservative treatment of carpal tunnel syndrome.
Lateral epicondylitis.
Rheumatoid arthritis.
Wrist osteoarthritis.

TARGET PATIENT

Patient without oedema

POINTS TO REMEMBER

Strict immobilisation of the joint: rigid, anatomical and conformable palmar stay

Comfort: soft and breathable 3D knit for prolonged wear



0	10 - 13 cm
1	13 - 15 cm
2	15,5 - 17,5 cm
3	18 - 20 cm
4	20,5 - 23 cm

Colour: Black ●

Ligaflex® Manu

Wrist and thumb immobilisation splint

2430 02



Indicated for

Thumb sprain.
Wrist and/or thumb sprain.
Non-surgical scaphoid fracture.
Non-surgical wrist fracture.
Post-operative or post-traumatic wrist/thumb immobilisation.
Wrist and/or thumb tendinopathies.
De Quervain tenosynovitis.
Rheumatoid arthritis.
Lateral epicondylitis.
Conservative treatment of carpal tunnel syndrome.
Thumb/wrist osteoarthritis.
Neuro-muscular impairments (contracture, hand drop or spasticity..).

POINTS TO REMEMBER

Immobilisation of the wrist and thumb via 2 rigid, malleable and anatomical palmar stays

Aerated, foam lined fabric

Dual strap system ensuring even compression




0	10 - 13 cm
1	13 - 15 cm
2	15,5 - 17,5 cm
3	18 - 20 cm
4	20,5 - 23 cm

Colour: Black ●

Ligaflex® pro

Wrist immobilisation splint with quick lacing system

2433 02

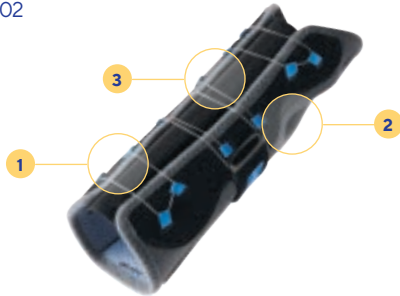

Indicated for

Wrist sprain.
Non-surgical wrist fracture.
Post-operative or post-traumatic wrist immobilisation.
Wrist tendinopathies.
Conservative treatment of carpal tunnel syndrome.
Lateral epicondylitis.
Rheumatoid arthritis.
Wrist osteoarthritis.



POINTS TO REMEMBER

- 1 - Easy and quick to put on:** Quick lacing system
- 2 - Firm support:** conformable palmar stay on the palm of the hand
- 3 - Comfortable and breathable:** soft, protective jersey lining and breathable 3D knit

Universal



Bilateral

- 1 13 - 17 cm
- 2 17 - 23 cm

Colour: Grey



Extra tip!

Relieve pain, sprains, tendinitis...

Apply **ARGICALM®**

Ligaflex® pro +

Wrist and thumb immobilisation splint with quick lacing system

2434 02



Indicated for

Thumb sprain.
Wrist and/or thumb sprain.
Non-surgical scaphoid fracture.
Non-surgical wrist fracture.
Post-operative or post-traumatic wrist/thumb immobilisation.
Wrist and/or thumb tendinopathies.
De Quervain tenosynovitis.
Rheumatoid arthritis.
Lateral epicondylitis.
Conservative treatment of carpal tunnel syndrome.
Thumb/wrist osteoarthritis.
Neuro-muscular impairments (contracture, hand drop or spasticity...).

POINTS TO REMEMBER

- Easy and quick to put on:** Quick lacing system
- Firm support for thumb and wrist:** 2 conformable palm stays
- Comfortable and breathable:** soft, protective jersey lining and breathable 3D knit
- 2 models:** right and left



Left



Right

- 1 13 - 15 cm
- 2 15,5 - 17,5 cm
- 3 18 - 20 cm
- 4 20,5 - 23 cm

Colour: Grey



Extra tip!



Basal joint arthritis

For night-time use, recommend Ligaflex® pro + to reduce muscle tension and relieve pain.



Presentation video available online

Ligaflex® Rhizo Pro

Thumb splint

T4802732



Indicated for

Thumb sprain.
Immobilisation in case of pain or trauma of the thumb.
Post-operative immobilisation.

POINTS TO REMEMBER


Efficacy of the thumb immobilisation: rigid plastic shell over the thumb

Comfort: perforated shell with soft ventilated fabric lining

Ease of use: easy to put on with wide strap at the wrist

Universal product

1	13 - 17 cm
2	17 - 23 cm

Colour: Black 

Ligaflex® Rhizo

Static thumb splint in analgesic position

7090 02



Indicated for

Immobilisation in case of pain or trauma of the thumb.
Post-operative immobilisation.
Thumb osteoarthritis (including rhizarthrosis).

POINTS TO REMEMBER


1 - Immobilisation in a painless position: metal frame

2 - Comfort: opening at the thumb to prevent any pressure on the painful zone

3 - Freedom of movement: preserves gripping and pinching movements and range of motion of the wrist

4 - Breathable: breathable 3D knit


2 models: right and left

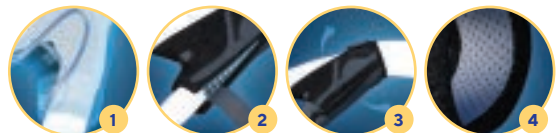

Left

Right

1	13 - 15 cm
2	17 - 23 cm

Colour: Grey 



Presentation video available online



Extra tip!




Rhizarthrosis

For daytime use, recommend Ligaflex® Rhizo to reduce muscle tension and relieve pain.

Ligaflex Finger®

Wrist and finger immobilisation splint 2nd, 3rd or 4th metacarpals


2432 02

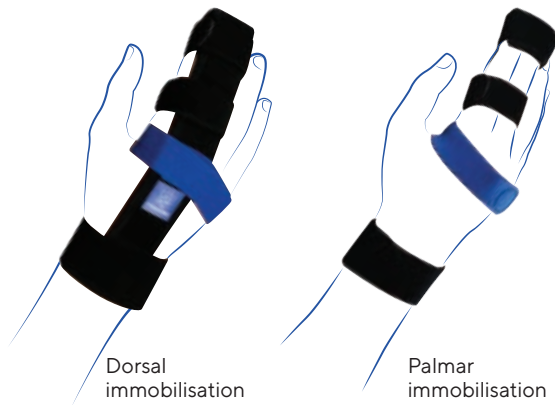

Indicated for

Finger injuries (sprains, dislocations, ligament ruptures...)
Post-traumatic immobilisation.

POINTS TO REMEMBER

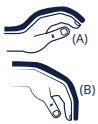

Modular immobilisation thanks to conformable stays for:

- Immobilisation of metacarpal-phalangeal joints (A)
 - Immobilisation of interphalangeal joints (B)
- Adaptability and support** 4 straps (2 fixed and 2 removable) and 2 immobilisation possibilities (palmar or dorsal) according to the prescriber's recommendations

Comfort: Protected injured fingertips and specific cutting of the wrist strap



- < 13 cm
- 13 - 16 cm
- 16 - 19 cm
- 19 - 22 cm

Colour: Black/Dark Blue



Presentation video available online

Ligaflex Boxer®

Immobilisation splint wrist, hand, fingers 4th or 5th metacarpals

2431 02



Indicated for

Non-surgical 4th and 5th metacarpal fracture.
Post-operative 4th and 5th metacarpal support.
Residual symptoms post fractured or sprained finger (of 4th and 5th).
Following strict plaster cast immobilisation

POINTS TO REMEMBER

Strict immobilisation of the wrist up to the 4th and 5th fingers: 2 conformable and removable stays with specific angulation (dorsal at 90° and palmar at 20°)

Comfort: Lightweight and breathable 3D fabric for prolonged wear. Protection of immobilised fingers thanks to interdigital separation

Easy to fit: full opening



- 13 - 15 cm
- 15,5 - 17,5 cm
- 18 - 20 cm
- 20,5 - 23 cm

Colour: Black/Dark Blue



Presentation video available online




KNEE

FUNCTION OF KNEE BRACES AND KNEE SPLINTS




PROPRIOCEPTIVE KNEE BRACE

Improved proprioception via the use of a compressive knit

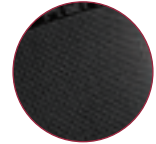

PATELLAR KNEE BRACE

Patella supported by the ring and proprioceptive effect thanks to the compressive knit

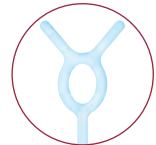

PATELLO-FEMORAL KNEE BRACE

Patella realignment in its physiological axis and proprioceptive effect thanks to compressive knit



LIGAMENT KNEE BRACE

Stabilizes the knee via the use of hinged reinforcements



OFFLOADING KNEE BRACE

Offload of the damaged compartment thanks to a 3-point system



KNEE SPLINT

Pain management and protective effect due to complete immobilisation of the knee



Did you know?

Knee injuries are often linked to sports.

For example, 1/3 of skiing injuries involve the knee¹

30% of arthritis cases concern the knee joint²

(1) "L'accidentologie des sports d'hiver. Association des médecins de montagne". Winter 2012-2013.
(2) Société Française de Rhumatologie. (2005). Report on osteoarthritis.

KNEE | Which knee brace for which condition?

Which prescription?

KNEE BRACE

Which indication?

Mild indeterminate pain

Tendinitis, dislocation of the patella, mild osteoarthritis of the knee

Patellar syndrome (or patellofemoral syndrome), patellofemoral arthritis

FLEXIBLE KNEE BRACE
with a proprioceptive effect*



PATELLAR KNEE BRACE



PATELLOFEMORAL KNEE BRACE



Limited activity



Moderate activity



Intense activity



Limited activity



GenuSoft



page
46

GenuAction



page
46

GenuExtrem



page
46

Silistab® Genu




page
48

PatellaReliever



page
49

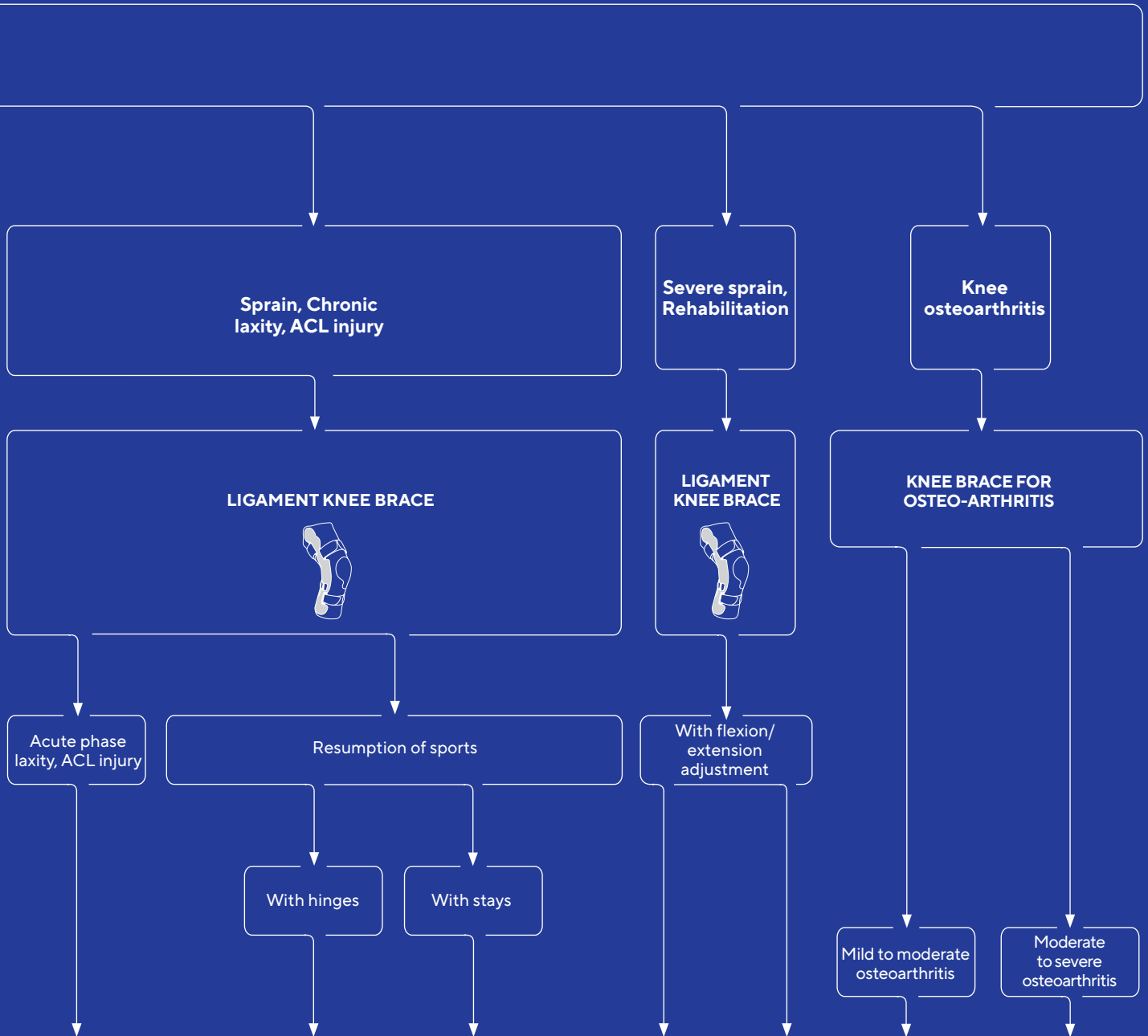

Genu Ligaflex® long



page
50

Genu Ligaflex® short



page
50

Ligastrap® Genu



page
47

Genu Ligaflex® ROM



page
50

Rebel Standard



page
52

ActionReliever



page
56

Rebel Reliever®



page
58

Find all our products on the following pages.

GenuSoft

Proprioceptive elastic knee support (15 – 20 mmHg)

2320 03



Indicated for
Knee pain and/or swelling management.
Proprioceptive support for rehabilitation.

Target patient
Patient with limited activity

POINTS TO REMEMBER

Comfortable to wear: anatomically-shaped knit with comfort zone behind the knee

Proprioception: elastic compressive knit (15 – 20 mmHg)



1	32 - 34 cm
2	35 - 37 cm
3	38 - 41 cm
4	42 - 45 cm
5	46 - 49 cm

Colour: Sliver ●

GenuAction

Proprioceptive elastic knee support (20 – 36 mmHg)

2325 03



New design

Indicated for
Joint instability/laxity.
Proprioceptive support for rehabilitation

Target patient
Patient with moderate activity

POINTS TO REMEMBER

Comfortable to wear: anatomically-shaped knit with comfort zone behind the knee and over the kneecap

Proprioception: elastic compressive knit (20 – 36 mmHg)



0	28 - 31 cm
1	32 - 34 cm
2	35 - 37 cm
3	38 - 41 cm
4	42 - 45 cm
5	46 - 49 cm
6	50 - 53 cm

Colour: Grey ●

GenuExtrem

Proprioceptive elastic knee support with lateral reinforcements (20 – 36 mmHg)

2321 02



New design

Indicated for
Protection and prevention during sports activities.
Proprioceptive support for rehabilitation.

Target patient
Patient with intense activity

POINTS TO REMEMBER

1 - Comfortable: anatomically-shaped knit with comfort zones behind the knee and over the kneecap

2 - Support and hold: flexible side stays with anti-slip system

3 - Practical: pull tabs for ease of application

4 - Proprioception: elastic compressive knit (20 – 36 mmHg)



1	32 - 34 cm
2	35 - 37 cm
3	38 - 41 cm
4	42 - 45 cm
5	46 - 49 cm
6	50 - 53 cm

Colour: Grey ●

Ligastrap® Genu

Ligament knee brace
(20 – 36 mmHg) with
functional strapping

2170 03



Indicated for

Joint instability/laxity.
Knee pain and/or swelling management.
Sport activities resumption.

Target patient

Resumption of sports, sport not authorised with rigid stays

POINTS TO REMEMBER

Reinforced support: double strapping system and flexible side stays

Comfortable to wear: anatomically-shaped knit with comfort zones behind the knee and over the kneecap

Practical: pull tabs and fitting guidance on the straps

Proprioception: elastic compressive knit (20 – 36 mmHg)



1	32 - 34 cm
2	35 - 37 cm
3	38 - 41 cm
4	42 - 45 cm
5	46 - 49 cm
6	50 - 53 cm

Colour: Grey ●



Presentation video available online




Knee



Silistab® Genu

Knee brace with shock-absorbing patellar guide (15 – 20 mmHg)

2340 05 (white) - 2345 04 (black)

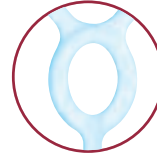

Patellar insert



Indicated for

Knee pain and/or swelling management.
Patella pain and instabilities.
Knee osteoarthritis (mild to moderate).

Target patient

Patient with limited activity
Patient with mild knee osteoarthritis

POINTS TO REMEMBER

Patella support: anatomical insert

Joint support and hold: flexible side stays with anti-slip system

Comfort: anatomically shaped knit, open behind the knee

Proprioception: elastic compressive knit (approximately 17 mmHg)



Product height
31 to 35 cm

- | | |
|---|------------|
| 1 | 31 - 33 cm |
| 2 | 34 - 36 cm |
| 3 | 37 - 39 cm |
| 4 | 40 - 42 cm |
| 5 | 43 - 45 cm |
| 6 | 46 - 48 cm |

White
with open kneecap

Black
with closed kneecap

Patellar bandage Preventive orthotic

2304 01



Indicated for

Prevention of tendinitis of the patellar tendon

Target patient

Patient taking part in sports with risk of recurrence

POINTS TO REMEMBER

Patella support: anatomical shape applying local pressure over the patellar tendon

Proprioception: silicone with dimpling for a massage effect



Unique size 26 - 50 cm

Colour: Black



PatellaReliever

Hinged knee orthosis with patella traction system

2348 01



Patellar insert



Indicated for

Femoro-patellar pain syndrome and/or patella instability.
Femoro-patellar osteoarthritis (mild to moderate).
Anterior knee or patella pain.
Patella luxation and subluxation.

Target patient

Patient with pain behind the patella, for example after relatively prolonged sitting, when going down steps

POINTS TO REMEMBER

Patella realignment: half-moon patellar insert with strap and lace kneecap traction system

Joint support: hinged stay

Hold: silicone integrated into the knit

Comfort: anatomically shaped knit with comfort zones

Practical: pull tabs for ease of application

2 models: right – left



1	32 - 34 cm
2	35 - 37 cm
3	38 - 41 cm
4	42 - 45 cm
5	46 - 49 cm
6	50 - 53 cm

Colour: Grey/Blue



Presentation video available online

Genu Dynastab®

Hinged ligament knee brace

2370 06



Indicated for

Conservative treatment of knee ligament injuries (lateral ligaments).
Joint instability/laxity.
Sport activities resumption.

Target patient

Resumption of sports, patient requiring moderate support

POINTS TO REMEMBER

Joint support: bi-axial hinge

Hold: silicone integrated into the knit

Comfort: anatomically shaped knit with comfort zones and half-straps to prevent any tourniquet effect

Practical: pull tabs for ease of application



1	32 - 34 cm
2	35 - 37 cm
3	38 - 41 cm
4	42 - 45 cm
5	46 - 49 cm
6	50 - 53 cm

Colour: Grey

Genu Ligaflex®

Ligament knee brace with TM5 hinge

2375 06 (closed) - 2375 05 (open)



Indicated for

Conservative treatment of knee ligament injuries.
Joint instability/laxity.
Knee pain and/or swelling management.
Sport activities resumption.

Target patient

Resumption of sports, patient requiring strong support

POINTS TO REMEMBER

Joint support: TM5 hinge reproducing the knee's natural movements*

Hold: silicone integrated into the knit and suspension strap above the calf

Comfort: anatomically shaped knit with comfort zones and half-straps to prevent any tourniquet effect

Practical: pull tabs and different versions to adapt to requirements (short and long version, open or closed model)



CLOSED VERSION		
	1	32 - 34 cm
	2	35 - 37 cm
	3	38 - 41 cm
	4	42 - 45 cm
	5	46 - 49 cm
	6	50 - 53 cm

Colour: Grey



Closed available in short only. Open available in short and long.



Presentation video available online

Genu Ligaflex® ROM

Ligament knee brace with flexion/extension control

2386 05 (open)



Indicated for

Conservative treatment of knee ligament injuries and/or ruptures (cruciate and/or lateral ligaments).
Post-operative rehabilitation.
Joint instability/laxity.

Target patient

Patient following severe sprain or during rehabilitation

POINTS TO REMEMBER

Joint support: TM5+ hinge reproducing the knee's natural movements*

Range of motion control: tool-less limitation of flexion and extension with extensive adjustment range

Hold: silicone integrated into the knit and suspension strap above the calf

Comfort: anatomically shaped knit with comfort zones and half-straps to prevent any tourniquet effect

Flexion/Extension adjustment ranges

Flexion	0°	30°	45°	60°	75°	90°	-
Extension	0°	5°	10°	15°	20°	30°	40°



1	32 - 34 cm
2	35 - 37 cm
3	38 - 41 cm
4	42 - 45 cm
5	46 - 49 cm
6	50 - 53 cm

Colour: Grey



Presentation video available online

Ligaflex® Immo

Modular immobilisation splint to fit all body shapes

2400 04



Indicated for

Knee pain management.
Post-operative and post-traumatic immobilisation.

Target patient

Difficult-to-fit patients

POINTS TO REMEMBER

Immobilisation: 4 conformable stays

Hold: anti-slip system and tightening straps

Practical: 3 mobile side panels to fit most calf and thigh circumferences



1	50 - 60 cm
2	60 - 70 cm
3	70 - 80 cm
4	> 80 cm

Colour: Navy



Extra tip!

For the resumption of activities, recommend **Ligastrap® Genu** to protect the joint!

ROM-R

Post-operative splint with flexion/extension limitation

U04602



Indicated for

Post-operative immobilisation or/and rehabilitation.
Post-traumatic immobilisation.
Joint instability/laxity.
Conservative treatment of knee ligament injuries and/or ruptures

Target patient

Patient with a severe ligament injury or post-operatively

POINTS TO REMEMBER

Immobilisation: 2 conformable lateral frames

Tool-less adjustment of the articulation, by 15° steps in flexion (from 0° to 120°) and extension (from 0° to 105°)

DropLock system can be locked by **15° steps from 0° to 90°**

Tool-less length adjustment (proximal and distal)

Easy to use buckles

Flexion/Extension adjustment ranges

Limitations flexion	0°	15°	30°	45°	60°	75°	90°	105°	120°
Limitations extension	0°	15°	30°	45°	60°	75°	90°	105°	
Total immobilisation	0°	15°	30°	45°	60°	75°	90°		



Product height
48 to 66 cm

SINGLE SIZE

Colour: Black



Presentation video available online

Rebel Standard Ligament

Off The Shelf

Patients who have experienced ligament injuries and/or reconstructive surgery. For rehabilitative or prophylactic use.

Features

- Stock sizes see chart
- Enhanced brace positioning via Townsend Motion TM5+ Hinge- tracks with the anatomical roll-and-glide motion of the knee to keep the brace properly positioned on the leg.
- Superior suspension via synergistic suspension strap
- Improved suspension via C/S package
- Rotational control via tibial shell contour
- Suspension augmentation via anti-migration straps
- Easy donning via clip buckles
- Suitable for the majority of patients
- ACL configuration



Anti-migration Technology



C/S Package
(Compression/Suspension)



Synergistic Suspension Strap

Size	6" Above	M-L Knee Width	6" Below
XS	12.5 - 15.5"	3 - 3.5"	11 - 12.5"
SM	15.5 - 18.5"	3.5 - 4"	12.25 - 13.25"
MD	18.5 - 21"	4 - 4.5"	13.25 - 15"
LG	21 - 23.5"	4.5 - 5"	14.25 - 15.75"
XL	23.5 - 25"	5 - 5.5"	15 - 17"
XXL	25 - 28"	5.5 - 6"	17 - 19"
XXXL*	28 - 31"	6 - 6.5"	19 - 21"

*XXXL available on Rebel Pro only. See Rebel Pro datasheet

Rebel PRO Ligament Custom

Patients who have experienced ligament injuries and/or reconstructive surgery. For rehabilitative or prophylactic use.

Features

- Optimised & customised fit - Custom assembled to 3 leg measurements
- Enhanced brace positioning via Townsend Motion TM5+Hinge- tracks with the anatomical roll-and-glide motion of the knee to keep the brace properly positioned on the leg.
- Superior suspension via synergistic suspension strap
- Improved suspension via C/S package
- Rotational control via tibial shell contour
- Suspension augmentation via anti-migration straps
- Easy donning via clip buckles
- Suitable for heavy users and contact sports- rigid aircraft aluminium frame

Options

- Extension assist kit- suitable for patients who present with quadriceps weakness or fatigable quadriceps
- Frame height options
- Frame configuration options
- Flexion Stop Kit
- Range of colours



Anti-migration Technology



C/S Package
(Compression/Suspension)



Synergistic Suspension Strap

Powdercoat Colours



To order a Rebel Pro please simply complete form (please see page 92)

Rebel Lock

Off The Shelf

Product Code:

Standard - U30202

Semi-Custom - U30201

Indications

- Patients with significant instabilities; weak extensor muscles, failed TKR, limiting or preventing knee flexion.
- A lightweight option for patients who need stability and functionality to control significant instabilities.

Features

- Superior suspension via synergistic suspension strap
- Improved suspension via C/S package
- Rotational control via tibial shell contour
- Suspension augmentation via anti-migration straps
- Lightweight Aircraft Aluminium Frame
- Drop Lock With Free Motion
- Adjustable Locking Position

Options

- Optimised & customised fit - Custom assembled to 3 leg measurements
- Extension assist kit - suitable for patients who present with quadriceps weakness or fatiguable quadriceps
- Frame height options
- Flexion stops at 30, 60, or 90 degrees
- Range of colours (stock frame colour is black)



Extension Assist



This optional extension assistance mechanism uses strong latex bands slipped over two posts – one proximal and one distal – to the knee centre. There is no limitation in flexion.

Size	6" Above	M-L Knee Width	6" Below
XS	12.5 - 15.5"	3 - 3.5"	11 - 12.5"
SM	15.5 - 18.5"	3.5 - 4"	12.25 - 13.25"
MD	18.5 - 21"	4 - 4.5"	13.25 - 15"
LG	21 - 23.5"	4.5 - 5"	14.25 - 15.75"
XL	23.5 - 25"	5 - 5.5"	15 - 17"

Powdercoat Colours



To order a Rebel Lock please simply complete form (please see page 93)

Sprystep Knee Orthosis (KO)

Custom

Strong, thin and lightweight. Custom knee orthoses that are elegant and functional.

Features

- The Sprystep Knee Orthosis leverages advanced composite materials to provide elegant and high quality solutions.
- Varied geometry options dependent upon the biomechanical aim
- Moderate stiffness with the option to reinforce.
- Anatomical contours for comfort on force pattern application
- Extremely lightweight and thin
- Full range of Townsend joints are applicable. Requests can be made for TM5, Single Pivot, Five Bar with accessories such as trigger release, twist cable or bump and range stops.



Customisation | Shell Configuration



Chevron-Shaped Tibia



Posterior Tibia



Full Shell Option & Flexible Edges Option


Knee Component



TM5



Five Bar

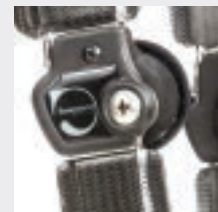

Single Pivot

*For any changes to default specification such as shell configuration or joint choice, please add to notes to manufacturer (please see page 94)

ActionReliever

Off-loading knee brace for the treatment
of knee osteoarthritis

2349 01



Indicated for

Symptomatic unicompartmental femorotibial osteoarthritis (mild to moderate).

Knee off-loading for post-traumatic, post-operative or degenerative conditions.

Knee pain and/or swelling management.

Target patient

Patient with mild to moderate pain.

Patient with moderate activity


POINTS TO REMEMBER

Offloading of the injured compartment: 3-point support system thanks to inelastic straps that ensure maximum off-loading of the knee in extension

Stabilisation of the joint: rigid joint on the injured side and flexible on the healthy side and patellar silicone insert

Support and comfort: silicone integrated into the knit and anatomical knitting with comfort areas

Available in medial left / lateral right and medial right / lateral left

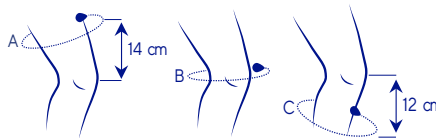

Left Leg Medial
or
Right Leg Lateral
Compartment

Right Leg Medial
or
Left Leg Lateral
Compartment

Size	A	B	C	Code	Code
1	38 - 41 cm	32 - 34 cm	28 - 31 cm	23490120100151	23490120100152
2	42 - 45 cm	35 - 37 cm	32 - 35 cm	23490120200151	23490120200152
3	45 - 48 cm	38 - 41 cm	35 - 38 cm	23490120300151	23490120300152
4	48 - 51 cm	42 - 45 cm	39 - 42 cm	23490120400151	23490120400152
5	50 - 53 cm	46 - 49 cm	41 - 44 cm	23490120500151	23490120500152
6	53 - 56 cm	50 - 53 cm	44 - 47 cm	23490120600151	23490120600152
7	56 - 61 cm	53 - 56 cm	48 - 51 cm	23490120700151	23490120700152
8	61 - 66 cm	56 - 59 cm	51 - 54 cm	23490120800151	23490120800152

Colour: Grey/blue




ActionReliever

Three point Pressure System

The Thuasne ActionReliever is an off-loading brace that has been designed to apply corrective forces on the leg to reduce the load of the compartment affected by osteoarthritis (referred to as three point pressure).

By helping hold the leg in a normal alignment and maintaining a homogeneous load on the knee joint as a whole, the ActionReliever knee brace will help reduce compression and inflammation. This load reduction is proven to relieve pain.

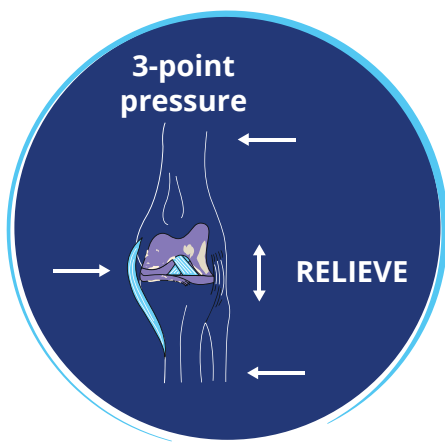

POINTS TO REMEMBER

Dynamic Off-Loading

- Straps creating a dynamic three point pressure system: most effective on heel strike.
- Combined rigid adjustable hinge on the affected side and flexible stay on the unaffected side.
- Self adjustment possible by tightening the front straps.
- For medial or lateral osteoarthritis.

Ease Of Use

- Only two steps for fitting.
- Easy to put on and take off.

Comfort


- Anatomically shaped straps, self-oriented and adjustable.
- Anatomical knitting with no compression areas at popliteal crease and at brace edges.
- Stays comfortable all day.
- Slim and discreet under clothing.

Patients with Knee Osteoarthritis

Randomised Controlled Study with 32 Patients on 8 Weeks:
17 bracing group, 15 control group*

Main Objective:

Effect of the brace on the pain free walking distance

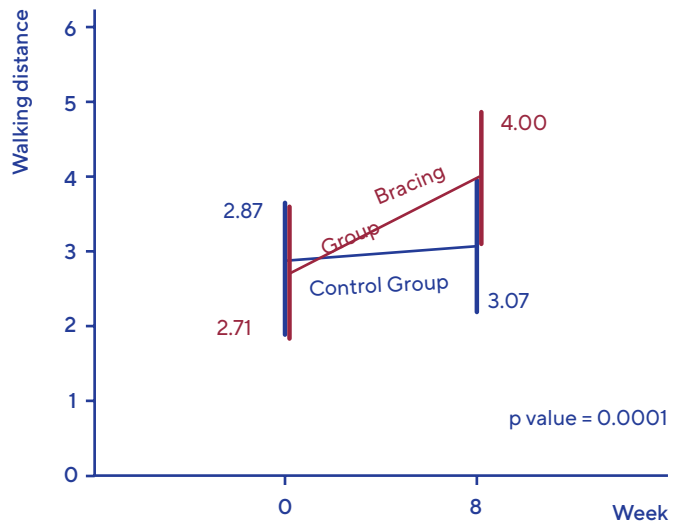

"By wearing the brace, the pain on exertion was reduced and the pain-free distance extended"*

*Benning M, Schneider-Nieskens R; Superiority of a Knee Relief Orthosis in the Treatment of Kne Osteoarthritis. Orthopädie Technik 08/17: 24-30

The following Action Reliever videos are available on [YouTube](#) by scanning the QR code: testimonial, measuring, practitioner fitting, patient fitting.



Presentation video available online

Rebel Reliever®

Off-loading knee brace

U03309

Indicated for

Moderate to severe unicompartement femoral-tibial osteoarthritis of the knee.

Unicompartement knee joint degeneration requiring offloading. Post-operative offloading of a single compartment (articular or meniscus cartilage defect or osteochondral defect).

Knee joint instability after injury or surgery on the cruciate and/or collateral ligaments (ACL, PCL, LCL, MCL).

Target patient

Arthritic patient requiring maximum offloading

POINTS TO REMEMBER

Offloading of the damaged compartment: 3-pressure point system with Loadshifter system

Freedom of movement: TM5+ hinge reproducing the knee's natural movements.*


Open zone at the back of the knee



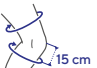
Hold: suspension strap above the calf and specially shaped tibial shell to prevent rotation

Comfort: inner foam lining

Practical: fitting buckle and tool-less adjustment of offloading level (Snap lock system)

2 models: right and left

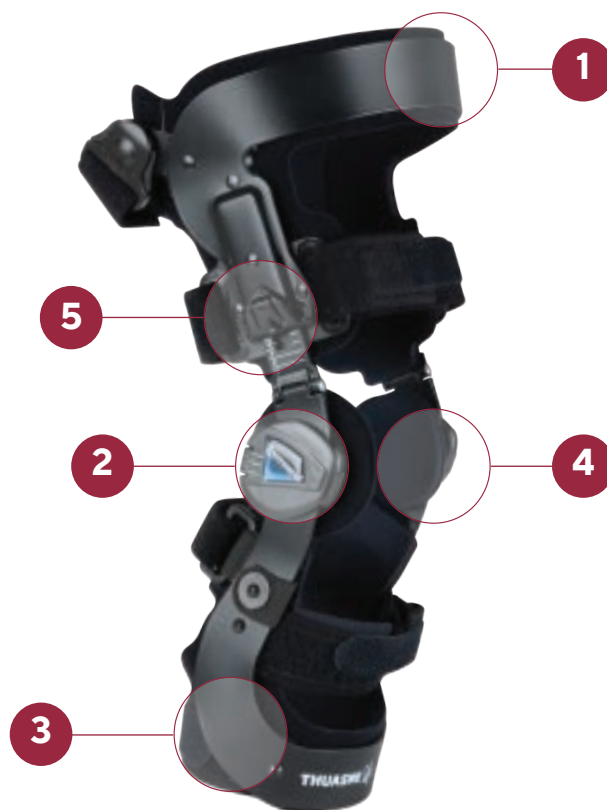


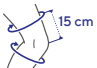			
	15 cm	ATTENTION Knee width	15 cm
XS	31 - 39 cm	7,5 - 9 cm	28 - 32 cm
S	39 - 47 cm	9 - 10 cm	31 - 35 cm
M	47 - 53 cm	10 - 11,5 cm	34 - 38 cm
L	53 - 60 cm	11,5 - 12,5 cm	37 - 40 cm
XL	60 - 64 cm	12,5 - 14 cm	38 - 43 cm
XXL	64 - 71 cm	14 - 15,5 cm	43 - 48 cm


*Also available in a custom format. Please see order form on page 95.




Presentation video available online

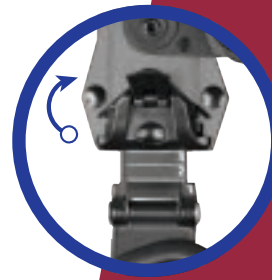

Lift

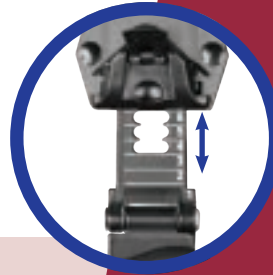

Slide

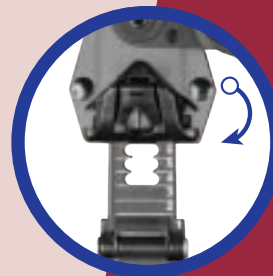

Lock


Adjustable Correction for OA Braces: LoadShifter

The patented LoadShifter mechanism and tool-less SnapLock are innovative features incorporated into the femoral section of Townsend’s Rebel Reliever™, Premier Reliever and Reliever Air rigid OA knee braces. By releasing the SnapLock, the fitter can utilize the LoadShifter to adjust the angle of the thigh shell. Increasing corrective pressure at the top of the leg amplifies the three-point counterforce at the knee and distal end of the brace – improving leg alignment and reducing compressive load on the damaged side of the knee.

Clinically Proven to Reduce Pain & Improve Quality of Life

Studies show Rebel Reliever reduces pain and symptoms of OA, improves balance and muscle function, and provides immediate, positive impacts to quality of life for patients.

Stony Brook University Study

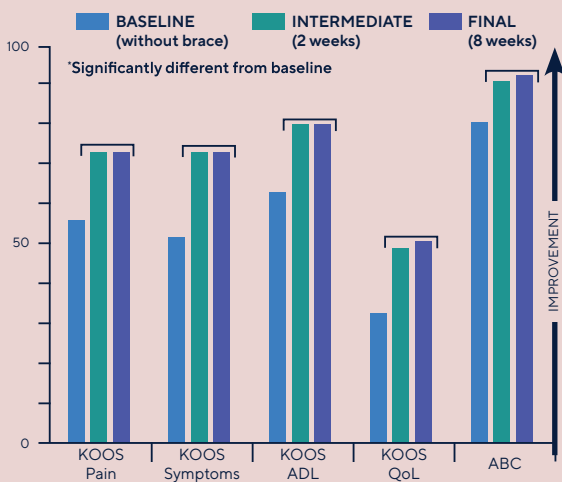


Studies demonstrate a reduction of pain and symptoms of OA, improved balance confidence and average daily living participation, and an overall improvement in quality of life.

Scan the QR code to see the full study



ROTOR Randomize Clinical Trial

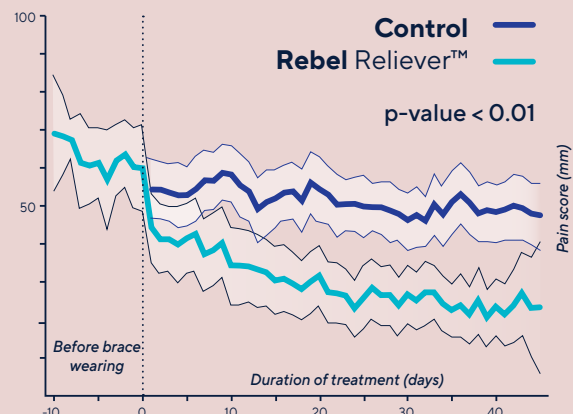

The Rebel Reliever™ knee brace significantly reduces pain immediately and for a sustainable time

Scan the QR code to see the full study






ANKLE

FUNCTION OF ANKLE BRACES, ANKLE STIRRUPS AND WALKING BOOTS



PROPRIOCEPTIVE ANKLE BRACE

Improved proprioception (perception of the body in space) via the use of a compressive knit

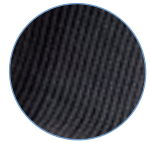


LIGAMENT ANKLE BRACE

Holds the ankle in an anatomical position using elastic straps



ANKLE STIRRUPS

Analgesic and protective action through a firmer immobilisation of the ankle. Its compressive effect can help reduce oedema

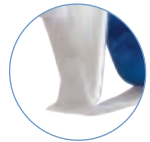

WALKING BOOT

Analgesic action through immobilisation of the ankle, reduction of the oedema thanks to adjustable compression, also suitable for managing post-operative treatment



Did you know?



6,000

people consult a health professional every day for an ankle sprain, **i.e. 250 per hour**¹


Athletes and young people

who are still growing are the most affected¹

(1) Ameli. (2018, avril). "Reconnaître une entorse de la cheville". Consulted on <https://www.ameli.fr/loire/assure/sante/themes/entorse-cheville/reconnaitre-entorse-cheville>

ANKLE | Which brace for which condition?

Which prescription?

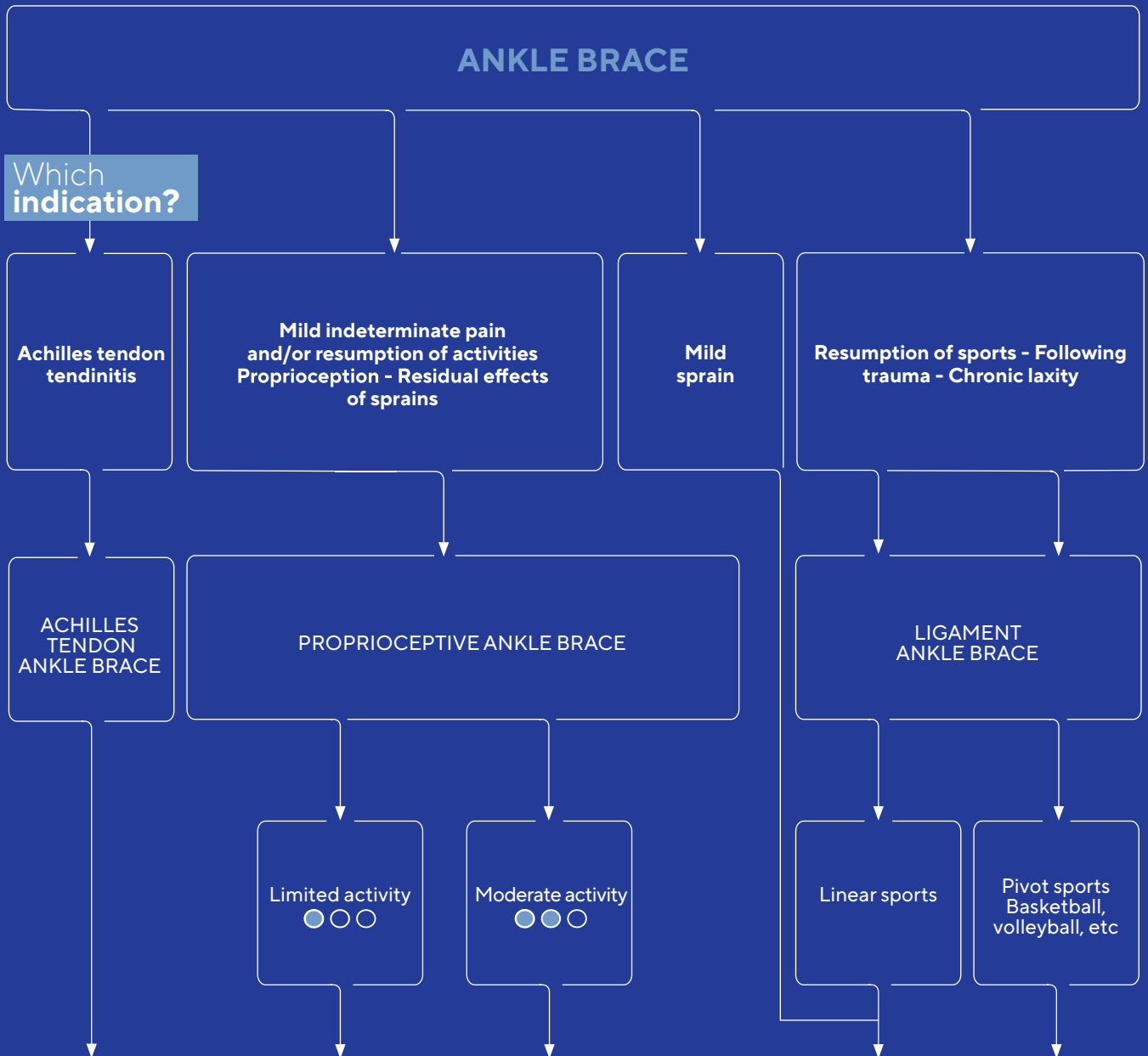

Silistab® Achillo



page 65

MalleoSoft



page 64

MalleoAction



page 64

Ligastrap® Malleo




page 67

Malleo Dynastab® BOA



page 66

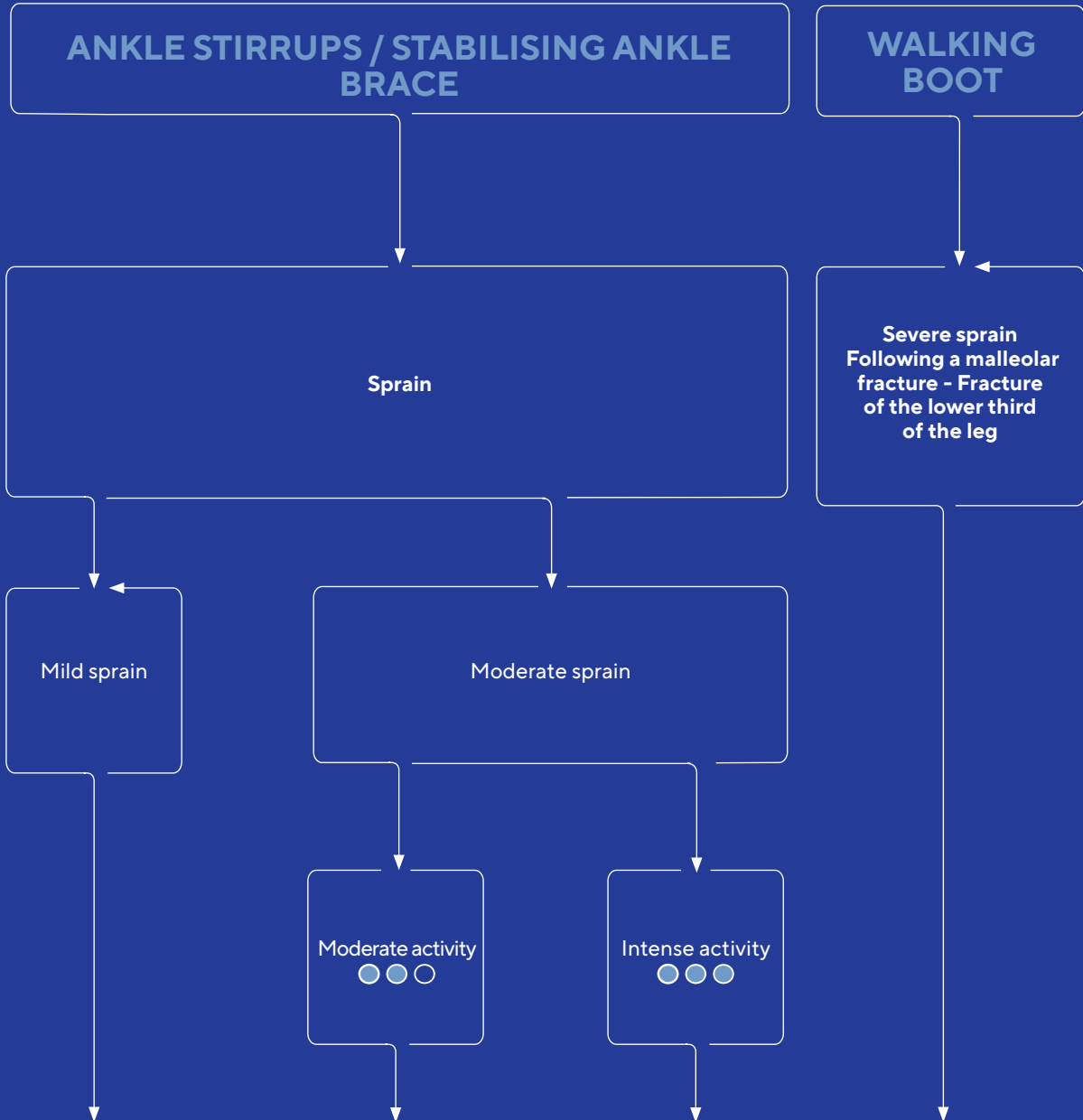

Ligacast® Air +



page
69

Ligastrap® Immo



page
69

Malleo
Dynastab® BOA



page
66

XLR8



page
68

Find all our products on the following pages.

MalleoSoft

Proprioceptive elastic ankle support (about 17 mmHg)

3600 02



Indicated for

Joint instability/laxity.
Mild ankle sprain (acute phase).
Post-operative, post-traumatic residual pain and/or oedema.
Proprioceptive support for rehabilitation.
Sport activities resumption.
Compression for scar/burns healing.

Target patient

Patient with limited activity

POINTS TO REMEMBER

Proprioception: elastic compressive knit (about 17 mmHg)

Comfort: anatomical knitting with comfort areas on the malleolus



1	19 - 21 cm
2	21 - 23 cm
3	23 - 25 cm
4	25 - 27 cm
5	27 - 30 cm

Colour: Silver



MalleoAction

Proprioceptive elastic ankle support (about 22.5 mmHg)

2361 03



Indicated for

Joint instability/laxity.
Mild ankle sprain (acute phase).
Post-operative, post-traumatic residual pain and/or oedema.
Proprioceptive support for rehabilitation.
Sport activities resumption.
Compression for scar/burns healing.

Target patient

Patient with moderate activity

POINTS TO REMEMBER

Proprioception: elastic compressive knit (about 22.5 mmHg)

Comfort: anatomical knitting with comfort areas on the malleolus

Practical: pull tabs for ease of application



0	16 - 19 cm	New
1	19 - 21 cm	
2	21 - 23 cm	
3	23 - 25 cm	
4	25 - 27 cm	
5	27 - 30 cm	

Colour: Grey



Silistab® Malleo

Proprioceptive anatomic protective ankle brace (about 21 mmHg) for the malleoli

2365 01



Indicated for

Joint instability/laxity.
Mild ankle sprain (acute phase).
Post-operative, post-traumatic residual pain and/or oedema.
Proprioceptive support for rehabilitation.
Sport activities resumption.
Compression for scar/burns healing.

Target patient

Patient with moderate activity

POINTS TO REMEMBER

Proprioception: elastic compressive knit (about 21 mmHg)

Comfort: anatomical knitting

Specific protection: silicone inserts protect the malleoli and instep



1	19 - 20 cm
2	21 - 22 cm
3	23 - 24 cm
4	25 - 27 cm
5	28 - 30 cm
6	31 - 33 cm

Colour: Black



Silistab® Achillo

Proprioceptive achilles tendon ankle support (about 23 mmHg)

2355 03

New design



Indicated for

Conservative treatment of Achilles tendon rupture.
 Post-operative, post-traumatic pain and/or oedema of the Achilles tendon.
 Prevention of sports injuries of the Achilles tendon.
 Prevention/treatment of Achilles tendon conditions (tendonitis, tenosynovitis, bursitis, achillodynia, Haglund's disease).

POINTS TO REMEMBER

- Offloading of the Achilles tendon:** stackable silicone heel pieces permitting 3 offloading heights (5, 8 or 13 mm)
- Tendon protection and relief:** peri-Achilles insert
- Comfort:** comfort zone over the malleoli
- Practical:** pull tabs for ease of application
- Proprioception:** elastic compressive knit (approximately 23 mmHg)



1	19 - 21 cm
2	21 - 23 cm
3	23 - 25 cm
4	25 - 27 cm
5	27 - 30 cm

Colour: Grey ●



Presentation and fitting video available online





Malleo Dynastab® BOA®

Stabilising ankle-immobilising brace
with BOA® fit system

2351 02

Also available with lace-up
fastening:

Malleo Dynastab®
Stabilising ankle brace
2350 03



Indicated for

Ankle immobilisation.
Conservative treatment of Achilles tendon injuries.
Joint laxity/chronic instability prevention during sport
activities resumption.
Moderate to severe ankle sprain.
Sport activities resumption.
Tendinopathy treatment.

Target Patient

Patients practicing pivot sports (basketball, volleyball,
handball...)

POINTS TO REMEMBER

- 1 - Increased ankle stabilisation:** lightweight metal reinforcements and strapping
- 2 - Easy fastening:** BOA® Fit System for even tightening that does not loosen over time
- 3 - Easy fitting:** wide opening to make it easier to put on and take off
- 4 - Comfort:** slim, anatomical design so that it can be worn in most types of shoes. Micro-ventilated 3D knit to wick away moisture



Malleo Dynastab® BOA

0*	16 - 19 cm	New
1	19 - 21 cm	
2	22 - 26 cm	
3	27 - 30 cm	



Malleo Dynastab®

1	19 - 22 cm
2	22 - 26 cm
3	26 - 30 cm

Colour: Grey



Presentation and fitting video
available online

* S0 version, Bilateral for the Malleo Dynastab® BOA®.
BOA® is a registered trademark of Boa Technology inc.

Ligastrap® Malleo

Proprioceptive ligament ankle brace
(about 27.5 mmHg) with functional strapping

2180 04



Indicated for

Joint instability/laxity.
Mild ankle sprain (acute phase).
Post-operative, post-traumatic residual pain and/or oedema.
Proprioceptive support for rehabilitation.
Sport activities resumption.
Compression for scar/burns healing.

Target Patient

Patients practicing linear sports

POINTS TO REMEMBER

Support: double stabilising strap for lateral ligaments and additional strap for light eversion

Proprioception: elastic compressive knit (about 27.5 mmHg)

Comfort: anti-slip on the calf



1	19 - 21 cm
2	21 - 23 cm
3	23 - 25 cm
4	25 - 27 cm
5	27 - 30 cm

Colour: Grey



Presentation and fitting video
available online

XLR8

Rigid shell walking boot with inflatable inserts

U08903 (long) - U09003 (short)



Indicated for

Ankle/foot post-operative/post-traumatic immobilisation.
 Severe ankle/foot sprain.
 Stable fracture of the lower third of the leg (tall version).
 Conservative treatment of ankle/foot fracture.
 Ankle/foot severe tendinopathy.
 Conservative treatment of Achilles tendon rupture.

POINTS TO REMEMBER

Strict immobilisation: integral rigid shell

Adjustable: air cells integrated into the boot to adjust the pressure according to the oedema and for an optimal anatomical adaptability

Comfort: curved anti-slip sole for a natural and comfortable walk and protection of the toes thanks to the adjustable arch (3 positions)

Bilateral model.



Shoe size

XS	< 3
S	3 - 5
M	5.5 - 9
L	9.5 - 12
XL	> 12

Two version:
Short and Long

Colour: Grey



Presentation video available
online



Ligastrap® Immo

Stabilising ankle stirrup with support strapping

2337 02




Indicated for

Ankle immobilisation.
 Conservative treatment of Achilles tendon injuries.
 Moderate to severe ankle sprain.
 Post-operative, post-traumatic residual pain and/or oedema.
 Sport activities resumption.

POINTS TO REMEMBER

Support: rigid lateral shells with asymmetrical hollows for the malleoli and additional reinforcement thanks to the removable strap

Easy to fit: closure with pulley system and self-fastening straps

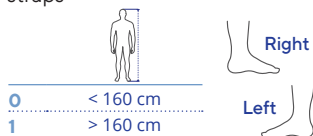

Colour: Black ●

Ligacast® Air +

Ankle stirrup with inflatable pads

2334 01



Indicated for

Ankle immobilisation.
 Conservative treatment of Achilles tendon injuries.
 Moderate to severe ankle sprain.

POINTS TO REMEMBER

Immobilisation: lateral rigid shells

Comfort: inflatable inserts with inflator bulb (included) for morphological adaptation after fitting the splint

Easy to fit: self-fastening straps and adjustable calcaneal strap

SINGLE SIZE

Bilateral version


Colour: White/blue ●

* Make sure to follow as a priority the recommendations of a healthcare professional


FOOT & ANKLE



Which solution for **which condition?**

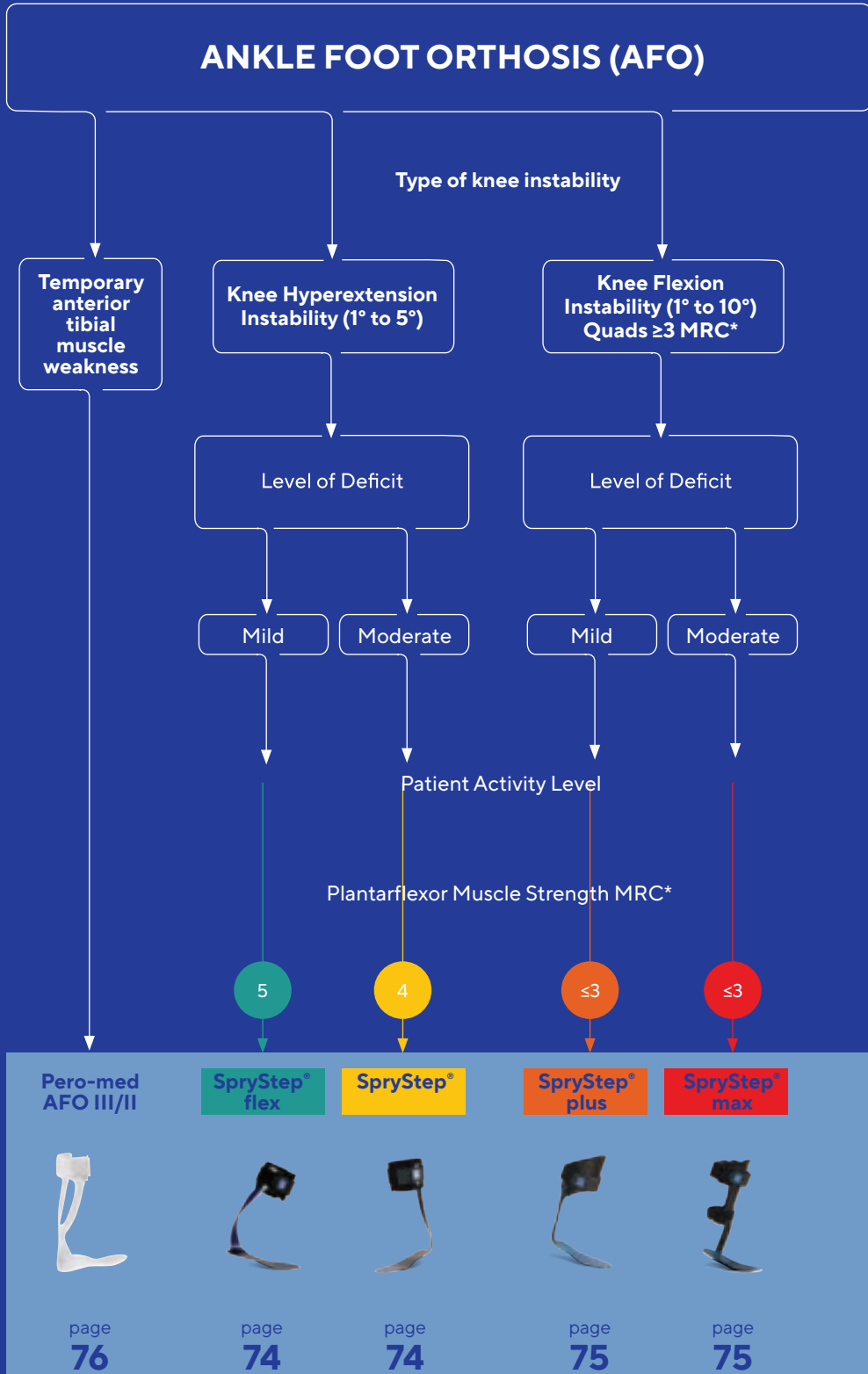

SpryStep® Range: DYNAMIC ANTERIOR AND POSTERIOR ANKLE-FOOT ORTHOSIS (AFO)

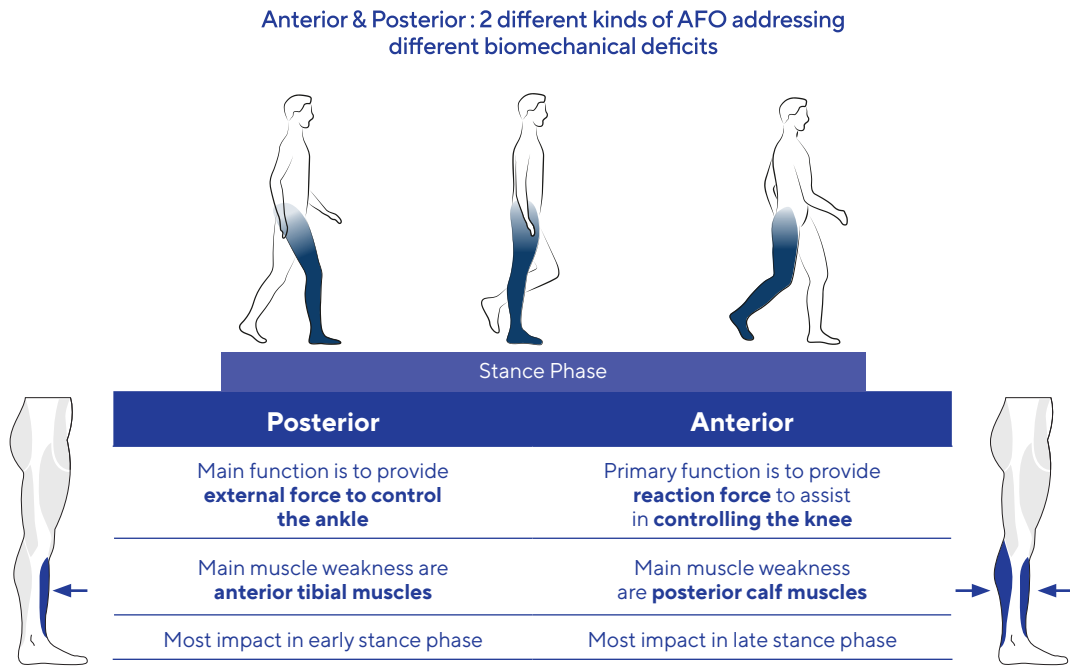

SpryStep® a complete range for patients suffering from lower limb weakness

Defined by elegance, Thuasne's SpryStep® range of AFOs is transforming the way people move. Using a proprietary blend of composite materials, the SpryStep® comes in a variety of solutions that are improving mobility and quality of life for patients.



**SpryStep Ortho-Neuro Solutions catalogue available –
To visit the online catalogue please scan the QR code below.**



SpryStep® Vector

Customizable in shape and stiffness the SpryStep® Vector offers superior control for the most complex conditions and patients.



Vector

Custom SpryStep®

Custom shape, molded inner boots and pre-tib shell options take the SpryStep®, SpryStep® Flex and SpryStep® Plus to the next level.



Original

Plus

Flex

SpryStep® OTS

With Five different models, the SpryStep® OTS range addresses a wide array of indications with variations in strut placement, ground reaction force and stiffness.



Flex

Original

Plus


Max

Pediatric

SpryStep® flex

Posterior dynamic ankle-foot orthosis

U01712

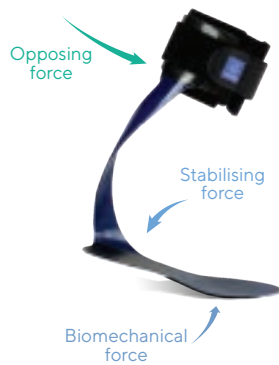

Indicated for

Foot drop of neurological origin from central nervous system, peripheral nervous system or muscular origin without knee impairment

Target patient


Patient with mild weakness of the lower limb (weakness of the anterior tibial muscle)
Patient with dynamic activities (fast walking, running, unstable surface...).

POINTS TO REMEMBER

Lightweight and durable composite structure designed with a unique combination of different materials blend and a precise manufacturing process.

The variety of materials used provides the patient with a good balance between flexibility and rigidity **to improve gait cycle and to offer greater energy return**

The 180° spiral strut provides great flexibility and allows an optimal energy return, ideal for people with intensive physical activity.

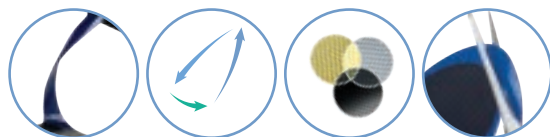

	Shoe size	Height
SpryStep® flex		
XS	1 - 4	30 cm
S	3 - 6	32 cm
M	5 - 8	34 cm
L	7 - 9.5	36 cm
XL	9.5 - 12	38 cm



Colour: Graphite




Presentation and fitting video available online

SpryStep®

Dynamic ankle-foot orthosis

U01703

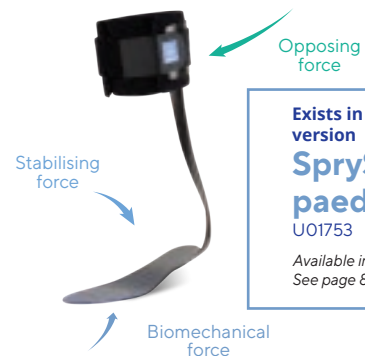

Exists in paediatric version
SpryStep® paediatric
U01753
Available in October 2021
See page 84

Indicated for

Foot drop of neurological origin, traumatic origin or muscular origin.

Target patient


Patients with mild to moderate weakness of the lower limb (weakness of the anterior tibial muscle)
Patient with moderate activity (walking on a flat surface)

POINTS TO REMEMBER

Lightweight and durable composite structure designed with a unique combination of materials blend and a precise manufacturing process.

The variety of materials used provides the patient with a good balance between flexibility and rigidity **to improve gait cycle and to offer a rebound effect**.

Posterior lateral position of the strut **to optimise compliance and energy restitution during physical activity**.

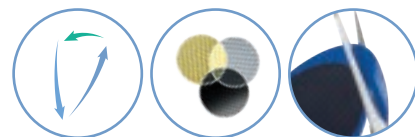

	Shoe size	Height
SpryStep®		
XS	1 - 4	30 cm
S	3 - 6	32 cm
M	5 - 8	34 cm
L	7 - 9.5	36 cm
XL	9.5 - 12	38 cm



Colour: Graphite




Presentation and fitting video available online

SpryStep® plus

Spiral anterior shell dynamic ankle-foot orthosis

U01722

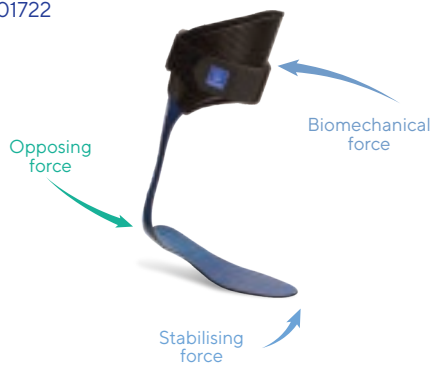

Indicated for

Lower limb weakness of neurological, traumatic or muscular origin

Target patient


Patients with mild to moderate weakness of the lower limb (weakness of the anterior tibial muscle)

POINTS TO REMEMBER

Anterior carbon structure that supports and guides the knee towards extension, **helps the weakened plantar flexors to propel and support the knee during the extension phase.**

The variety of materials used allows the patient to have a good balance between flexibility and rigidity **to improve gait and allow the tibia to be tilted during the support phase.**

Posterior lateral position of the strut **to optimise compliance and energy restitution during physical activity.**

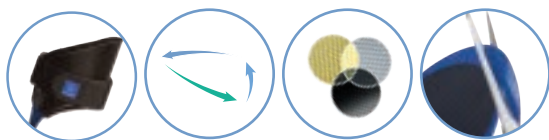

SpryStep® plus measurement

Shoe size	Foot length		Foot length		Product height
	Mini	Maxi	Mini	Maxi	
XS	1 - 4	21,5 cm	24,5 cm	29 cm	33 cm
S	3 - 6	23 cm	26 cm	32,5 cm	36 cm
M	5 - 8	24,5 cm	27,5 cm	35,5 cm	39,5 cm
L	7 - 9,5	26 cm	29 cm	38,5 cm	42,5 cm
XL	9,5 - 12	29 cm	30,5 cm	42 cm	45,5 cm



Colour: Graphite




Presentation and fitting video available online

SpryStep® max

Anterior shell dynamic ankle-foot orthosis

U01732

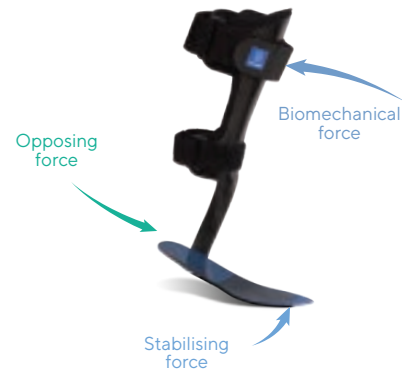

Indicated for

Lower limb weakness of neurological, traumatic or muscular origin

Target patient

Patients with mild to moderate weakness of the lower limb (weakness of the anterior tibial muscle and gastrocnemius muscle)


Lower activity level.

POINTS TO REMEMBER

Anterior carbon structure that supports and guides the knee towards extension, **helps the weakened plantar flexors to propel and maintain the knee during the support phase.**

The rigidity of the structure combined with the flexibility of the sole **provides great stability and energy return**

Lateral position of the upper, anterior to the malleolus, **to allow easy insertion into the shoe.**

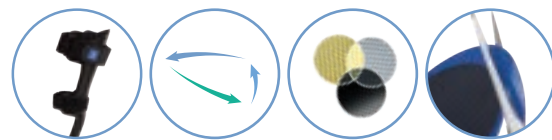

SpryStep® max measurement

Shoe size	Foot length		Product height	
	Mini	Maxi		
XS	12,5 - 2	19,5 cm	21,5 cm	33,5 cm
S	2 - 4	21 cm	23 cm	37 cm
M	4 - 6,5	23 cm	25,5 cm	39,5 cm
L	6,5 - 9	25,5 cm	27,5 cm	42 cm
XL	9 - 11	27,5 cm	29 cm	42 cm



Colour: Graphite



Presentation and fitting video available online

Pero-med AFO III/II

Peroneal orthosis

T479652



Indicated for

Mild to medium foot lifting weakness, mild peroneal paresis of the IIIrd and IInd degree without spasticity

POINTS TO REMEMBER

Effective right-angle foot support to secure the walking motion

Extra comfort thanks to its **anatomical** shape and **light weight**

Its design allows it to be worn with all types of flat shoes

Shoe size		Right		Left	Colour: White ○
S	2 - 4				
M	5 - 6				
L	6 - 7				
XL	7 - 9,5				

Podo-med T500501

Heel offloading shoe



Indicated for

Post-operative use
Plantar ulcers
Injuries and fractures

POINTS TO REMEMBER

Offload: wedge shaped base with dorsal flexion at 10°

Comfort: seamless interior to avoid pressure areas

Easy to fit and care: complete opening of the shoe and adjustment by self-fastening straps

Shoe size		Foot length		Colour: Black ●
1	S	2 - 4	22 - 23 cm	
2	M	5 - 6,5	24 - 25 cm	
3	L	7 - 9	26 - 27 cm	
4	XL	9,5 - 11	28 - 29 cm	

Podo-med T500511

Heel offloading shoe



Indicated for

Post-operative use
Plantar ulcers
Injuries and fractures

POINTS TO REMEMBER

Same as: the Podo-med T500501 model but with a removable heel cap with additional reinforcement of the back of the foot

Offload: wedge shaped base with dorsal flexion at 10°

Comfort: seamless interior to avoid pressure areas

Easy to fit and care: complete opening of the shoe and adjustment by self-fastening straps

Shoe size		Length		Colour: Black ●
0	XS	13 - 2	20 - 21 cm	
1	S	2 - 4	22 - 23 cm	
2	M	5 - 6,5	24 - 25 cm	
3	L	7 - 9	26 - 27 cm	
4	XL	9,5 - 11	28 - 29 cm	

Podo-med T500301

Forefoot offloading shoe



Indicated for
Post-operative use
Plantar ulcers
Injuries and fractures

POINTS TO REMEMBER

Offloading: wedge shaped base with dorsal flexion at 10°
Comfort: rounded heel for safe and comfortable walking.
Seamless interior to avoid pressure areas
Easy to fit and care: complete opening of the shoe and adjustment by self-fastening straps



	Shoe size	Foot length
1 S	2 - 4	22 - 23 cm
2 M	5 - 6.5	24 - 25 cm
3 L	7 - 9	26 - 27 cm
4 XL	9.5 - 11	28 - 29 cm

Colour: Black ●

Podo-med T500401

Forefoot offloading shoe with long sole (with toes protection)



Indicated for
Post-operative use
Plantar ulcers
Trauma and fractures

POINTS TO REMEMBER

Offload: wedge shaped base with dorsal flexion at 10°.
Toes protection: long sole (extended surface) and specific toes protection edges.
Comfort: rounded heel for a safe and comfortable walk. Seamless interior to avoid pressure areas.



	Shoe size	Length
0 XS	13 - 2	20 - 21 cm
1 S	2 - 4	22 - 23 cm
2 M	5 - 6.5	24 - 25 cm
3 L	7 - 9	26 - 27 cm
4 XL	9.5 - 11	28 - 29 cm

Colour: Black ●

Podo-med T500451

Forefoot offloading shoe with a long sole without edges



Indicated for
Post-operative use
Plantar ulcers
Injuries and fractures

POINTS TO REMEMBER

Offload: wedge shaped base with dorsal flexion at 10°
Foot protection: extended sole
Comfort: rounded heel for safe and comfortable walking.
Seamless interior to avoid pressure areas



	Shoe size	Length
0 XS	13 - 2	20 - 21 cm
1 S	2 - 4	22 - 23 cm
2 M	5 - 6.5	24 - 25 cm
3 L	7 - 9	26 - 27 cm
4 XL	9.5 - 11	28 - 29 cm

Colour: Black ●

Podo-med T500471

Forefoot offloading shoe with long sole and forefoot support strap



Indicated for
Post-operative use
Plantar ulcers
Injuries and fractures

POINTS TO REMEMBER

Offload: wedge shaped base with dorsal flexion at 10°
Foot protection and support: extended sole and end of foot fitting with self-fastening strap
Comfort: rounded heel for safe and comfortable walking.
Seamless interior to avoid pressure areas



	Shoe size	Length
0 XS	13 - 2	20 - 21 cm
1 S	2 - 4	22 - 23 cm
2 M	5 - 6.5	24 - 25 cm
3 L	7 - 9	26 - 27 cm
4 XL	9.5 - 11	28 - 29 cm

Colour: Black ●

Pedipro

Shock-absorbing insoles and heel inserts

6510 01 (insoles) / 6500 01 (heel inserts)



Indicated for

Heel and foot pain, including metatarsalgia and plantar fasciitis. Rheumatoid foot. Relief for tendons.

POINTS TO REMEMBER

Distribution and absorption of vibrations: flexible silicone

Comfort and breathability: ventilation channels

Reinforced hold: non-slip pads

Under-heel thickness: 8 mm

Sold in pairs



	INSOLES	HEEL INSERTS
1	3 - 6	3 - 6
2	6.5 - 7	6.5 - 11
3	8 - 9	
4	9.5 - 11	

Colour: Translucid

Pedipro plus

Shock-absorbing insoles and heel inserts with cotton coating

6610 01 (insoles) / 6600 01 (heel inserts)



Indicated for

Heel and foot pain, including metatarsalgia and plantar fasciitis. Rheumatoid foot

POINTS TO REMEMBER

Distribution and absorption of vibrations: flexible silicone

Comfort: anatomically shaped inserts at the heel and metatarsals (insole) or at the heel spur (heel piece)

Reinforced hold in the shoe: non-slip pads

Under-heel thickness: 8 mm

Sold in pairs




	INSOLES	HEEL INSERTS
1	4 - 6	3 - 6
2	6.5 - 7	6.5 - 11
3	8 - 9	
4	9.5 - 11	

Colour: White

Pedipro softer

Adaptable shock-absorbing heelpieces

6605 01

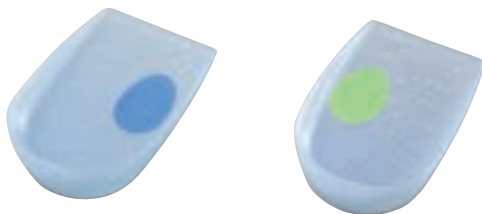

Indicated for

Heel and foot pain, including metatarsalgia and plantar fasciitis. Rheumatoid foot. Relief for tendons.

POINTS TO REMEMBER

Flexible silicone providing **shock absorption and distribution**

Progressive heel inserts with concave areas for suitable inserts: aniseed green insert for heel spur comfort, blue insert for healthy feet or other conditions

Comfortable thanks to **ventilation channels for better air flow**

High sides for better fit inside shoe

Under-heel thickness: 10 mm

Sold in pairs



1	3 - 6
2	6.5 - 11

Colour: Translucid

Elastocalx N

T479801



Indicated for

Osteoarthritis of the ankle, knee and hip joints. Degenerative diseases of the spine. Achilles tendon irritation. Muscle and tendon disorders. Overuse syndrome. Differences in leg length. Tilted pelvis irritation.



POINTS TO REMEMBER


Symmetrical, flat ended without sides

Easily fits into most types of shoes

Composition: silicone


Single right/ left model

		 Shoe size	 Length
0	XS	13 - 2	20,3 - 21,7 cm
1	S	2 - 4	22,3 - 23,7 cm
2	M	5 - 6,5	24,3 - 25,7 cm
3	L	7 - 9	26,3 - 26,7 cm
4	XL	9,5 - 11	27,3 - 30,7 cm

Colour: Translucid 

Sohli

T479841

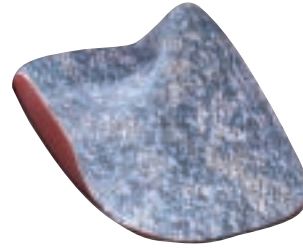

Indicated for

Medial calcaneal correction.

POINTS TO REMEMBER

Increased comfort with textile covering on the foot side

Thin sides with no compression points

Sold in pairs

		Shoe size	Length
		EUR	cm
1		19 - 20	10,7 - 11,3
2		21 - 22	12 - 12,7
3		23 - 24	13,3 - 14
4		25 - 26	14,7 - 15,3
5		27 - 28	16 - 16,7
6		28 - 30	17,3 - 18
7		31 - 32	18,7 - 19,3
8		33 - 34	20 - 20,7
9		35 - 36	21,3 - 22
10		37 - 38	22,7 - 23,3

Colour: Blue/red 

Elastocalx S/P

T479811



Indicated for



Correction of supination (outward rolling of the foot) and pronation (inward rolling of the foot). Valgus and varus deformity of legs (knock kneed or bow-legged)


POINTS TO REMEMBER

Easily fits into most types of shoes

Composition: silicone

Single right/ left model

		 Shoe size	 Length
0	XS	13 - 2	20,3 - 21,7 cm
1	S	2 - 4	22,3 - 23,7 cm
2	M	5 - 6,5	24,3 - 25,7 cm
3	L	7 - 9	26,3 - 27,7 cm
4	XL	9,5 - 11	28,3 - 29,7 cm

Colour: Translucid 



Ortel C1 junior

Light cervical collar

2620 001 004




Indicated for

Torticollis, mild whiplash / mild sprain, cervical osteoarthritis, dropped head syndrome, post-operative use, cervical syndrome, rehabilitation after wearing a rigid cervical collar.

POINTS TO REMEMBER

Modular support: high quality foam with removable rigid reinforcement

Support: precise adjustment with self-fastening closure

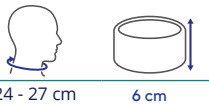


Colour: Blue 

Ligaflex® junior

Clavicular immobilisation straps

2640 02

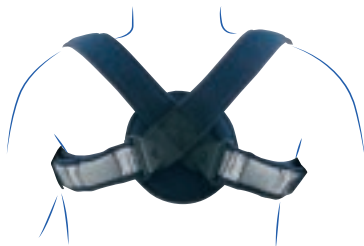

Indicated for

Displaced or non-displaced clavicle fractures.

POINTS TO REMEMBER

Adaptability: trimmable straps to fit all patients (one size fits all)

Easy to fit: self-fastening system and buckle closure

Comfort: dorsal foam anchoring pad and quilted straps

Universal size (shoulder width < 34 cm)

Unique size

Colour: Blue 

Ligaflex® Classic

Wrist immobilisation splint

2435 02



Indicated for

Wrist sprain.
Non-surgical wrist fracture.
Post-operative or post-traumatic wrist immobilisation.
Wrist tendinopathies.
Conservative treatment of carpal tunnel syndrome.
Lateral epicondylitis.
Rheumatoid arthritis.
Wrist osteoarthritis.

POINTS TO REMEMBER

Immobilisation: via a rigid, malleable and anatomical palmar stay

Comfort: aerated, foam lined fabric

Even compression: via dual strap system



Colour: Black 

Ligaflex® Manu

Wrist and thumb immobilisation splint

2430 02



Indicated for

Thumb sprain.
Wrist and/or thumb sprain.
Non-surgical scaphoid fracture.
Non-surgical wrist fracture.
Post-operative or post-traumatic wrist/thumb immobilisation.
Wrist and/or thumb tendinopathies.
De Quervain tenosynovitis.
Rheumatoid arthritis.
Lateral epicondylitis.
Conservative treatment of carpal tunnel syndrome.
Thumb/wrist osteoarthritis.
Neuro-muscular impairments (contracture, hand drop or spasticity...).

POINTS TO REMEMBER

Wrist immobilisation: via a rigid, malleable and anatomical palmar stay

Thumb immobilisation: via a rigid, malleable thumb stay

Comfort: aerated, foam lined fabric

Even compression: via dual strap system



Colour: Black 

GenuAction

Proprioceptive elastic
knee support

2325 03

New
design*



Indicated for

Knee pain and/or swelling management
Joint instability/laxity
Proprioceptive support for rehabilitation
Conservative treatment of mild knee
ligament injuries.

POINTS TO REMEMBER

Anatomically-shaped knit

Comfort zones over the patella and at
the back of the knee

Class 3 compression level (French stan-
dard: 20-36mmHg /27,1-48 hPa) ensures
good support and proprioception



0 28 - 31 cm

Colour: Dark blue/Grey



Ligaflex Immo 0°

Knee immobilisation splint at 0°

2410 04



Indicated for

Post-operative immobilisation.
Post-traumatic immobilisation.
Knee pain management.

POINTS TO REMEMBER

Knee joint stabilisation via rigid stays

Total immobilisation at 0°

Adjustable closure and support via
self-fastening straps



0 30 - 45 cm

Colour: Navy



MalleoAction

Proprioceptive elastic
ankle support

2361 03

New
design*



Indicated for

Joint instability/laxity.
Mild ankle sprain (acute phase).
Post-operative, post-traumatic residual
pain and/or oedema.
Proprioceptive support for rehabilitation.
Sport activities resumption.
Compression for scar/burns healing.

POINTS TO REMEMBER

Proprioception: elastic compressive knit
(about 22.5 mmHg)

Comfort: anatomical knitting with
comfort zone over the malleoli

Practical: pull tabs for ease of application



0 16 - 19 cm

Colour: Dark blue/Grey



Malleo Dynastab® BOA®

Stabilising ankle-immobilising brace with BOA® Fit System

2351 02



Indicated for

Ankle Immobilisation.
Conservative treatment of Achilles tendon injuries.
Joint laxity/chronic instability prevention during sport activities resumption.
Moderate to severe ankle sprain.
Sport activities resumption.
Tendinopathy treatment.

Target patient

Patients practicing pivot sports (basketball, volleyball, handball...)

POINTS TO REMEMBER

Ankle stabilisation: lightweight, lateral reinforcements and strapping for stability and immobilisation

Easy fastening: BOA® Fit System for a custom fit and even compression

Easy fitting: wide opening for ease of application and removal

Comfort: slim, anatomical design, fits easily in most types of shoes



0 16 - 19 cm
Bilateral

Colour: Grey

Ligacast® junior

Stabilising ankle brace with foam inserts

2600 001 004



Indicated for

Ankle Immobilisation.
Conservative treatment of Achilles tendon injuries.
Moderate to severe ankle sprain.

POINTS TO REMEMBER

Immobilisation of the ankle: lateral rigid shells

Comfort: padded foam inserts

Easy to fit: adjustment by self-fastening straps

Adaptability: height-adjustable calcaneal strap for adaptation to the patient's morphology



< 1,40 m
Single size

Colour: White

LigaStep Pedi Walker

Paediatric walking boot with rigid side uprights

U09201



Indicated for

Post-operative or post-traumatic immobilisation of the ankle or foot.
Severe ankle and/or foot sprains.
Stable fractures of the lower third of the leg.
Conservative treatment of ankle and/or foot fractures.

POINTS TO REMEMBER

Extra sturdy design for active kids

Rigid uprights provide protection and immobilisation of the injured limb

Lightweight structure reduces impact on gait pattern

Wide and anti-slip sole increases stability



Shoe size

S 5 - 7,5
M 7,5 - 12,5
L 12,5 - 2

Colour: Grey



PAEDIATRIC AFO SPRYSTEP SOLUTIONS



POINTS TO REMEMBER

Prefabricated from size chart

Custom option available = Custom Sprystep Original

Broadly trimmable with scissors at the toe, medial side of footplate and calf cuff

Built with **medium flexibility** for dynamic children

Lightweight, durable composite construction

Lateral strut sweeping upwards to posterior

Fits easily into most styles of shoe.

Shoe size	Foot length		Calf height	
	Mini	Maxi	Mini	Maxi
XS 2 - 5.5	9.5 cm	14.5 cm	16 cm	18 cm
S 5.5 - 8.5	11 cm	16 cm	19 cm	21 cm
M 8.5 - 10	12.5 cm	17.5 cm	22 cm	24 cm
L 10 - 12.5	14 cm	19 cm	25 cm	27 cm
XL 12.5 - 1	15.5 cm	20.5 cm	28 cm	30 cm



Lumbar support belt


0361 01

1

Indications: Non-specific low back pain (acute, subacute and chronic phases). Spinal stenosis

Properties: ventilated elastic fabric

cm	S	M	L	XL	XXL
	60-70	71-86	87-102	103-121	122-140

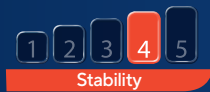

Neoprene shoulder padding

0576 03

2

Indications: Conservative treatment for non-surgical proximal humeral fractures
Post-operative immobilisation (such as after rotator cuff surgery or dislocation reduction...)
Shoulder and/or elbow injuries or inflammatory conditions
Shoulder dislocation
Shoulder subluxation or dislocation prevention (joint instability)
Support for paralytic upper limb

Thermal insulation: neoprene

Anatomical fit: elastic knit coating

cm	S	M	L
	28-31	32-35	36-40



Anti-epicondylitis elbow brace

0336 03

3

Indications: Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).
Prevention of elbow tendinopathy recurrence.
Support for physical activity resumption.

Modular product: prevention and acute pain

Evaporation of perspiration: perforated neoprene

Anatomical fit: anatomically shaped cut-outs

cm	S	M	L	XL
	22-24	25-27	28-30	31-33



Elbow support

0338 01

3

Indications: Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).
Tendinopathy treatment.
Prevention of recurrence.
Helps prevent pain during excessive effort (sport or work activities).

Compressive knit: improved proprioception

Comfort zone: easy centring of the elbow

Enhanced stretch: provides greater comfort, avoids excessive thickness

cm	S	M	L	XL
	22-24	25-27	28-30	31-33



Tennis elbow armband

0352 02

3

Indications: prevention of tendinitis of the elbow lateral epicondylitis (tennis elbow) or medial epicondylitis (golfer's elbow)

Properties: two silicone inserts provide localised compression and reduce tension on the elbow

cm	1	2
	22-30	30-38



Strapping thumb brace

0332 01

4

Indications: Immobilisation in case of pain or trauma of the thumb.
Post-operative immobilisation.
Thumb osteoarthritis (including rhizarthrosis).

Fragile thumb ligaments Wrist/thumb support: elastic straps

Evacuation of perspiration: perforated neoprene

Anatomical fit: anatomical shapes

cm	S	M	L
	14-15,5	16-17,5	18-20



Braces

Upper LIMBS



Strapping wrist band

0340 02

4

Indications: prevention of ligament injuries.
Resumption of activities after injury

Reinforces the wrist: strong compression strap

Evaporation of perspiration: breathable strap

Anatomical fit: one size only



Neoprene wrist support

0575 03

4

Indications: Wrist tendinopathies.
Wrist sprain.
Wrist osteoarthritis.
Rheumatoid arthritis.

Reinforced support: complete cover
of the joint

Thermal insulation: neoprene

Anatomical fit: elastic knit coating



BOA®
THE BOA
GUARANTEE

Wrist immobilisation brace with BOA® Fit System

0356 01

4

Indications: Wrist sprain.
Wrist osteoarthritis.
Rheumatoid arthritis.

Properties: firm support of wrist in correct
position (anatomical and conformable stay).
BOA® closure system for easy, uniform
tightening

Optimum comfort: ventilated fabric, inner
foam lining

Boa® is a registered trademark of Boa®
Technology Inc.



One size only



Stability

One size only



Stability




1

2

cm

13-17

17-23

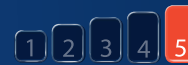

Stability



Patellar bandage

0329 01

5

Indications: Patella joint or ligament inflammation and tendinopathy (runner's knee and jumper's knee syndrome)
Mild patella joint instabilities
Anterior knee or patella pain.

Stimulation of proprioception: silicone combined with the fabric band

One size only



Neoprene thigh support

0572 03

5

Indications: muscle tears in the acute phase or during the physiotherapy phase.
Muscle pulls. Muscle strains

Thermal insulation: neoprene

	S	M	L	XL
cm	44-48	49-53	54-58	59-63



Neoprene knee support

0573 03

5

Indications: prevention of ligament injuries (sprains). Resumption of activities after a sprain. Weak knee ligaments

Thermal insulation: neoprene

Stimulation of proprioception: elastic knit

	S	M	L	XL	XXL
cm	31-33	34-36	37-39	40-42	43-45



Knee support

0334 03


5

Indications: gentle support for fragile or painful joints (mild sprains, etc.)

Compressive knit: improved proprioception

Comfort zone: easy centring of the kneecap

Enhanced stretch: provides greater comfort, avoids excessive thickness

	S	M	L	XL	XXL
cm	32-34	35-37	38-41	42-45	46-49



Open knee support

0307 01

5

Indications: gentle support for fragile or painful joints (mild sprains, etc.)

Compressive knit: improved proprioception

Patellar opening: aids centring of the kneecap

	S	M	L	XL	XXL
cm	31-33	34-36	37-39	40-42	43-45



Reinforced knee support

0354 01

5

Indications: chronic knee instability (cruciate ligament rupture). Resumption of sporting activities following mild knee sprains and injuries (including surgery)

Properties and specific characteristics of the product: the elastic knit exerts uniform pressure on the joint and improves proprioception (the joint's sensory perception)

Comfort zone over the kneecap and the crease behind the knee. Flexible lateral stays ensure good support

	S	M	L	XL	XXL
cm	32-34	35-37	38-41	42-45	46-49



Braces

Lower LIMBS



Open strapping knee brace

0331 03

5

Indications: functional treatment of mild to medium knee sprains. Resumption of activities. Lateral and cruciate ligament sprain. Chronic instability of the knee

Properties: total opening for an easy fitting. Anatomical 3D design. Elastic crossed and adjustable straps for a dynamic and lateral support. Flexible side stays. Aerated popliteal area. Breathable perforated neoprene, with heating effect

S	M	L	XL
32-36	37-40	41-44	45-48



Novelistic knee strap

0270 02

5

Indications: functional treatment of mild to medium knee sprains. Resuming activity after a severe sprain including surgery

Properties: practical cross-over straps to strengthen the lateral support. Flexible lateral frame for better stability. Comfortable knitted patch at the kneecap and the popliteal area

S	M	L	XL	XXL
32-34	35-37	38-41	42-45	46-49



Reinforced knee support

0570 03

5

Indications: prevention of ligament injuries (sprains). Resumption of activities after a sprain. Weak knee ligaments

Thermal insulation: neoprene

Stimulation of proprioception: elastic knit coating

Better joint support: lateral stays

S	M	L	XL	XXL
31-33	34-36	37-39	40-42	43-45



Reinforced patellar knee brace

0355 01

5

Indications: patellar tendinitis and cartilage pain in the kneecap. Mild sprain of the lateral ligaments of the knee. Resumption of sporting activities following mild knee sprains and injuries

Properties: comfort and guiding zone at the kneecap (patellar ring). Comfort zone at the back of the knee. Flexible side stays ensuring good support

S	M	L	XL	XXL
32-34	35-37	38-41	42-45	46-49



Neoprene ankle support

0571 03

6

Indications: prevention of ligament injuries (sprains). Resumption of activities after a sprain. Weak ankle ligaments

Thermal insulation: neoprene

Stimulation of proprioception: elastic knit coating

S	M	L	XL
19-20	21-22	23-24	25-27



Open ankle support

0306 02

6

Indications: Joint instability/laxity. Mild ankle sprain (acute phase). Post-operative, post-traumatic residual pain and/or oedema. Proprioceptive support for rehabilitation. Sport activities resumption.

Compressive knit: improved proprioception

Heel opening: aids centring of the heel

Compression for scar/burns healing

S	M	L	XL
21-23	24-26	27-29	>29



Braces

Lower LIMBS



Ankle support

0333 03

6

Indications: Joint instability/laxity. Mild ankle sprain (acute phase). Post-operative, post-traumatic residual pain and/or oedema. Proprioceptive support for rehabilitation. Sport activities resumption. Compression for scar/burns healing.

Compressive knit: improve proprioception

Comfort zone: easy centring of the malleolus

Enhanced stretch: provides greater comfort, avoids excessive thicknesses

cm	S	M	L	XL	XXL
19-21	21-23	23-25	25-27	27-30	



Reinforced ankle support

0353 01

6

Indications: Joint instability/laxity. Mild ankle sprain (acute phase). Post-operative, post-traumatic residual pain and/or oedema. Proprioceptive support for rehabilitation. Sport activities resumption. Compression for scar/burns healing.

Properties: the elastic knit exerts uniform pressure on the joint and improves proprioception (the joint's sensory perception). Comfort zone over the malleolus

cm	S	M	L	XL	XXL
19-21	21-23	23-25	25-27	27-30	



Strapping ankle brace

0330 03

6

Indications: Joint instability/laxity. Mild ankle sprain (acute phase). Post-operative, post-traumatic residual pain and/or oedema. Proprioceptive support for rehabilitation. Sport activities resumption. Compression for scar/burns healing.

Perfect support for the ankle: elastic straps

Evacuation of perspiration: perforated neoprene

Anatomical fit: adjustable neoprene Tab

cm	S	M	L	XL
19-22	22-25	25-28	28-30	



Novelistic ankle strap

0280 02

6

Indications: Joint instability/laxity. Mild ankle sprain (acute phase). Post-operative, post-traumatic residual pain and/or oedema. Proprioceptive support for rehabilitation. Sport activities resumption. Compression for scar/burns healing.

Properties: two-way support straps "in an 8" (for a sprain of the lateral collateral or medial collateral ligament) protecting the flexion-extension. Extra thick sole for more comfort

cm	S	M	L	XL	XXL
19-21	21-23	23-25	25-27	27-30	



Stabilising ankle brace Boa®

0351 02

6

Indications: Ankle immobilisation. Conservative treatment of Achilles tendon injuries. Joint laxity/chronic instability prevention during sport activities resumption. Moderate to severe ankle sprain. Sport activities resumption. Tendinopathy treatment.

Properties: rigid side reinforcements to ensure ankle stability. BOA® Fit System: fast and reliable. Slim product that fits easily inside a shoe

cm	S	M	L
19-22	22-26	26-30	



BOA®

THE BOA
GUARANTEE

Boa® is a registered trademark of Boa® Technology Inc.

THUASNE **SPORT**

Argicalm[®]

MULTIPURPOSE, REUSABLE THERMAL CLAY PACK

7200 02

Relieves pain



Extra tip!

Self-adhering strap to fit all.



Attached to itself



or attached to the tab

INDICATED FOR



Hot pain relief:

Back pain, joint and muscle stiffness, spasms, cramps, torticollis, abdominal pain

Cold pain relief:

Sprains, tendinitis, toothache, bruises, strains, migraines, eye pain, arthritis

POINTS TO REMEMBER

Adaptability:

Ergonomic, easily conformable pack thanks to the clay's density

Comfort:

Reusable, washable, velvet-touch protective sleeve
Elastic self-adhering straps to fit all

Size	Recommended uses	Dimension (cm)
Size 1	Wrist - Elbow - Face	19 x 11
Size 2	Ankle - Elbow - Knee	26 x 13
Size 3	Knee - Back - Abdomen	30 x 20
Size 4	Neck - Shoulder - Back	43 x 17



Presentation video available online

Rebel Pro

ORDER FORM

Account Contact Information

Clinician Name: _____ Clinician Email: _____ Clinician Phone: _____

Account Name: _____ Purchase Order Number: _____

For Thuasne Office Use Only

Account Number: _____ Order Number: _____

Purchase Order Number: _____ Purchase Order Number: _____

Patient Information

Fit Date: _____

Patient's Last Name: _____

Patient's First Name: _____

Male Female Age _____

Weight _____ (LBS) Height _____ (IN)

Leg: Left Right

Brace Configuration

Thigh Shell Length 7" anterior 8" anterior

Tibia Shell Length 6" anterior 7" anterior
 8" anterior 7" posterior

Colour

Matte Finish

Black (Standard) Grey White
 Atlantic (Light Blue) Red

Satin Finish

Lemon Orange Lime
 Fuchsia Pacific (Dark Blue)

Accessories

- Flexion Stop Kit (Extension stops included) £10 charge
 Extension assist bands/posts* £75 charge
 Quick release buckles
 Anti-migration silicon infused strap pads
 Spooner patella stabilizing attachment* £65 charge
 C/S Package

Sizing

If your patient has proportional leg sizing (see sizing reference, below) or if you are ordering a brace for stock inventory please select from the size options.

X-Small Small Medium Large X-Large

Size	6" Above	M-L Knee	6" Below
XS	12.5 - 15.5"	3 - 3.5"	11 - 12.5"
SM	15.5 - 18.5"	3.5 - 4"	12.25 - 13.25"
MD	18.5 - 21"	4 - 4.5"	13.25 - 15"
LG	21 - 23.5"	4.5 - 5"	14.25 - 15.75"
XL	23.5 - 25"	5 - 5.5"	15 - 17"
XXL	25 - 28"	5.5 - 6"	17 - 19"
XXXL*	28 - 31"	6 - 6.5"	19 - 21"

*XXXL available on Rebel Pro only

Measurement Data


Required for "custom-fitted" Assembly*

Custom-fitted (Fabricated from three leg measurements)*

If your patient has a disproportionate leg, Thuasne USA will customize the assembly of your patient's brace. Please provide leg measurements beside the illustration, below.

More than one size difference between thigh and calf may require a full custom knee brace - please refer to the SpryStep Knee Brace

- _____ Circumference 6 inches above mid-patella
- _____ Medial-Lateral Knee Width (not circumference) at knee center
- _____ Circumference 6 inches below mid-patella

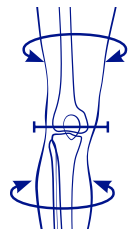

Special Instructions: _____

Thuasne UK Ltd

Unit 4 Orchard Business Centre, North Farm Road, Tunbridge Wells, TN2 3XF
 Tel: 01892 481620 - Fax: 01892 481609 - Email: orders@thuasne.co.uk
 www.thuasne.co.uk

Rebel Lock ORDER FORM

LIGAMENT BRACING SOLUTIONS

Account Contact Information

Clinician Name: _____ Clinician Email: _____ Clinician Phone: _____

Account Name: _____ Purchase Order Number: _____

For Thuasne Office Use Only

Account Number: _____ Order Number: _____

Purchase Order Number: _____ Purchase Order Number: _____

Patient Information

Fit Date: _____

Patient's Last Name: _____

Patient's First Name: _____

Male Female Age _____

Weight _____ (LBS) Height _____ (IN)

Leg: Left Right

Brace Configuration

Thigh Shell Length 7 Inch 8 Inch

Tibia Shell Length 7 Inch 8 Inch

Locking Position (if not indicated, zero degree lock position will be applied)

Zero Degree Locking Position

Five Degree Locking Position

Colour

Matte Finish

Black (Standard) Grey White

Atlantic (Light Blue) Red

Satin Finish

Lemon Orange Lime

Fuchsia Pacific (Dark Blue)

Accessories

Flexion Stop Kit* (installed upon request) £10 charge

30° 60° 90°

Extension assist bands/posts* £75 charge

Anti-migration silicon infused strap pads

C/S Package

Sizing

If your patient has proportional leg sizing (see sizing reference, below) or if you are ordering a brace for stock inventory please select from the size options.

X-Small Small Medium Large X-Large

Sizing Reference

Size	6" above	M-L width	6" below
XS	12.5" to 15.5"	3" to 3.5"	11" to 12.5"
S	15.5" to 18.5"	3.5" to 4"	12.25" to 13.75"
M	18.5" to 21"	4" to 4.5"	13.25" to 15"
L	21" to 23.5"	4.5" to 5"	14.25" to 15.75"
XL	23.5" to 25"	5" to 5.5"	15" to 17"

Measurement Data

Required for "custom-fitted" Assembly*

Custom-fitted (Fabricated from three leg measurements)*


If your patient has a disproportionate leg, Thuasne USA will customize the assembly of your patient's brace. Please provide leg measurements beside the illustration, below.

More than one size difference between thigh and calf may require a full custom knee brace - please refer to the SpryStep Knee Brace

_____ Circumference 6 inches above mid-patella

_____ Medial-Lateral Knee Width (not circumference) at knee center

_____ Circumference 6 inches below mid-patella

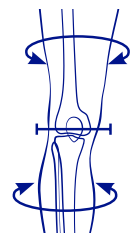

Special Instructions: _____

Thuasne UK Ltd

Unit 4 Orchard Business Centre, North Farm Road, Tunbridge Wells, TN2 3XF

Tel: 01892 481620 - Fax: 01892 481609 - Email: orders@thuasne.co.uk

www.thuasne.co.uk

THUASNE / 93

PRESCRIPTION FORM

**Sections marked with an asterisk
MUST be completed otherwise
manufacturing will not proceed.*

ADMINISTRATIVE DETAILS

Date: ____/____/____ Facility: _____ PO No.: _____

Orthotist E-mail: _____ Appointment Date: ____/____/____

END USER DETAILS

Client Reference: _____ Left Right

End User Relevant Medical History: _____

_____ Weight* _____ Height _____

SHAPE CAPTURE TECHNIQUE

Plaster/Fibreglass cast Direct Scan

OBJECTIVES

ACL PCL LCL MCL Meniscus Damage: Medial Lateral

SPRYSTEP KO


DEFAULT SPECIFICATION

- Custom to cast or digital scan
- Anterior shell configuration
- Anatomical contouring
- TF1 aluminium knee component
- Range adjustable bumper stoppers
- Replaceable strapping package
- Condyle pads
- Breathable liner
- 360mm height (180mm tibial and thigh)

_____ Circumference 6 inches
above mid-patella

_____ Medial-Lateral Knee Width (not
circumference) at knee center

_____ Circumference 6 inches
below mid-patella

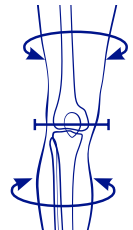

Knee contracture or ROM limitation (Full ROM is assumed): _____

STRAP OPTIONS

**Force changes to default style*

SHELL CONFIGURATION

Tibial: Posterior Tibia Chevron Tibia Full Shell In-Fill 25mm Extension 50mm Extension

Thigh: Posterior Thigh Full Shell In-Fill 25mm Extension 50mm Extension

STIFFNESS

Decrease structural rigidity

Increase structural rigidity

NOTES TO MANUFACTURER

STRAPPING OPTION

CS Package

PC Strap

Include spare set

Thuasne UK Ltd

Unit 4 Orchard Business Centre, North Farm Road, Tunbridge Wells, TN2 3XF

Tel: 01892 481620 - Fax: 01892 481609 - Email: orders@thuasne.co.uk

www.thuasne.co.uk

Rebel Reliever

ORDER FORM

LIGAMENT BRACING SOLUTIONS

Account Contact Information

Clinician Name: _____ Clinician Email: _____ Clinician Phone: _____

Account Name: _____ Purchase Order Number: _____

For Thuasne Office Use Only

Account Number: _____ Order Number: _____

Purchase Order Number: _____ Purchase Order Number: _____

Patient Information

Fit Date: _____

Patient's Last Name: _____

Patient's First Name: _____

Male Female Age _____

Weight _____ (LBS) Height _____ (IN)

Leg: Left Right

Compartment

Medial Compartment Lateral Compartment

Universal (select when ordering stock inventory)
Medial QA pads installed, Lateral QA pads included in kit.

Shell Length Thigh shell extends 7" to 9" on all braces

Tibia 6" 7" 8"

TM5+ Hinges Includes Extension Stops

Optional Flexion Stop Kit*

Colour

Matte Finish

Black (Standard) Grey White

Atlantic (Light Blue) Red

Satin Finish

Lemon Orange Lime

Fuchsia Pacific (Dark Blue)

Accessories

C/S Package

Sizing

If your patient has proportional leg sizing (see sizing reference, below) or if you are ordering a brace for stock inventory please select from the size options.

X-Small Small Medium Large X-Large

Size	6" Above	M-L Knee	6" Below
XS	NA	3 - 3.5"	11 - 12.5"
SM	15.5 - 18.5"	3.5 - 4"	12.25 - 13.25"
MD	18.5 - 21"	4 - 4.5"	13.25 - 15"
LG	21 - 23.5"	4.5 - 5"	14.25 - 15.75"
XL	23.5 - 25"	5 - 5.5"	15 - 17"
XXL	25 - 28"	5.5 - 6"	17 - 19"
XXXL*	28 - 31"	6 - 6.5"	19 - 21"

*XXXL available on Rebel Pro only

Measurement Data

Required for "custom-fitted" Assembly*

Custom-fitted (Fabricated from three leg measurements)*


If your patient has a disproportionate leg, Thuasne USA will customize the assembly of your patient's brace. Please provide leg measurements beside the illustration, below.

More than one size difference between thigh and calf may require a full custom knee brace - please refer to the SpryStep Knee Brace

_____ Circumference 6 inches above mid-patella

_____ Medial-Lateral Knee Width (not circumference) at knee center

_____ Circumference 6 inches below mid-patella

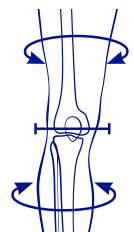

Special Instructions: _____

Thuasne UK Ltd

Unit 4 Orchard Business Centre, North Farm Road, Tunbridge Wells, TN2 3XF

Tel: 01892 481620 - Fax: 01892 481609 - Email: orders@thuasne.co.uk

www.thuasne.co.uk

THUASNE / 95

PRODUCT CODES

	PRODUCT DESCRIPTION	PRODUCT CODE
LOMBAFIRST	LombaFirst Lumbar Belt 21cm Black Size 1	08760220100399
	LombaFirst Lumbar Belt 21cm Black Size 2	08760220200399
	LombaFirst Lumbar Belt 21cm Black Size 3	08760220300399
	LombaFirst Lumbar Belt 21cm Black Size 4	08760220400399
	LombaFirst Lumbar Belt 21cm Black Size 5	08760220500399
	LombaFirst Lumbar Belt 26cm Black Size 1	08750220100399
	LombaFirst Lumbar Belt 26cm Black Size 2	08750220200399
	LombaFirst Lumbar Belt 26cm Black Size 3	08750220300399
	LombaFirst Lumbar Belt 26cm Black Size 4	08750220400399
	LombaFirst Lumbar Belt 26cm Black Size 5	08750220500399
LOMBASTART	LombaStart Lumbar Belt 21cm Black Size 1	08740220100399
	LombaStart Lumbar Belt 21cm Black Size 2	08740220200399
	LombaStart Lumbar Belt 21cm Black Size 3	08740220300399
	LombaStart Lumbar Belt 21cm Black Size 4	08740220400399
	LombaStart Lumbar Belt 21cm Black Size 5	08740220500399
	LombaStart Lumbar Belt 26cm Black Size 1	08730220100399
	LombaStart Lumbar Belt 26cm Black Size 2	08730220200399
	LombaStart Lumbar Belt 26cm Black Size 3	08730220300399
	LombaStart Lumbar Belt 26cm Black Size 4	08730220400399
	LombaStart Lumbar Belt 26cm Black Size 5	08730220500399
LOMBASKIN	LombaSkin Lumbar Belt Black 21cm Size 1	08710320109899
	LombaSkin Lumbar Belt Black 21cm Size 2	08710320209899
	LombaSkin Lumbar Belt Black 21cm Size 3	08710320309899
	LombaSkin Lumbar Belt Black 21cm Size 4	08710320409899
	LombaSkin Lumbar Belt Black 21cm Size 5	08710320509899
	LombaSkin Lumbar Belt Black 26cm Size 1	08700320109899
	LombaSkin Lumbar Belt Black 26cm Size 2	08700320209899
	LombaSkin Lumbar Belt Black 26cm Size 3	08700320309899
	LombaSkin Lumbar Belt Black 26cm Size 4	08700320409899
	LombaSkin Lumbar Belt Black 26cm Size 5	08700320509899
LOMBATECH	LombaTech Lumbar Belt 21cm Size 1	08520220100199
	LombaTech Lumbar Belt 21cm Size 2	08520220200199
	LombaTech Lumbar Belt 21cm Size 3	08520220300199
	LombaTech Lumbar Belt 21cm Size 4	08520220400199
	LombaTech Lumbar Belt 21cm Size 5	08520220500199
	LombaTech Lumbar Belt 26cm Size 1	08510320100199
	LombaTech Lumbar Belt 26cm Size 2	08510320200199
	LombaTech Lumbar Belt 26cm Size 3	08510320300199
	LombaTech Lumbar Belt 26cm Size 4	08510320400199
	LombaTech Lumbar Belt 26cm Size 5	08510320500199
	LombaMum Lumbar Maternity Belt	08050129905799
ORTEL P	Ortel P Pelvic Maternity Belt Size 1	27400120100099
	Ortel P Pelvic Maternity Belt Size 2	27400120200099
	Ortel P Pelvic Maternity Belt Size 3	27400120300099
	Ortel P Pelvic Maternity Belt Size 4	27400120400099
	Ortel P Pelvic Maternity Belt Size 5	27400120500099
	Ortel P Pelvic Maternity Belt Size 6	27400120600099

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
LOMBASTAB	LombaStab Lumbar Belt 21cm Size 1	08540220109899
	LombaStab Lumbar Belt 21cm Size 2	08540220209899
	LombaStab Lumbar Belt 21cm Size 3	08540220309899
	LombaStab Lumbar Belt 21cm Size 4	08540220409899
	LombaStab Lumbar Belt 21cm Size 5	08540220509899
	LombaStab Lumbar Belt 26cm Size 1	08530220109899
	LombaStab Lumbar Belt 26cm Size 2	08530220209899
	LombaStab Lumbar Belt 26cm Size 3	08530220309899
	LombaStab Lumbar Belt 26cm Size 4	08530220409899
	LombaStab Lumbar Belt 26cm Size 5	08530220509899
LOMBASTAB HIGH	LombaStab High Lumbar Belt 35cm Size 1	08550120109899
	LombaStab High Lumbar Belt 35cm Size 2	08550120209899
	LombaStab High Lumbar Belt 35cm Size 3	08550120309899
	LombaStab High Lumbar Belt 35cm Size 4	08550120409899
	LombaStab High Lumbar Belt 35cm Size 5	08550120509899
	LombaStab High Lumbar Belt 40cm Size 1	08560120109899
	LombaStab High Lumbar Belt 40cm Size 2	08560120209899
	LombaStab High Lumbar Belt 40cm Size 3	08560120309899
	LombaStab High Lumbar Belt 40cm Size 4	08560120409899
	LombaStab High Lumbar Belt 40cm Size 5	08560120509899
LOMBASTAB IMMO BELT	LombaStab Immo Belt Size 1	08590120109899
	LombaStab Immo Belt Size 2	08590120209899
	LombaStab Immo Belt Size 3	08590120309899
	LombaStab Immo Belt Size 4	08590120409899
	LombaStab Immo Belt Size 5	08590120509899
LOMBASTAB DORSO	LombaStab Dorso Size 1	08470120109899
	LombaStab Dorso Size 2	08470120209899
	LombaStab Dorso Size 3	08470120309899
	LombaStab Dorso Size 4	08470120409899
	LombaStab Dorso Size 5	08470120509899
	Sleeq Flex	U50401004001
	Sleeq Max TLSO	U50401007001
BOA DUEL TLSO	BOA DUEL TLSO Large	U5030120499999
	BOA DUEL TLSO Medium	U5030120399999
	BOA DUEL TLSO Small	U5030120299999
	BOA DUEL TLSO Xlarge	U5030120599999
	BOA DUEL TLSO XXLlarge	U5030120699999
DORSO RIGID 35	Dorso Rigid 35 Spinal Hyper-extension Brace Size L	08900230600099
	Dorso Rigid 35 Spinal Hyper-extension Brace Size LS	08900230500099
	Dorso Rigid 35 Spinal Hyper-extension Brace Size M	08900230300099
	Dorso Rigid 35 Spinal Hyper-extension Brace Size S	08900230100099
	Dorso Rigid 35 Spinal Hyper-extension Brace Size SM	08900230200099
	Dorso Rigid 35 Spinal Hyper-extension Brace Size XL	08900230800099
CEMEN THORACIC	Cemen Thoracic Belt Small 18cm x 1.25m	28000218012500
	Cemen Thoracic Belt Large 18cm x 2.5m	28000218025000
	Cemen Thoracic Belt Medium 25cm x 1.25m	28000225012500
	Cemen Thoracic Belt X-Large 25cm x 2.5m	28000225025000

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
CEMEN THORACIC	Cemen Thoracic Belt Female Size 1	29000120100002
	Cemen Thoracic Belt Female Size 2	29000120200002
	Cemen Thoracic Belt Female Size 3	29000120300002
	Cemen Thoracic Belt Female Size 4	29000120400002
	Cemen Thoracic Belt Male Size 1	29000120100001
	Cemen Thoracic Belt Male Size 2	29000120200001
	Cemen Thoracic Belt Male Size 3	29000120300001
	Cemen Thoracic Belt Male Size 4	29000120400001
STOMEX ABDOMINAL	Stomex Abdominal Support Size 1	27000220100099
	Stomex Abdominal Support Size 2	27000220200099
	Stomex Abdominal Support Size 3	27000220300099
	Stomex Abdominal Support Size 4	27000220400099
DYNABELT SUPPORT	Dynabelt Support Belt White 16cm Size 1	70100720100099
	Dynabelt Support Belt White 16cm Size 2	70100720200099
	Dynabelt Support Belt White 16cm Size 3	70100720300099
	Dynabelt Support Belt White 16cm Size 4	70100720400099
	Dynabelt Support Belt White 16cm Size 5	70100720500099
	Dynabelt Support Belt White 26cm Size 1	70100820100099
	Dynabelt Support Belt White 26cm Size 2	70100820200099
	Dynabelt Support Belt White 26cm Size 3	70100820300099
	Dynabelt Support Belt White 26cm Size 4	70100820400099
	Dynabelt Support Belt White 26cm Size 5	70100820500099
	Dynabelt Support Belt White 33cm Size 1	70100920100099
	Dynabelt Support Belt White 33cm Size 2	70100920200099
	Dynabelt Support Belt White 33cm Size 3	70100920300099
	Dynabelt Support Belt White 33cm Size 4	70100920400099
	Dynabelt Support Belt White 33cm Size 5	70100920500099
	Dynabelt Support Belt White 33cm Size 6	70100920600099
ORTEL HERNIA	Ortel Hernia Belt Bilateral Size 1	27200120100199
	Ortel Hernia Belt Bilateral Size 2	27200120200199
	Ortel Hernia Belt Bilateral Size 3	27200120300199
	Ortel Hernia Belt Bilateral Size 4	27200120400199
	Ortel Hernia Belt Unilateral Size 1	27300120100199
	Ortel Hernia Belt Unilateral Size 2	27300120200199
	Ortel Hernia Belt Unilateral Size 3	27300120300199
	Ortel Hernia Belt Unilateral Size 4	27300120400199
ORTEL C1 CLASSIC	Ortel C1 Classic Cervical Collar 10cm Size 1	23930220101000
	Ortel C1 Classic Cervical Collar 10cm Size 2	23930220201000
	Ortel C1 Classic Cervical Collar 10cm Size 3	23930220301000
	Ortel C1 Classic Cervical Collar 8cm Size 1	23930220100800
	Ortel C1 Classic Cervical Collar 8cm Size 2	23930220200800
	Ortel C1 Classic Cervical Collar 8cm Size 3	23930220300800

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
ORTEL C1 ANATOMIC	Ortel C1 Anatomic Cervical Collar Beige 11cm Size 1	23940220101102
	Ortel C1 Anatomic Cervical Collar Beige 11cm Size 2	23940220201102
	Ortel C1 Anatomic Cervical Collar Beige 11cm Size 3	23940220301102
	Ortel C1 Anatomic Cervical Collar Beige 7.5cm Size 1	23940220100702
	Ortel C1 Anatomic Cervical Collar Beige 7.5cm Size 2	23940220200702
	Ortel C1 Anatomic Cervical Collar Beige 7.5cm Size 3	23940220300702
	Ortel C1 Anatomic Cervical Collar Beige 9cm Size 1	23940220100902
	Ortel C1 Anatomic Cervical Collar Beige 9cm Size 2	23940220200902
	Ortel C1 Anatomic Cervical Collar Beige 9cm Size 3	23940220300902
	Ortel C1 Anatomic Cervical Collar Blue 11cm Size 1	23940220101104
	Ortel C1 Anatomic Cervical Collar Blue 11cm Size 2	23940220201104
	Ortel C1 Anatomic Cervical Collar Blue 11cm Size 3	23940220301104
	Ortel C1 Anatomic Cervical Collar Blue 7.5cm Size 1	23940220100704
	Ortel C1 Anatomic Cervical Collar Blue 7.5cm Size 2	23940220200704
	Ortel C1 Anatomic Cervical Collar Blue 7.5cm Size 3	23940220300704
	Ortel C1 Anatomic Cervical Collar Blue 9cm Size 1	23940220100904
	Ortel C1 Anatomic Cervical Collar Blue 9cm Size 2	23940220200904
	Ortel C1 Anatomic Cervical Collar Blue 9cm Size 3	23940220300904
ORTEL C2 PLUS	Ortel C2 Plus Cervical Collar Beige 11cm Size 1	23920420101102
	Ortel C2 Plus Cervical Collar Beige 11cm Size 2	23920420201102
	Ortel C2 Plus Cervical Collar Beige 11cm Size 3	23920420301102
	Ortel C2 Plus Cervical Collar Beige 7.5cm Size 1	23920420100702
	Ortel C2 Plus Cervical Collar Beige 7.5cm Size 2	23920420200702
	Ortel C2 Plus Cervical Collar Beige 7.5cm Size 3	23920420300702
	Ortel C2 Plus Cervical Collar Beige 9cm Size 1	23920420100902
	Ortel C2 Plus Cervical Collar Beige 9cm Size 2	23920420200902
	Ortel C2 Plus Cervical Collar Beige 9cm Size 3	23920420300902
	Ortel C2 Plus Cervical Collar Navy 11cm Size 1	23920420101104
	Ortel C2 Plus Cervical Collar Navy 11cm Size 2	23920420201104
	Ortel C2 Plus Cervical Collar Navy 11cm Size 3	23920420301104
	Ortel C2 Plus Cervical Collar Navy 7.5cm Size 1	23920420100704
	Ortel C2 Plus Cervical Collar Navy 7.5cm Size 2	23920420200704
	Ortel C2 Plus Cervical Collar Navy 7.5cm Size 3	23920420300704
	Ortel C2 Plus Cervical Collar Navy 9cm Size 1	23920420100904
	Ortel C2 Plus Cervical Collar Navy 9cm Size 2	23920420200904
	Ortel C2 Plus Cervical Collar Navy 9cm Size 3	23920420300904
ORTEL C3	Ortel C3 Cervical Collar 10cm Size 1	23910220101003
	Ortel C3 Cervical Collar 10cm Size 2	23910220201003
	Ortel C3 Cervical Collar 10cm Size 3	23910220301003
	Ortel C3 Classic Cervical Collar Size 1	23900120100099
	Ortel C3 Classic Cervical Collar Size 2	23900120200099
	Ortel C3 Classic Cervical Collar Size 3	23900120300099
Ortel T-Neck	T49100199999911	
Ortel T-Neck Replacement Pads	T491011	
Ortel T-Neck Plus	T49105199999911	
Ortel T-Neck Plus Replacement Pads	T491061	

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
IMMO CLASSIC	Immo Classic Shoulder Immobiliser Size 1	24450120100399
	Immo Classic Shoulder Immobiliser Size 2	24450120200399
	Immo Classic Shoulder Immobiliser Size 3	24450120300399
	Immo Classic Shoulder Immobiliser Size 4	24450120400399
	Immo Vest Size 1	24410220100399
	Immo Vest Size 2	24410220200399
IMMO ELBOW	Immo Elbow Immobiliser Size 1	24400120100399
	Immo Elbow Immobiliser Size 2	24400120200399
	Immo Elbow Immobiliser Size 3	24400120300399
	Immo Elbow Immobiliser Size 4	24400120400399
	Ligaflex Clavicular Straps	24500229900399
SILISTAB EPI	Silistab Epi Elbow Support Size 1	23050320100199
	Silistab Epi Elbow Support Size 2	23050320200199
	Silistab Epi Elbow Support Size 3	23050320300199
	Silistab Epi Elbow Support Size 4	23050320400199
	Silistab Epi Elbow Support Size 5	23050320500199
	Silistab Epi Elbow Support Size 6	23050320600199
EPIMED BLUE WHITE	Epimed Blue White Clasp L	T48001320401199
	Epimed Blue White Clasp M	T48001320301199
	Epimed Blue White Clasp S	T48001320201199
	Epimed Blue White Clasp XL	T48001320501199
	Epimed Blue White Clasp XS	T48001320101199
EPIMED PROMASTER	Epimed Promaster Elbow Clasp L	T35087320401299
	Epimed Promaster Elbow Clasp M	T35087320301299
	Epimed Promaster Elbow Clasp S	T35087320201299
	Epimed Promaster Elbow Clasp XL	T35087320501299
	Epimed Promaster Elbow Clasp XS	T35087320101299
LIGAFLEX ACTION	Ligaflex Action Size 1 Left	24360220100102
	Ligaflex Action Size 1 Right	24360220100101
	Ligaflex Action Size 2 Left	24360220200102
	Ligaflex Action Size 2 Right	24360220200101
	Ligaflex Action Size 3 Left	24360220300102
	Ligaflex Action Size 3 Right	24360220300101
	Ligaflex Action Size 4 Left	24360220400102
	Ligaflex Action Size 4 Right	24360220400101
	Ligaflex Action Size 5 Left	24360220500102
	Ligaflex Action Size 5 Right	24360220500101
	Ligaflex Action Size 6 Left	24360220600102
	Ligaflex Action Size 6 Right	24360220600101
LIGAFLEX CLASSIC	Ligaflex Classic Wrist Brace Black Left Size 0	24350220000302
	Ligaflex Classic Wrist Brace Black Left Size 1	24350220100302
	Ligaflex Classic Wrist Brace Black Left Size 2	24350220200302
	Ligaflex Classic Wrist Brace Black Left Size 3	24350220300302
	Ligaflex Classic Wrist Brace Black Left Size 4	24350220400302
	Ligaflex Classic Wrist Brace Black Right Size 0	24350220000301
	Ligaflex Classic Wrist Brace Black Right Size 1	24350220100301
	Ligaflex Classic Wrist Brace Black Right Size 2	24350220200301
	Ligaflex Classic Wrist Brace Black Right Size 3	24350220300301
	Ligaflex Classic Wrist Brace Black Right Size 4	24350220400301

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
	Ligaflex Pro Wrist Brace Size 1	24330220100199
	Ligaflex Pro Wrist Brace Size 2	24330220200199
LIGAFLEX PRO+	Ligaflex Pro+ Wrist Brace Left Size 1	24340220100102
	Ligaflex Pro+ Wrist Brace Left Size 2	24340220200102
	Ligaflex Pro+ Wrist Brace Left Size 3	24340220300102
	Ligaflex Pro+ Wrist Brace Left Size 4	24340220400102
	Ligaflex Pro+ Wrist Brace Right Size 1	24340220100101
	Ligaflex Pro+ Wrist Brace Right Size 2	24340220200101
	Ligaflex Pro+ Wrist Brace Right Size 3	24340220300101
	Ligaflex Pro+ Wrist Brace Right Size 4	24340220400101
LIGAFLEX MANU	Ligaflex Manu Wrist Brace Black Left Size 0	24300220000302
	Ligaflex Manu Wrist Brace Black Left Size 1	24300220100302
	Ligaflex Manu Wrist Brace Black Left Size 2	24300220200302
	Ligaflex Manu Wrist Brace Black Left Size 3	24300220300302
	Ligaflex Manu Wrist Brace Black Left Size 4	24300220400302
	Ligaflex Manu Wrist Brace Black Right Size 0	24300220000301
	Ligaflex Manu Wrist Brace Black Right Size 1	24300220100301
	Ligaflex Manu Wrist Brace Black Right Size 2	24300220200301
	Ligaflex Manu Wrist Brace Black Right Size 3	24300220300301
	Ligaflex Manu Wrist Brace Black Right Size 4	24300220400301
LIGAFLEX RHIZO	Ligaflex Rhizo Thumb Splint Left Size 1	70900220101402
	Ligaflex Rhizo Thumb Splint Left Size 2	70900220201402
	Ligaflex Rhizo Thumb Splint Right Size 1	70900220101401
	Ligaflex Rhizo Thumb Splint Right Size 2	70900220201401
	Rhizo Pro Wrist and Thumb Brace Black size 1	T48027320102799
	Rhizo Pro Wrist and Thumb Brace Black size 2	T48027320202799
LIGAFLEX FINGER	Ligaflex Finger Splint Size 1	24320220100399
	Ligaflex Finger Splint Size 2	24320220200399
	Ligaflex Finger Splint Size 3	24320220300399
	Ligaflex Finger Splint Size 4	24320220400399
LIGAFLEX BOXER	Ligaflex Boxer Wrist Brace Size 1 Left	24310220109502
	Ligaflex Boxer Wrist Brace Size 1 Right	24310220109501
	Ligaflex Boxer Wrist Brace Size 2 Left	24310220209502
	Ligaflex Boxer Wrist Brace Size 2 Right	24310220209501
	Ligaflex Boxer Wrist Brace Size 3 Left	24310220309502
	Ligaflex Boxer Wrist Brace Size 3 Right	24310220309501
	Ligaflex Boxer Wrist Brace Size 4 Left	24310220409502
	Ligaflex Boxer Wrist Brace Size 4 Right	24310220409501
GENUSOFT	Genusoft Knee Support Size 1	23200320100199
	Genusoft Knee Support Size 2	23200320200199
	Genusoft Knee Support Size 3	23200320300199
	Genusoft Knee Support Size 4	23200320400199
	Genusoft Knee Support Size 5	23200320500199
GENUACTION	Genuaction Knee Support Size 0	23250320000199
	Genuaction Knee Support Size 1	23250320100199
	Genuaction Knee Support Size 2	23250320200199
	Genuaction Knee Support Size 3	23250320300199
	Genuaction Knee Support Size 4	23250320400199
	Genuaction Knee Support Size 5	23250320500199
	Genuaction Knee Support Size 6	23250320600199

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
GENUEXTREM	Genuextrem Knee Support Size 1	23210220100199
	Genuextrem Knee Support Size 2	23210220200199
	Genuextrem Knee Support Size 3	23210220300199
	Genuextrem Knee Support Size 4	23210220400199
	Genuextrem Knee Support Size 5	23210220500199
	Genuextrem Knee Support Size 6	23210220600199
LIGASTRAP GENU	Ligastrap Genu Knee Brace Size 1	21700320100199
	Ligastrap Genu Knee Brace Size 2	21700320200199
	Ligastrap Genu Knee Brace Size 3	21700320300199
	Ligastrap Genu Knee Brace Size 4	21700320400199
	Ligastrap Genu Knee Brace Size 5	21700320500199
	Ligastrap Genu Knee Brace Size 6	21700320600199
SILISTAB GENU CLOSED PATELLA	Silistab Genu Closed Patella Knee Brace Black Size 1	23450420100399
	Silistab Genu Closed Patella Knee Brace Black Size 2	23450420200399
	Silistab Genu Closed Patella Knee Brace Black Size 3	23450420300399
	Silistab Genu Closed Patella Knee Brace Black Size 4	23450420400399
	Silistab Genu Closed Patella Knee Brace Black Size 5	23450420500399
	Silistab Genu Closed Patella Knee Brace Black Size 6	23450420600399
	Silistab Genu Open Patella Knee Brace White Size 1	23400520100099
	Silistab Genu Open Patella Knee Brace White Size 2	23400520200099
	Silistab Genu Open Patella Knee Brace White Size 3	23400520300099
	Silistab Genu Open Patella Knee Brace White Size 4	23400520400099
	Silistab Genu Open Patella Knee Brace White Size 5	23400520500099
	Silistab Genu Open Patella Knee Brace White Size 6	23400520600099
PATELLA RELIEVER	Patella Reliever Left Size 1	23480120100102
	Patella Reliever Left Size 2	23480120200102
	Patella Reliever Left Size 3	23480120300102
	Patella Reliever Left Size 4	23480120400102
	Patella Reliever Left Size 5	23480120500102
	Patella Reliever Left Size 6	23480120600102
	Patella Reliever Right Size 1	23480120100101
	Patella Reliever Right Size 2	23480120200101
	Patella Reliever Right Size 3	23480120300101
	Patella Reliever Right Size 4	23480120400101
	Patella Reliever Right Size 5	23480120500101
	Patella Reliever Right Size 6	23480120600101
GENU DYNASTAB	Genu Dynastab Knee Brace Size 1	23700620100199
	Genu Dynastab Knee Brace Size 2	23700620200199
	Genu Dynastab Knee Brace Size 3	23700620300199
	Genu Dynastab Knee Brace Size 4	23700620400199
	Genu Dynastab Knee Brace Size 5	23700620500199
	Genu Dynastab Knee Brace Size 6	23700620600199
	Patellar Bandage	23040129900399

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
GENU LIGAFLEX	Genu Ligaflex Knee Brace Short Sleeve Size 1	23750620100101
	Genu Ligaflex Knee Brace Short Sleeve Size 2	23750620200101
	Genu Ligaflex Knee Brace Short Sleeve Size 3	23750620300101
	Genu Ligaflex Knee Brace Short Sleeve Size 4	23750620400101
	Genu Ligaflex Knee Brace Short Sleeve Size 5	23750620500101
	Genu Ligaflex Knee Brace Short Sleeve Size 6	23750620600101
	Genu Ligaflex Knee Brace Short Wrap Size 1	23750520100101
	Genu Ligaflex Knee Brace Short Wrap Size 2	23750520100102
	Genu Ligaflex Knee Brace Short Wrap Size 3	23750520200101
	Genu Ligaflex Knee Brace Short Wrap Size 4	23750520200102
	Genu Ligaflex Knee Brace Short Wrap Size 5	23750520300101
	Genu Ligaflex Knee Brace Short Wrap Size 6	23750520300102
	Genu Ligaflex Knee Brace Long Wrap Size 1	23750520400101
	Genu Ligaflex Knee Brace Long Wrap Size 2	23750520400102
	Genu Ligaflex Knee Brace Long Wrap Size 3	23750520500101
	Genu Ligaflex Knee Brace Long Wrap Size 4	23750520500102
	Genu Ligaflex Knee Brace Long Wrap Size 5	23750520600101
	Genu Ligaflex Knee Brace Long Wrap Size 6	23750520600102
GENU LIGAFLEX ROM	Genu Ligaflex ROM Knee Brace Long Wrap Size 1	23860520100102
	Genu Ligaflex ROM Knee Brace Long Wrap Size 2	23860520200102
	Genu Ligaflex ROM Knee Brace Long Wrap Size 3	23860520300102
	Genu Ligaflex ROM Knee Brace Long Wrap Size 4	23860520400102
	Genu Ligaflex ROM Knee Brace Long Wrap Size 5	23860520500102
	Genu Ligaflex ROM Knee Brace Long Wrap Size 6	23860520600102
LIGAFLEX IMMO	Ligaflex Immo Knee Splint Size 1	24000320100499
	Ligaflex Immo Knee Splint Size 2	24000320200499
	Ligaflex Immo Knee Splint Size 3	24000320300499
	Ligaflex Immo Knee Splint Size 4	24000320400499
	ROM-R	U0460202300199
REBEL STANDARD	Rebel Standard Knee Brace Left XS	U0300220106502
	Rebel Standard Knee Brace Left S	U0300220206502
	Rebel Standard Knee Brace Left M	U0300220306502
	Rebel Standard Knee Brace Left L	U0300220406502
	Rebel Standard Knee Brace Left XL	U0300220506502
	Rebel Standard Knee Brace Right XS	U0300220106501
	Rebel Standard Knee Brace Right S	U0300220206501
	Rebel Standard Knee Brace Right M	U0300220306501
	Rebel Standard Knee Brace Right L	U0300220406501
	Rebel Standard Knee Brace Right XL	U0300220506501
	Rebel Standard Knee Brace Semi-Custom	U03001001001
	Rebel Standard Liners Kit Xsmall	UU00720199999
	Rebel Standard Liners Kit Small	UU00720299999
	Rebel Standard Liners Kit Medium	UU00720399999
	Rebel Standard Liners Kit Large	UU00720499999
	Rebel Standard Liners Kit Xlarge	UU00720599999
	Rebel Standard 14 Refurb Kit XSmall	UU00620199999
	Rebel Standard 14 Refurb Kit Small	UU00620299999
	Rebel Standard 14 Refurb Kit Medium	UU00620399999
	Rebel Standard 14 Refurb Kit Large	UU00620499999
Rebel Standard 14 Refurb Kit XLarge	UU00620599999	

PRODUCT CODES

	PRODUCT DESCRIPTION	PRODUCT CODE
REBEL PRO	Rebel Pro Knee Brace Left XS	U0310220106502
	Rebel Pro Knee Brace Left S	U0310220206502
	Rebel Pro Knee Brace Left M	U0310220306502
	Rebel Pro Knee Brace Left L	U0310220406502
	Rebel Pro Knee Brace Left XL	U0310220506502
	Rebel Pro Knee Brace Right XS	U0310220106501
	Rebel Pro Knee Brace Right S	U0310220206501
	Rebel Pro Knee Brace Right M	U0310220306501
	Rebel Pro Knee Brace Right L	U0310220406501
	Rebel Pro Knee Brace Right XL	U0310220506501
	Rebel Pro Knee Brace Semi-Custom	U03101001001
	Rebel Pro Liners Kit Xsmall	UU01120199999
	Rebel Pro Liners Kit Small	UU01120299999
	Rebel Pro Liners Kit Medium	UU01120399999
	Rebel Pro Liners Kit Large	UU01120499999
	Rebel Pro Liners Kit Xlarge	UU01120599999
	Rebel Pro Liners Kit XXLarge	UU01120699999
	Rebel Pro Refurb Kit XSmall	UU01020199999
	Rebel Pro Refurb Kit Small	UU01020299999
	Rebel Pro Refurb Kit Medium	UU01020399999
Rebel Pro Refurb Kit Large	UU01020499999	
Rebel Pro Refurb Kit XLarge	UU01020599999	
Rebel Pro Refurb Kit XXLarge	UU01020699999	
REBEL LOCK	Rebel Lock Knee Brace Left XS	U3020220106602
	Rebel Lock Knee Brace Left S	U3020220206602
	Rebel Lock Knee Brace Left M	U3020220306602
	Rebel Lock Knee Brace Left L	U3020220406602
	Rebel Lock Knee Brace Left XL	U3020220506602
	Rebel Lock Knee Brace Right XS	U3020220106601
	Rebel Lock Knee Brace Right S	U3020220206601
	Rebel Lock Knee Brace Right M	U3020220306601
	Rebel Lock Knee Brace Right L	U3020220406601
	Rebel Lock Knee Brace Right XL	U3020220506601
	Rebel Lock Knee Brace Semi-Custom	U30201001001
	Rebel Lock Liners Kit XS	UU02620199999
	Rebel Lock Liners Kit S	UU02620299999
	Rebel Lock Liners Kit M	UU02620399999
	Rebel Lock Liners Kit L	UU02620499999
	Rebel Lock Liners Kit XL	UU02620599999
	Rebel Lock Refurb Kit XSmall	UU02520199999
	Rebel Lock Refurb Kit Small	UU02520299999
	Rebel Lock Refurb Kit Medium	UU02520399999
	Rebel Lock Refurb Kit Large	UU02520499999
Rebel Lock Refurb Kit XLarge	UU02520599999	
SPRYSTEP	SPRYSTEP KO	MM3060060019901
	SPRYSTEP RELIEVER KO	MM3060060029901
	SPRYSTEP LOCKING KO	MM3060060039901

PRODUCT CODES

	PRODUCT DESCRIPTION	PRODUCT CODE
ACTION RELIEVER	Action Reliever Left Medial/Right Lateral Size 1	23490120100151
	Action Reliever Left Medial/Right Lateral Size 2	23490120200151
	Action Reliever Left Medial/Right Lateral Size 3	23490120300151
	Action Reliever Left Medial/Right Lateral Size 4	23490120400151
	Action Reliever Left Medial/Right Lateral Size 5	23490120500151
	Action Reliever Left Medial/Right Lateral Size 6	23490120600151
	Action Reliever Left Medial/Right Lateral Size 7	23490120700151
	Action Reliever Left Medial/Right Lateral Size 8	23490120800151
	Action Reliever Right Medial/Left Lateral Size 1	23490120100152
	Action Reliever Right Medial/Left Lateral Size 2	23490120200152
	Action Reliever Right Medial/Left Lateral Size 3	23490120300152
	Action Reliever Right Medial/Left Lateral Size 4	23490120400152
	Action Reliever Right Medial/Left Lateral Size 5	23490120500152
	Action Reliever Right Medial/Left Lateral Size 6	23490120600152
	Action Reliever Right Medial/Left Lateral Size 7	23490120700152
	Action Reliever Right Medial/Left Lateral Size 8	23490120800152
REBEL RELIEVER	Rebel Reliever OA Knee Brace Left XS	U0330920106602
	Rebel Reliever OA Knee Brace Left S	U0330920206602
	Rebel Reliever OA Knee Brace Left M	U0330920306602
	Rebel Reliever OA Knee Brace Left L	U0330920406602
	Rebel Reliever OA Knee Brace Left XL	U0330920506602
	Rebel Reliever OA Knee Brace Left XXL	U0330920606602
	Rebel Reliever OA Knee Brace Right XS	U0330920106601
	Rebel Reliever OA Knee Brace Right S	U0330920206601
	Rebel Reliever OA Knee Brace Right M	U0330920306601
	Rebel Reliever OA Knee Brace Right L	U0330920406601
	Rebel Reliever OA Knee Brace Right XL	U0330920506601
	Rebel Reliever OA Knee Brace Right XXL	U0330920606601
MALLEOSOFT	Malleosoft Ankle Support Size 1	36000220100199
	Malleosoft Ankle Support Size 2	36000220200199
	Malleosoft Ankle Support Size 3	36000220300199
	Malleosoft Ankle Support Size 4	36000220400199
	Malleosoft Ankle Support Size 5	36000220500199
MALLEOACTION	Malleoaction Ankle Brace Size 0	23610320000199
	Malleoaction Ankle Brace Size 1	23610320100199
	Malleoaction Ankle Brace Size 2	23610320200199
	Malleoaction Ankle Brace Size 3	23610320300199
	Malleoaction Ankle Brace Size 4	23610320400199
Malleoaction Ankle Brace Size 5	23610320500199	
SILISTAB MALLEO	Silistab Malleo Ankle Support Black Size 1	23650120100399
	Silistab Malleo Ankle Support Black Size 2	23650120200399
	Silistab Malleo Ankle Support Black Size 3	23650120300399
	Silistab Malleo Ankle Support Black Size 4	23650120400399
	Silistab Malleo Ankle Support Black Size 5	23650120500399
	Silistab Malleo Ankle Support Black Size 6	23650120600399
SILISTAB ACHILLO	Silistab Achillo Ankle Brace Size 1	23550320100199
	Silistab Achillo Ankle Brace Size 2	23550320200199
	Silistab Achillo Ankle Brace Size 3	23550320300199
	Silistab Achillo Ankle Brace Size 4	23550320400199
	Silistab Achillo Ankle Brace Size 5	23550320500199

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
MALLEO DYNASTAB	Malleo Dynastab Boa Ankle Brace Size 0	23510220000199
	Malleo Dynastab Boa Ankle Brace Size 1	23510220100199
	Malleo Dynastab Boa Ankle Brace Size 2	23510220200199
	Malleo Dynastab Boa Ankle Brace Size 3	23510220300199
	Malleo Dynastab Lace-up Ankle Brace Size 1	23500320100199
	Malleo Dynastab Lace-up Ankle Brace Size 2	23500320200199
	Malleo Dynastab Lace-up Ankle Brace Size 3	23500320300199
LIGASTRAP MALLEO	Ligastrap Malleo Ankle Wrap Size 1	21800420100199
	Ligastrap Malleo Ankle Wrap Size 2	21800420200199
	Ligastrap Malleo Ankle Wrap Size 3	21800420300199
	Ligastrap Malleo Ankle Wrap Size 4	21800420400199
	Ligastrap Malleo Ankle Wrap Size 5	21800420500199
XLR8	XLR8 Pneumatic Short Walker XS	U0900320100399
	XLR8 Pneumatic Short Walker S	U0900320200399
	XLR8 Pneumatic Short Walker M	U0900320300399
	XLR8 Pneumatic Short Walker L	U0900320400399
	XLR8 Pneumatic Short Walker XL	U0900320500399
	XLR8 Pneumatic Walker XS	U0890320100399
	XLR8 Pneumatic Walker S	U0890320200399
	XLR8 Pneumatic Walker M	U0890320300399
	XLR8 Pneumatic Walker L	U0890320400399
	XLR8 Pneumatic Walker XL	U0890320500399
	XLR8 Heel Wedges Left	UU99301001003
	XLR8 Heel Wedges Right	UU99301001002
	XLR8 Replacement Liner Large	UU08920499999
	XLR8 Replacement Liner Medium	UU08920399999
	XLR8 Replacement Liner Small	UU08920299999
	XLR8 Replacement Liner Xlarge	UU08920599999
	XLR8 Replacement Liner Xsmall	UU08920199999
LIGASTRAP IMMO	Ligastrap Immo Ankle Stirrup Left Size 0	23370220000302
	Ligastrap Immo Ankle Stirrup Left Size 1	23370220100302
	Ligastrap Immo Ankle Stirrup Right Size 0	23370220000301
	Ligastrap Immo Ankle Stirrup Right Size 1	23370220100301
	Ligacast Air+ Ankle Stirrup	23340129900099
SPRYSTEP FLEX	Sprystep Flex Extra Small Left	U0171260005002
	Sprystep Flex Extra Small Right	U0171260005001
	Sprystep Flex Small Left	U0171260105002
	Sprystep Flex Small Right	U0171260105001
	Sprystep Flex Medium Left	U0171260205002
	Sprystep Flex Medium Right	U0171260205001
	Sprystep Flex Large Left	U0171260305002
	Sprystep Flex Large Right	U0171260305001
	Sprystep Flex Extra Large Left	U0171260405002
	Sprystep Flex Extra Large Right	U0171260405001

PRODUCT CODES

	PRODUCT DESCRIPTION	PRODUCT CODE
SPRYSTEP ORIGINAL	Sprystep Original Extra Small Left	U0170360005002
	Sprystep Original Extra Small Right	U0170360005001
	Sprystep Original Small Left	U0170360105002
	Sprystep Original Small Right	U0170360105001
	Sprystep Original Medium Left	U0170360205002
	Sprystep Original Medium Right	U0170360205001
	Sprystep Original Large Left	U0170360305002
	Sprystep Original Large Right	U0170360305001
	Sprystep Original Extra Large Left	U0170360405002
	Sprystep Original Extra Large Right	U0170360405001
SPRYSTEP PLUS	Sprystep Plus Extra Small Left	U0172260005001
	Sprystep Plus Extra Small Right	U0172160005001
	Sprystep Plus Small Left	U0172260005002
	Sprystep Plus Small Right	U0172260105001
	Sprystep Plus Medium Left	U0172260205002
	Sprystep Plus Medium Right	U0172260205001
	Sprystep Plus Large Left	U0172260305002
	Sprystep Plus Large Right	U0172260305001
	Sprystep Plus Extra Large Left	U0172260405002
	Sprystep Plus Extra Large RIGHT	U0172260405001
SPRYSTEP MAX	Sprystep Max Extra Small Left	U0173260005002
	Sprystep Max Extra Small Right	U0173260005001
	Sprystep Max Small Left	U0173260105002
	Sprystep Max Small Right	U0173260105001
	Sprystep Max Medium Left	U0173260205002
	Sprystep Max Medium Right	U0173260205001
	Sprystep Max Large Left	U0173260305002
	Sprystep Max Large Right	U0173260305001
	Sprystep Max Extra Large Left	U0173260405002
	Sprystep Max Extra Large Right	U0173260405001
SPRYSTEP STRAP	Sprystep Plus Strap Kit Small - Medium	UU017220102
	Sprystep Plus Strap Kit Large - Extra Large	UU017220302
	Sprystep Max Strap Kit Right Extra Small	UU017320001
	Sprystep Max Strap Kit Right Small	UU017320101
	Sprystep Max Strap Kit Right Medium	UU017320201
	Sprystep Max Strap Kit Right Large	UU017320301
	Sprystep Max Strap Kit Right Extra Large	UU017320401
	Sprystep Max Strap Kit Left Extra Small	UU017320002
	Sprystep Max Strap Kit Left Small	UU017320102
	Sprystep Max Strap Kit Left Medium	UU017320202
	Sprystep Max Strap Kit Left Large	UU017320302
	Sprystep Max Strap Kit Left Extra Large	UU017320402
PERO-MED AFO	Pero-Med AFO Left S	T47965220203302
	Pero-Med AFO Right S	T47965220203301
	Pero-Med AFO Left M	T47965220303302
	Pero-Med AFO Right M	T47965220303301
	Pero-Med AFO Left L	T47965220403302
	Pero-Med AFO Right L	T47965220403301
	Pero-Med AFO Left XL	T47965220503302
	Pero-Med AFO Right XL	T47965220503301

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
PEDO-MED FOREFOOT	Podo-Med Forefoot Long Toe Ridge Shoe XSmall	T50040120102799
	Podo-Med Forefoot Long Toe Ridge Shoe Small	T50040120202799
	Podo-Med Forefoot Long Toe Ridge Shoe Medium	T50040120302799
	Podo-Med Forefoot Long Toe Ridge Shoe Large	T50040120402799
	Podo-Med Forefoot Long Toe Ridge Shoe XLarge	T50040120502799
PEDO-MED OPEN HEEL SHOE/PLUS	Podo-Med Open Heel Shoe Small	T50050120202799
	Podo-Med Open Heel Shoe Medium	T50050120302799
	Podo-Med Open Heel Shoe Large	T50050120402799
	Podo-Med Open Heel Shoe XLarge	T50050120502799
	Podo-Med Open Heel Shoe Plus Cover Small	T50051120202799
	Podo-Med Open Heel Shoe Plus Cover Medium	T50051120302799
	Podo-Med Open Heel Shoe Plus Cover Large	T50051120402799
	Podo-Med Open Heel Shoe Plus Cover XLarge	T50051120502799
PEDO-MED FOREFOOT S	Podo-Med Forefoot Short Shoe Small	T50030120202799
	Podo-Med Forefoot Short Shoe Medium	T50030120302799
	Podo-Med Forefoot Short Shoe Large	T50030120402799
	Podo-Med Forefoot Short Shoe XLarge	T50030120502799
PEDO-MED FOREFOOT L	Podo-Med Forefoot Long Shoe XSmall	T50045120102799
	Podo-Med Forefoot Long Shoe Small	T50045120202799
	Podo-Med Forefoot Long Shoe Medium	T50045120302799
	Podo-Med Forefoot Long Shoe Large	T50045120402799
	Podo-Med Forefoot Long Shoe XLarge	T50045120502799
PEDO-MED OFFLOADING	Podo-Med Forefoot Offloading Shoe XSmall	T50047120102799
	Podo-Med Forefoot Offloading Shoe Small	T50047120202799
	Podo-Med Forefoot Offloading Shoe Medium	T50047120302799
	Podo-Med Forefoot Offloading Shoe Large	T50047120402799
	Podo-Med Forefoot Offloading Shoe XLarge	T50047120502799
PEDIPRO	Pedipro Heel Insert Size 1	65000120199999
	Pedipro Heel Insert Size 2	65000120299999
	Pedipro Insole Size 1	65100120199999
	Pedipro Insole Size 2	65100120299999
	Pedipro Insole Size 3	65100120399999
	Pedipro Insole Size 4	65100120499999
PEDIPRO PLUS	Pedipro Plus Heel Inserts Size 1	66000120199999
	Pedipro Plus Heel Inserts Size 2	66000120299999
	Pedipro Plus Insoles Size 1	66100120199999
	Pedipro Plus Insoles Size 2	66100120299999
	Pedipro Plus Insoles Size 3	66100120399999
	Pedipro Plus Insoles Size 4	66100120499999
	Pedipro Softer Heel Insert Size 1	66050120100599
	Pedipro Softer Heel Insert Size 2	66050120200599
ELASTOCALX N	Elastocalx N X-Small	T47980120002999
	Elastocalx N Small	T47980120102999
	Elastocalx N Medium	T47980120202999
	Elastocalx N Large	T47980120302999
	Elastocalx N X-Large	T47980120402999
ELASTOCALX S/P	Elastocalx S/P X-Small	T47981120002999
	Elastocalx S/P Small	T47981120102999
	Elastocalx S/P Medium	T47981120202999
	Elastocalx S/P Large	T47981120302999
	Elastocalx S/P X-Large	T47981120402999

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
SOHLI	Sohli Insoles (Pair) Size 19/20	T47984102002399
	Sohli Insoles (Pair) Size 21/22	T47984102202399
	Sohli Insoles (Pair) Size 23/24	T47984102402399
	Sohli Insoles (Pair) Size 25/26	T47984102602399
	Sohli Insoles (Pair) Size 27/28	T47984102802399
	Sohli Insoles (Pair) Size 29/30	T47984103002399
	Sohli Insoles (Pair) Size 31/32	T47984103202399
	Sohli Insoles (Pair) Size 33/34	T47984103402399
	Sohli Insoles (Pair) Size 35/36	T47984103602399
	Sohli Insoles (Pair) Size 37/38	T47984103802399

PAEDIACTRICS

PRODUCT DESCRIPTION		PRODUCT CODE
	Ligaflex Classic Wrist Brace Black Left Size 0	24350220000302
	Ligaflex Classic Wrist Brace Black Right Size 0	24350220000301
	Ligaflex Manu Wrist Brace Black Left Size 0	24300220000302
	Ligaflex Manu Wrist Brace Black Right Size 0	24300220000301
	Ligaflex Junior Clavicular Straps	26400229900499
	Genuaction Knee Support Size 0	23250320000199
	Ligaflex Immo Knee Splint Size 0	24100450104004
	Malleoaction Ankle Brace Size 0	23610320000199
	Malleo Dynastab Boa Ankle Brace Size 0	23510220000199
	Ligacast Junior	2620001004
LIGASTEP	LIGASTEP PEDI WALKER S	U0920120200199
	LIGASTEP PEDI WALKER M	U0920120300199
	LIGASTEP PEDI WALKER L	U0920120400199
SPRYSTEP PAEDIATRIC	Sprystep Paediatric Left XS	U0175260100502
	Sprystep Paediatric Left S	U0175260200502
	Sprystep Paediatric Left M	U0175260300502
	Sprystep Paediatric Left L	U0175260400502
	Sprystep Paediatric Left XL	U0175260000501
	Sprystep Paediatric Right XS	U0175260100501
	Sprystep Paediatric Right S	U0175260200501
	Sprystep Paediatric Right M	U0175260300501
	Sprystep Paediatric Right L	U0175260400501
	Sprystep Paediatric Right XL	U0175260400501
	Sprystep Paediatric Kit XS - M	UU017520003
	Sprystep Paediatric Kit L - XL	UU017520004

THUASNE SPORT

PRODUCT DESCRIPTION		PRODUCT CODE
LUMBAR	Thuasne Sport Lumbar Support Belt Small	03610120100399
	Thuasne Sport Lumbar Support Belt Medium	03610120200399
	Thuasne Sport Lumbar Support Belt Large	03610120300399
	Thuasne Sport Lumbar Support Belt X-Large	03610120400399
	Thuasne Sport Lumbar Support Belt XX-Large	03610120500399
NEO-PRENE	Thuasne Sport Neoprene Shoulder Strapping Small	05760320100399
	Thuasne Sport Neoprene Shoulder Strapping Medium	05760320200399
	Thuasne Sport Neoprene Shoulder Strapping Large	05760320300399

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
ANTI-EPICONDYLITIS	Thuasne Sport Anti-Epicondylitis Brace Small	03360320100199
	Thuasne Sport Anti-Epicondylitis Brace Medium	03360320200199
	Thuasne Sport Anti-Epicondylitis Brace Large	03360320300199
	Thuasne Sport Anti-Epicondylitis Brace X-Large	03360320400199
TENNIS ELBOW	Thuasne Sport Elbow Support Small	03380120100199
	Thuasne Sport Elbow Support Medium	03380120200199
	Thuasne Sport Elbow Support Large	03380120300199
	Thuasne Sport Elbow Support X-Large	03380120400199
	Thuasne Sport Tennis Elbow Armband Size 1	03520220100199
	Thuasne Sport Tennis Elbow Armband Size 2	03520220200199
THUMB	Thuasne Sport Thumb Strap Small	03320420100399
	Thuasne Sport Thumb Strap Medium	03320420200399
	Thuasne Sport Thumb Strap Large	03320420300399
	Thuasne Sport Strapping Wrist Band	03400229900399
	Thuasne Sport Neoprene Wrist Support	05750329900399
	Thuasne Sport Wrist Immobilisation Brace Size 1	03560120100199
	Thuasne Sport Wrist Immobilisation Brace Size 2	03560120200199
	Thuasne Sport Patella Bandage	03290129900199
NEOPRENE THIGH	Thuasne Sport Neoprene Thigh Support Small	05720320100399
	Thuasne Sport Neoprene Thigh Support Medium	05720320200399
	Thuasne Sport Neoprene Thigh Support Large	05720320300399
	Thuasne Sport Neoprene Thigh Support X-Large	05720320400399
NEOPRENE KNEE	Thuasne Sport Neoprene Knee Support Small	05730320100399
	Thuasne Sport Neoprene Knee Support Medium	05730320200399
	Thuasne Sport Neoprene Knee Support Large	05730320300399
	Thuasne Sport Neoprene Knee Support X-Large	05730320400399
	Thuasne Sport Neoprene Knee Support XX-Large	05730320500399
KNEE	Thuasne Sport Knee Support Small	03340320100199
	Thuasne Sport Knee Support Medium	03340320200199
	Thuasne Sport Knee Support Large	03340320300199
	Thuasne Sport Knee Support X-Large	03340320400199
	Thuasne Sport Knee Support XX-Large	03340320500199
OPEN KNEE	Thuasne Sport Open Knee Support Small	03070120100099
	Thuasne Sport Open Knee Support Medium	03070120200099
	Thuasne Sport Open Knee Support Large	03070120300099
	Thuasne Sport Open Knee Support X-Large	03070120400099
	Thuasne Sport Open Knee Support XX-Large	03070120500099
REINFORCED KNEE	Thuasne Sport Reinforced Knee Support Small	03540120100199
	Thuasne Sport Reinforced Knee Support Medium	03540120200199
	Thuasne Sport Reinforced Knee Support Large	03540120300199
	Thuasne Sport Reinforced Knee Support X-Large	03540120400199
	Thuasne Sport Reinforced Knee Support XX-Large	03540120500199
KNEE STRAP	Thuasne Sport Knee Strap Small	03310320100399
	Thuasne Sport Knee Strap Medium	03310320200399
	Thuasne Sport Knee Strap Large	03310320300399
	Thuasne Sport Knee Strap X-Large	03310320400399
STRAPPING ELASTIC KNEE	Thuasne Sport Strapping Elastic Knee Brace Small	02700220100399
	Thuasne Sport Strapping Elastic Knee Brace Medium	02700220200399
	Thuasne Sport Strapping Elastic Knee Brace Large	02700220300399
	Thuasne Sport Strapping Elastic Knee Brace X-Large	02700220400399
	Thuasne Sport Strapping Elastic Knee Brace XX-Large	02700220500399

PRODUCT CODES

PRODUCT DESCRIPTION		PRODUCT CODE
OPEN ANKLE	Thuasne Sport Open Ankle Support Small	03060220100099
	Thuasne Sport Open Ankle Support Medium	03060220200099
	Thuasne Sport Open Ankle Support Large	03060220300099
	Thuasne Sport Open Ankle Support X-Large	03060220400099
ANKLE SUPPORT	Thuasne Sport Ankle Support Small	03330320100199
	Thuasne Sport Ankle Support Medium	03330320200199
	Thuasne Sport Ankle Support Large	03330320300199
	Thuasne Sport Ankle Support X-Large	03330320400199
	Thuasne Sport Ankle Support XX-Large	03330320500199
REINFORCED ANKLE	Thuasne Sport Reinforced Ankle Support Small	03530120100199
	Thuasne Sport Reinforced Ankle Support Medium	03530120200199
	Thuasne Sport Reinforced Ankle Support Large	03530120300199
	Thuasne Sport Reinforced Ankle Support X-Large	03530120400199
	Thuasne Sport Reinforced Ankle Support XX-Large	03530120500199
ANKLE STRAP	Thuasne Sport Ankle Strap Small	03300320100399
	Thuasne Sport Ankle Strap Medium	03300320200399
	Thuasne Sport Ankle Strap Large	03300320300399
	Thuasne Sport Ankle Strap X-Large	03300320400399
STRAPPING ELASTIC ANKLE	Thuasne Sport Strapping Elastic Ankle Brace Small	02800220100399
	Thuasne Sport Strapping Elastic Ankle Brace Medium	02800220200399
	Thuasne Sport Strapping Elastic Ankle Brace Large	02800220300399
	Thuasne Sport Strapping Elastic Ankle Brace X-Large	02800220400399
	Thuasne Sport Strapping Elastic Ankle Brace XX-Large	02800220500399
STABILIZ-ING ANKLE	Thuasne Sport Stabilising Ankle Brace BOA Small	03510220100199
	Thuasne Sport Stabilising Ankle Brace BOA Medium	03510220200199
	Thuasne Sport Stabilising Ankle Brace BOA Large	03510220300199
REINFORCED NEOPRENE KNEE	Thuasne Sport Reinforced Neoprene Knee Support Small	05700320100399
	Thuasne Sport Reinforced Neoprene Knee Support Medium	05700320200399
	Thuasne Sport Reinforced Neoprene Knee Support Large	05700320300399
	Thuasne Sport Reinforced Neoprene Knee Support X-Large	05700320400399
	Thuasne Sport Reinforced Neoprene Knee Support XX-Large	05700320500399
REINFORCED PATELLA KNEE	Thuasne Sport Reinforced Patella Knee Brace Small	03550120100199
	Thuasne Sport Reinforced Patella Knee Brace Medium	03550120200199
	Thuasne Sport Reinforced Patella Knee Brace Large	03550120300199
	Thuasne Sport Reinforced Patella Knee Brace X-Large	03550120400199
	Thuasne Sport Reinforced Patella Knee Brace XX-Large	03550120500199
NEOPRENE ANKLE	Thuasne Sport Neoprene Ankle Support Small	05710320100399
	Thuasne Sport Neoprene Ankle Support Medium	05710320200399
	Thuasne Sport Neoprene Ankle Support Large	05710320300399
	Thuasne Sport Neoprene Ankle Support X-Large	05710320400399

ARGICALM

PRODUCT DESCRIPTION		PRODUCT CODE
ARGICALM	Argicalm Size 1	72000220105699
	Argicalm Size 2	72000220205699
	Argicalm Size 3	72000220305699
	Argicalm Size 4	72000220405699
	Argicalm Size 5	72000220505699



THUASNE

Thuasne UK Ltd
Unit 4 Orchard Business Centre, North Farm Road, Tunbridge Wells, TN2 3XF
Tel: 01892 481620 - Fax: 01892 481609
Email: orders@thuasne.co.uk
www.thuasne.co.uk

We reserve the right to modify our products as shown and described in this catalogue,
without previous notice. Non-contractual photographs.
Please check the availability of products from your supplier.