# THUASNE

# SPORT

<b>fr</b>	<b>Chevillère strapping Novelastic Thuasne Sport</b> .....	4
<b>en</b>	<b>Thuasne Sport Novelastic strapping ankle brace</b> .....	5
<b>de</b>	<b>Novelastic Strapping Knöchelbandage Thuasne Sport</b> .....	6
<b>nl</b>	<b>Thuasne Sport Novelastic enkelbrace met elastische banden</b> .....	8
<b>it</b>	<b>Cavigliera strapping Novelastic Thuasne Sport</b> .....	10
<b>es</b>	<b>Tobillera strapping Novelastic Thuasne Sport</b> .....	11
<b>pt</b>	<b>Tornozeleira de strapping Novelastic Thuasne Sport</b> .....	13
<b>da</b>	<b>Ankelbind med stropper Novelastic Thuasne Sport</b> ..	14
<b>fi</b>	<b>Nauhallinen Novelastic Thuasne Sport -nilkkatuki</b> ...	16
<b>sv</b>	<b>Thuasne Sport ankelortos med elastiska band Novelastic</b> .....	17
<b>el</b>	<b>Επιστραγαλίδα strapping Novelastic Thuasne Sport</b> .....	18
<b>cs</b>	<b>Pásková kotníková bandáž Novelastic Thuasne Sport</b> .....	20
<b>pl</b>	<b>Bandażowa orteza skokowo-stopowa Novelastic Thuasne Sport</b> .....	21
<b>lv</b>	<b>Thuasne Sport Novelastic potītes ortoze</b> .....	23
<b>lt</b>	<b>Apjuosiamas kulkšnies įtvaras „Novelastic Thuasne Sport“</b> .....	24
<b>et</b>	<b>Novelastic Thuasne Sport hüppeliigese rihm</b> .....	26
<b>sl</b>	<b>Opornica iz trakov za gleženj Novelastic Thuasne Sport</b> .....	27
<b>sk</b>	<b>Ortéra členku s páskami Novelastic Thuasne Sport</b> ..	28
<b>hu</b>	<b>Thuasne Sport Novelastic pántos bokarögzítő</b> .....	30
<b>bg</b>	<b>Наглезеник с каишки Novelastic Thuasne Sport</b> ....	31
<b>ro</b>	<b>Glezniere cu benzi de continutie Novelastic Thuasne Sport</b> .....	33
<b>ru</b>	<b>Ременный голеностопный ортез Novelastic Thuasne Sport</b> .....	34
<b>hr</b>	<b>Steznik za gležanj s trakom Novelastic Thuasne Sport</b> .....	36
<b>zh</b>	<b>Novelastic Thuasne Sport 绑带式护踝</b> .....	37
<b>ar</b>	<b>Novelastic Thuasne Sport لفافة الكاحل</b> .....	40

 cm	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>	<b>XXL</b>
	19 - 21	21 - 23	23 - 25	25 - 27	27 - 30

<b>fr</b>	Tour de cheville
<b>en</b>	Ankle circumference
<b>de</b>	Knöchelumfang
<b>nl</b>	Omvang van de enkel
<b>it</b>	Giro caviglia
<b>es</b>	Circunferencia del tobillo
<b>pt</b>	Perímetro do tornozelo
<b>da</b>	Anklens omkreds
<b>fi</b>	Nilkanympärys
<b>sv</b>	Runt fotleden
<b>el</b>	Περιμετρος αστραγάλου
<b>cs</b>	Obvod kotníku
<b>pl</b>	Obwód kostki
<b>lv</b>	Potītes apkārtmērs
<b>lt</b>	Kulkšnies apimtis
<b>et</b>	Pahkluu ümbermõõt
<b>sl</b>	Obseg gležnja
<b>sk</b>	Obvod členka
<b>hu</b>	Boka kerülete
<b>bg</b>	Обиколка на глезена
<b>ro</b>	Circumferința gleznei
<b>ru</b>	Окружность голени
<b>hr</b>	Opseg gležnja
<b>zh</b>	脚踝尺寸
<b>ar</b>	محيط الكاحل

## CHEVILLÈRE STRAPPING NOVELASTIC THUASNE SPORT

### Description/Destination

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## THUASNE SPORT NOVELASTIC STRAPPING ANKLE BRACE

### Description/purpose

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Polyamide - Elastodiene - Elastane - Polyurethane - Polyester.

### Properties/Mode of action

Elastic compressive knitting.  
Comfort zone over the ankle bones.

### Indications

Joint instability/laxity.

Proprioceptive support for rehabilitation.

Mild ankle sprain (acute phase).

Sport activities resumption.

### Contraindications

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

### Precautions

Read the instruction leaflet carefully.

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Consult a healthcare professional:

- In case of doubt before use,
- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,
- In case of changes in the device's performance.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

#### **Undesirable side-effects**

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity. Possible risk of venous thrombosis.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### **Instructions for use/Application**

Put on the ankle support (logo facing up) positioning the heel in the appropriate area and limiting the extension strap to the rear of the foot. **1**

Depending on the nature of the injury (sprain of the external or more rarely internal lateral ligament) and the foot concerned (left or right), the positioning of the strap varies:

- on the right foot, start wrapping the strap on the right.
- on the left foot, begin wrapping the strap on the left.

#### **Care**

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Machine washable at 30 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

#### **Storage**

Store at room temperature, preferably in the original packaging.

#### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2033001 (2022-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186