

THUASNE®

en	Sport reinforced knee support	4
fr	Genouillère baleinée Sport	5
de	Sport-Kniebandage mit Stützstäben	6
nl	Kniebrace met baleinen sport	8
it	Ginocchiera dotata di stecche per uso sportivo	9
es	Rodillera con ballenas para uso deportivo	11
pt	Joelheira almofadada para uso desportivo	12
da	Knæbind med stivere sport	14
fi	Vahvikkeellinen polvituki urheilukäyttöön	15
sv	Styvt knästöd sport	17
el	Αθλητική επιγονατίδα με μπανέλες	18
cs	Kolenní ortéza s kosticemi sport	19
pl	Orteza kolanowa z fiszbinami sportowa	21
lv	Stingrā sporta ceļa ortoze	22
lt	Kelio įtvaras su plokštelėmis sportui	24
et	Spordi küljetugevdusega põlvkaitse	25
sl	Opornica za koleno z ojačitvami šport	26
sk	Športová kolenná ortéza s bočnými výstužami	28
hu	Térdmerevítő sportoláshoz	29
bg	Наколенка с банели за спорт	31
ro	Genunchieră cu balene sport	32
hr	Sport steznik za potporu koljena	34

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

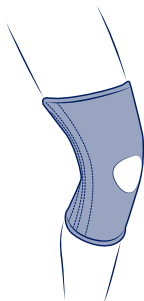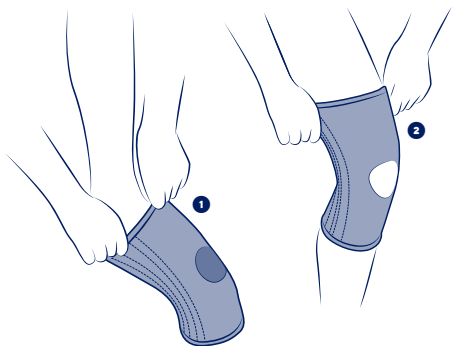cm
S
31 - 33
M
34 - 36
L
37 - 39
XL
40 - 42
XXL
43 - 45

en	Knee circumference in slight flexion
fr	Circonférence de genou en légère flexion
de	Knieumfang bei leichter Beugung
nl	Omvang van de licht geplooidde knie
it	Circonferenza del ginocchio in leggera flessione
es	Circunferencia de la rodilla con ligera flexión
pt	Circunferência do joelho em ligeira flexão
da	Knæets omkreds i let bøjning
fi	Polven ympärys hieman koukistettuna
sv	Omkrets runt knäet i lätt böjning
el	Περιφέρεια γόνατος σε ελαφρά κάμψη
cs	Obvod kolene v mírném ohybu
pl	Obwód kolana przy lekkim zgięciu
lv	Ceļgala apkārtmērs nelielā saliekumā
lt	Šiek tiek sulenкто kelio apimtis
et	Kergelt pinguldatult põlve ümber
sl	Obseg kolena pri rahlem upogibu
sk	Obvod kolena s miernym ohybom
hu	Térd kerülete enyhe hajlításban
bg	Обиколка на коляното при лека флексия
ro	Circumferința genunchiului în flexiune ușoară
hr	Obuhvaća koljeno u blagoj fleksiji

SPORT REINFORCED KNEE SUPPORT

Support level 2

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 5 sizes.

The device is composed of:

- flexible side boning,
- a synthetic rubber (chloroprene) with an opening in the kneecap.

This device can be used when resuming sports activities.

Composition

Textile components: synthetic rubber (chloroprene) - polyamide - elastane - polyester - natural rubber.

Rigid components: steel.

Properties/Mode of action

The double stays allow a knee joint support.

Limitation of the heat loss by the synthetic rubber (Chloroprene).

Indications

Knee pain and/or swelling management.

Joint instability/laxity.

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Contains natural latex.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Do not wear the device in a medical imaging machine.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Pull the knee support ❶.

Centre the front opening over the kneecap ❷.

Thusne Logo must be on the upper part, centered and focused on the patella.

Care/Maintenance

Hand washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations

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©Thuasne - 2048301 (2023-10)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186