
THUASNE®

Ligaflex Finger®



fr	Attelle d'immobilisation poignet-doigt	4
en	Wrist-finger immobilisation splint	4
de	Immobilisierungsschiene für Handgelenk und Finger	5
nl	Immobilisatiebrace voor pols en vinger.....	6
it	Tutore di immobilizzazione polso-dita	7
es	Férula de inmovilización muñeca-dedo	8
pt	Tala de imobilização para o punho/dedo	9
da	Skinne til immobilisering af håndled-finger	10
fi	Ranteen ja sormien immobilisaatiolasta	11
sv	Immobilieringsskena till handled-finger.....	12
el	Νάρθηκας ακινητοποίησης καρπού-δακτύλου	12
cs	Imobilizační ortéza zápěstí a prstů.....	13
pl	Orteza unieruchamiająca palec i nadgarstek.....	14
lv	Rokas pirksta imobilizācijas šina.....	15
lt	Riešo ir pirštų imobilizavimo įtvaras	16
et	Randme-sõrme fikseerimise lahas	17
sl	Opornica za imobilizacijo zapestja in prsta	18
sk	Dlaha na znehybnenie zápästia a prstov	19
hu	Csukló-ujjrögzítő sín.....	20
bg	Шина за обездвижване китка-пръст	20
ro	Atelă de imobilizare încheietura mâinii-deget	21
ru	Шина для иммобилизации запястья-пальца.....	22
hr	Udlaga za imobilizaciju ručnog zgloba	23
zh	手腕和手指固定夹板	24
ar	جبيرة لتثبيت الرسغ والأصابع	27

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال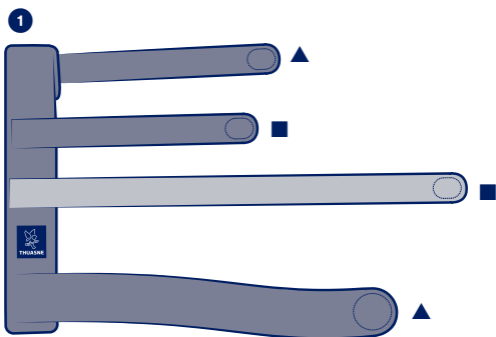

fr	▲ Sangle fixe
en	▲ Fixed strap
de	▲ Fester Gurt
nl	▲ Vaste riem
it	▲ Cinghia fissa
es	▲ Correa fija
pt	▲ Tira fixa
da	▲ Fast strop
fi	▲ Kiinteä remmi
sv	▲ Fast rem
el	▲ Σταθερός μάνταξ
cs	▲ Pevný pásek
pl	▲ Pasek stały
lv	▲ Fiksēta sikсна
lt	▲ Pritvirtintas diržas
et	▲ Püsirihm
sl	▲ Nesnemljivi trak
sk	▲ Fixný popruh
hu	▲ Rögzített pánt
bg	▲ Фиксиран ремък
ro	▲ Chingă fixă
ru	▲ Фиксированный ремень
hr	▲ Fiksni remen
zh	▲ 固定束帶
ar	▲ حزام ثابت

■ Sangle amovible
■ Removable strap
■ Abnehmbarer Gurt
■ Afneembare riem
■ Cinghia rimovibile
■ Correa amovible
■ Tira amóvível
■ Aftagelig strop
■ Irrotettava remmi
■ Avtagbar rem
■ Αφαιρούμενος μάνταξ
■ Odnímatelný pásek
■ Pasek zdejmowany
■ Noņemama sikсна
■ Nuimamas diržas
■ Eemaldatav rihm
■ Snemljivi trak
■ Odnímatelný popruh
■ Levehető pánt
■ Подвижен ремък
■ Chingă amovibilă
■ Съемный ремень
■ Pomični remen
■ 可拆卸束帶
■ حزام قابل للنزع

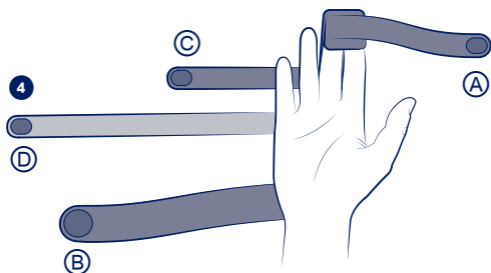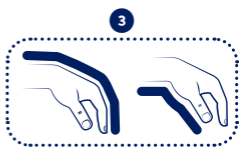	1	< 13 cm
	2	13 - 16 cm
	3	16 - 19 cm
	4	19 - 22 cm

fr	Distance bout doigt lésé-poignet
en	Tip of injured finger - wrist distance
de	Abstand zwischen Fingerspitze des verletzten Fingers und Handgelenk
nl	Afstand top gekwetste vinger-pols
it	Distanza dal polso alla punta del dito lesa
es	Distancia extremo dedo lesionado-muñeca
pt	Distância da ponta do dedo lesionado - punho
da	Afstand skadet finger (spids) - håndled
fi	Etäisyys vahingoittuneesta sormesta ranteeseen
sv	Avstånd mellan skadad fingertopp - handled
el	Απόσταση άκρης τραυματισμένου δακτύλου - καρπού
cs	Vzdálenost špičky poškozeného prstu a zápěstí
pl	Odległość od zakończenia uszkodzonego palca do nadgarstka
lv	Attālums līdz traumētā pirksta galam no plaukstas locītavas
lt	Atstumas nuo riešo iki sužaloto piršto galo
et	Kaugus haigest sõrmest randmeni
sl	Razdalja na poškodovanem prstu med vrhom prsta in zapestjem
sk	Vzdialenosť od špičky poraneného prsta - zápästie
hu	A sérült ujjhegy és a csukló közötti távolság
bg	Разстояние от върха на увредения пръст до китката
ro	Distanță vârful deget vătămat - încheietură mână
ru	Расстояние между концом пальца-запястьем
hr	Udaljenost između ozlijeđenog vrha prsta-ručnog zgloba
zh	受伤手指指尖与腕部的距离
ar	المسافة بين طرف الأصبع المجرّوح - الرسغ

fr	Immobilisation
en	Immobilisation
de	Ruhigstellung
nl	Immobilisatie
it	Immobilizzazione
es	Inmovilización
pt	Imobilização
da	Støtte
fi	Immobilisoi
sv	Immobilisering
el	Ακινητοποίηση
cs	Znehybnění
pl	Unieruchomienie
lv	Imobilizācija
lt	Imobilizavimas
et	Liikumatuks muutmine
sl	Imobilizacija
sk	Imobilizácia
hu	Rögzítés
bg	Обездвижване
ro	Imobilizare
ru	Иммобилизация
hr	Imobilizacija
zh	固定
ar	التثبيت

fr

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WRIST-FINGER IMMOBILISATION SPLINT

Description/Destination

This device is a wrist-finger splint providing immobilisation of the fingers.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral model.

Composition

polyamide - elastane - ethylene vinyl acetate - polyethylene - polyester - aluminum.

Properties/Mode of action

Finger immobilisation splint (2nd, 3rd or 4th).

Conformable and adjustable stay, adapts to the required angulation (flexion or extension).

The splint features 4 straps (2 fixed and 2 removable) to adjust support and tightening. ❶

Indications

Finger injuries (sprains, dislocations, ligament ruptures...).

Post-operative/post-traumatic immobilisation.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use in case of unstable fractures.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

For hygiene and performance reasons, do not re-use the product for another patient.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

According to the complaint and medical advice, the product may be fitted in the palmar position or in the dorsal position.

The injured finger can be immobilised on its own or using double-finger support (injured finger + healthy finger).

Configure the stay according to the type of support required:

- Metacarpophalangeal joints: maximum flexion ❷

- Interphalangeal joints: extension ❸

Position the brace, ensuring that the fingertip(s) is/are inserted into the upper end of the brace. ❹

Fasten the straps in the order shown below Ⓐ Ⓑ Ⓒ Ⓓ. ❺

Tighten the straps reasonably well for proper use of the device (sufficient support).

Care/Maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Machine washable at 30°C (delicate programme). If possible use a washing net. Close the self-fastening tabs before washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2050901 (2023-11)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186