<b>en</b>	Wrist circumference. Universal size
<b>fr</b>	Tour de poignet. Taille unique
<b>de</b>	Handgelenkumfang. Einheitsgröße
<b>nl</b>	Polsomtrek. Eén maat
<b>it</b>	Circonferenza del polso. Taglia unica
<b>es</b>	Contorno de la muñeca. Talla única
<b>pt</b>	Perímetro do punho. Tamanho único
<b>da</b>	Håndleddets omkreds. Kun én størrelse
<b>fi</b>	Ranteen ympärysmitta. Yksi koko
<b>sv</b>	Handledens omkrets. En storlek
<b>el</b>	Περίμετρος καρπού. Ένα μέγεθος
<b>cs</b>	Obvod zápěstí. Univerzální velikost
<b>pl</b>	Obwód nadgarstka. Rozmiar uniwersalny
<b>lv</b>	Plaukstu locītavas apkārtmērs. Universāls izmērs
<b>lt</b>	Riešo apimtis. Vienas dydis
<b>et</b>	Randme ümbermõõt. Üks suurus
<b>sl</b>	Obseg zapestja. Enotna velikost
<b>sk</b>	Obvod zápästia. Univerzálna veľkosť
<b>hu</b>	Csukló kerülete. Egy méret
<b>bg</b>	Обиколка на китката. Един размер
<b>ro</b>	Circumferința încheieturii. Mărime universală
<b>hr</b>	Obujam ručnog zgloba. Univerzalna veličina

en

## **SPORT ELASTIC WRIST SUPPORT**

*Support level 1*

### **Description/Intended use**

This device is an elastic wrist orthosis providing joint support. The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### **Composition**

Polyamide - elastane - synthetic rubber (CR).

### **Properties/Mode of action**

Adjustable fastening system.

Bilateral product.

### **Indications**

Wrist sprain.

Wrist tendinopathies.

### **Contraindications**

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use the product if the diagnosis has not been confirmed.

### **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application**

Open the hook-and-loop fasteners.

Place the support on the back of the hand and insert the thumb. **1**

Tighten the support using the hook-and-loop fasteners. **2**

### **Garment care**

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Hand wash. Close the self-fastening tabs before washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

**Storage**

Store at room temperature, preferably in the original packaging.

**Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2052901 (2024-03)
**THUASNE UK Ltd**
Thuasne SAS – SIREN/RCS Nanterre 542 091 186