


# THUASNE

# SPORT

<b>fr</b>	Strapping de pouce Thuasne Sport.....	4
<b>en</b>	Thuasne Sport thumb strapping.....	5
<b>de</b>	Strapping Daumenbandage Thuasne Sport .....	6
<b>nl</b>	Thuasne Sport duimbrace met elastische banden .....	7
<b>it</b>	Strapping per il pollice Thuasne Sport .....	9
<b>es</b>	Strapping de pulgar Thuasne Sport .....	10
<b>pt</b>	Strapping de polegar Thuasne Sport.....	11
<b>da</b>	Tommelfingerrem Thuasne Sport.....	12
<b>fi</b>	Nauhallinen Thuasne Sport -peukalotuki.....	14
<b>sv</b>	Thuasne Sport elastiskt band för tumme .....	15
<b>el</b>	Strapping αντίχειρα Thuasne Sport .....	16
<b>cs</b>	Pásková bandáž palce Thuasne Sport.....	17
<b>pl</b>	Bandażowa orteza na kciuk Thuasne Sport .....	19
<b>lv</b>	Thuasne Sport īkšķa ortoze .....	20
<b>lt</b>	Apjuosiamas nykščio įtvaras „Thuasne Sport“ .....	21
<b>et</b>	Thuasne Sport põidlarihm.....	22
<b>sl</b>	Opornica za palec Thuasne Sport.....	23
<b>sk</b>	Páska na palec Thuasne Sport.....	25
<b>hu</b>	Thuasne Sport hüvelykujj rögzítőpánt .....	26
<b>bg</b>	Кайшка за палеца Thuasne Sport .....	27
<b>ro</b>	Chingă pentru degetul mare Thuasne Sport .....	28
<b>ru</b>	Ременный ортез для большого пальца Thuasne Sport .....	30
<b>hr</b>	Traka za palac Thuasne Sport .....	31
<b>zh</b>	Thuasne Sport 拇指绷带 .....	32
<b>ar</b>	Thuasne Sport لفافة الإبهام.....	35

	<b>S</b>	<b>M</b>	<b>L</b>
	cm 14 - 15,5	16 - 17,5	18 - 20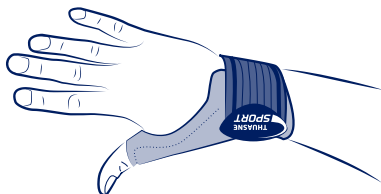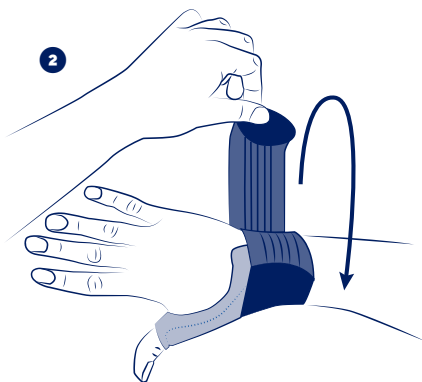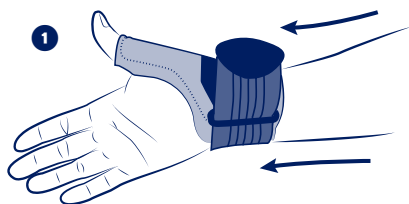

<b>fr</b>	tour de poignet
<b>en</b>	wrist circumference
<b>de</b>	Handgelenkumfang
<b>nl</b>	polsomtrek
<b>it</b>	circonferenza del polso
<b>es</b>	contorno de la muñeca
<b>pt</b>	perímetro do punho
<b>da</b>	håndleddets omkreds
<b>fi</b>	ranteen ympärysmitta
<b>sv</b>	håndledens omkrets
<b>el</b>	Περίμετρος καρπού
<b>cs</b>	obvod zápěstí
<b>pl</b>	obwód nadgarstka
<b>lv</b>	plaukstas locītavas apkārtmērs
<b>lt</b>	riešo apimtis
<b>et</b>	Randme ümbermõõt
<b>sl</b>	obseg zapestja
<b>sk</b>	obvod zápästia
<b>hu</b>	csukló kerülete
<b>bg</b>	обиколка на китката
<b>ro</b>	circumferința încheieturii
<b>ru</b>	Обхват запястья
<b>hr</b>	obujam ručnog zgloba
<b>zh</b>	腕围
<b>ar</b>	محيط الرسغ

## fr **STRAPPING DE POUCE THUASNE SPORT**

### **Description/Destination**

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## **en THUASNE SPORT THUMB STRAPPING**

### **Description/purpose**

This device is a thumb brace providing joint stabilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### **Composition**

Polyamide - Polyester - Synthetic rubber (SBR) - Polyurethane - Elastane.

### **Properties/Mode of action**

Thumb stabilisation via a plastic wire stay.

Embossed synthetic rubber (SBR).

A hook-and-loop fastening strap to hold the product in place.

### **Indications**

Thumb osteoarthritis (including rhizarthrosis).

Immobilisation in case of pain or trauma of the thumb.

### **Contraindications**

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

### **Precautions**

Read the instruction leaflet carefully.

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional:

- In case of doubt before use,

- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,

- In case of changes in the device's performance.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

### **Undesirable side-effects**

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application**

Thread the brace over the wrist and insert the thumb into its housing, ensuring the whole base of the thumb is covered ①.

Fold the elastic strap over and tighten it around the wrist ②.

### **Care**

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Machine washable at 30 °C. Close the self-fastening tabs before washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Do not dry clean. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

### **Storage**

Store at room temperature, preferably in the original packaging.

### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne -2033901 (2022-02)
THUASNE UK Ltd**
Thuasne SAS – SIREN/RCS Nanterre 542 091 186