
THUASNE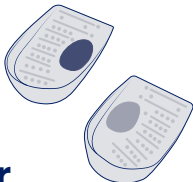

# Pedipro Softer


<b>fr</b>	Talonnets silicone évolutives.....	5
<b>en</b>	Adaptable silicone heelpieces.....	7
<b>de</b>	Anpassbare Ferseneinlagen aus Silikon.....	8
<b>nl</b>	Aanpasbare siliconen zolen.....	10
<b>it</b>	Alzatacchi in silicone evolutivi.....	13
<b>es</b>	Talonerías de silicona evolutivas.....	15
<b>pt</b>	Palmilhas de calcanhar de silicone evolutivas.....	17
<b>da</b>	Tilpasselige indlægshæle i silikone.....	19
<b>fi</b>	Silikonikantapäätuet.....	20
<b>sv</b>	Anpassningsbara silikoninlägg.....	22
<b>el</b>	Προσαρμόσιμα υποπέδιλα σιλικόνης.....	24
<b>cs</b>	Silikonové podpatěnky s výměnnými prvky.....	26
<b>pl</b>	Podpiętki silikonowe z możliwością zmiany rozmiaru.....	28
<b>lv</b>	Pielāgojami silikona papēži.....	30
<b>lt</b>	Prisitaikančios silikoninės pakulnės.....	32
<b>et</b>	Reguleeritava suurusega silikoonist kannapadjad.....	34
<b>sl</b>	Silikonski progresivni petni nastavki.....	36
<b>sk</b>	Prispôsobiteľné silikónové podpätenky.....	38
<b>hu</b>	Méretezhető szilikon sarokemelő.....	40
<b>bg</b>	Разработени силиконови подложки за пета.....	42
<b>ro</b>	Talonete evolutive din silicon.....	44
<b>ru</b>	Адаптивные силиконовые подпяточники.....	46
<b>hr</b>	Promjenjivi silikonski jastučići za petu.....	48
<b>zh</b>	可演变的硅胶足跟护垫.....	50
<b>ar</b>	وسائد كعب سيليكونية تطورية.....	53

		Europe	UK
	1	36 - 39	3 <sup>1/2</sup> - 5 <sup>1/2</sup>
	2	40 - 46	6 <sup>1/2</sup> - 11

<b>fr</b>	Pointure - Europe - UK
<b>en</b>	Shoe size - Europe - UK
<b>de</b>	Schuhgröße - Europa - UK
<b>nl</b>	Schoenmaat - Europa - VK
<b>it</b>	Numero di scarpa - Europa - Regno Unito
<b>es</b>	Número de pie - Europa - Reino Unido
<b>pt</b>	Número do sapato - Europa - Reino Unido
<b>da</b>	Størrelse - Europa - UK
<b>fi</b>	Kengän koko - Eurooppa - Yhdistynyt kuningaskunta
<b>sv</b>	Skostorlek - Europa - Storbritannien
<b>el</b>	Νούμερο υποδημάτων - Ευρώπη - ΗΒ
<b>cs</b>	Velikost obuvi - Evropa - Spojené království
<b>pl</b>	Numer buta - Europa - Wielka Brytania
<b>lv</b>	Apavu izmērs - Eiropa - Apvienotā Karaliste
<b>lt</b>	Avalynės dydis - Europa - JK
<b>et</b>	Suurus - Euroopa - Ühendkuningriigid
<b>sl</b>	Velikost obutve - Evropa - UK
<b>sk</b>	Veľkosť - Európa - Spojené kráľovstvo
<b>hu</b>	Cipőméret - Európa - UK (Egyesült Királyság)
<b>bg</b>	Размер - Европа - Обединено кралство
<b>ro</b>	Mărimea pantofului - Europa - Regatul Unit
<b>ru</b>	Размер обуви - Европа - Великобритания
<b>hr</b>	Broj cipela - Europa - UK
<b>zh</b>	鞋码-欧洲-英国
<b>ar</b>	المقاس - أوروبا - المملكة المتحدة

Décharge	Restaure l'équilibre	Correction biomécanique
Off-loading	Balance restoration	Biomechanical correction
Entlastung	Stellt das Gleichgewicht wieder her	Biomechanische Korrektur
Losmaken	Herstelt het evenwicht	Biomechanische correctie
Scarico	Restauro l'equilibrio	Correzione biomeccanica
Descarga	Restauro el equilibrio	Corrección biomecánica
Alívio do peso	Restauro o equilíbrio	Correção biomecânica
Aflastning	Genopretter ligevægten	Biomekanisk korrektion
Vaputus	Palauttaa tasapainon	Biomekaaninen korjaus
Avlastning	Förbättrar balansen	Biomekanisk korrigering
Αποφόρτιση	Αποκαθιστά την ισορροπία	Βιο-μηχανικός έλεγχος
Snížení zátěže	Obnovuje rovnováhu	Biomechanická korekce
Odciążenie	Produkt przywraca równowagę	Korekcja biomechaniczna
Atbrīvošana	Līdzsvara atjaunošana	Biomehāniskā korekcija
Apkrovis sumažinimas	Pusiausvyros palaikymas	Mechaninė biologinė korekcija
Koormuse vähendamine	Taastab tasakaalutunnetuse	Biomehaaniline korrektsioon
Razbremenitev	Vzpostavi nazaj ravnovesje	Popravek biomehanske motnje
Uvoľnenie	Obnovenie rovnováhy	Biomechanická korekcia
Tehermentesítés	Az egyensúly helyreállítása	Biomechanikus korrekció
Отбремявање	Възстановява баланса	Биомеханична корекция
Descărcare	Restabilirea echilibrului	Corecție biomecanică
Разгрузка	Восстановление равновесия	Биомеханическая коррекция
Rasterečenje	Uspostavljanje ravnoteže	Biomehānička korekcija
减压	恢复平衡	生物力学矫正
تخفيف ثقل	إعادة التوازن.	تصحيح ميكانيكي حيوي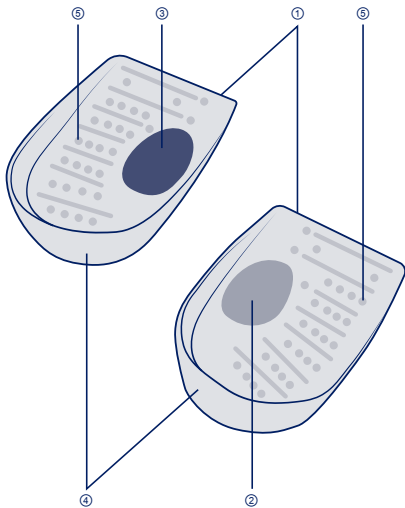

**fr**

## **TALONNETTES SILICONE ÉVOLUTIVES**

### **Description/Destination**

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## **ADAPTABLE SILICONE HEELPIECES**

### **Description/Destination**

Components providing even distribution of pressure, shock absorption and promoting hygienic freshness. ①

Choice of two (green and blue) anatomically-shaped inserts designed to be placed in the cut-out provided:

- Green anatomically-shaped inserts which are more flexible and designed for heel spurs. ②

- Blue anatomically-shaped inserts designed for healthy feet. ③

Wide rim at the heel optimising hold in the shoe. ④

Ventilation channels allow air flow in the shoe. ⑤

### **Composition**

silicone - silver derivative.

### **Properties/Mode of action**

Off-loading.

Balance restoration.

Biomechanical correction.

### **Indications**

Heel and foot pain (metatarsalgia, plantar fasciitis).

Rheumatoid foot.

Relief for tendinitis.

### **Contraindications**

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

### **Precautions**

Consult a healthcare professional:

- In case of doubt before use,

- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in color of the extremities,

- In case of changes in the device's performance.

Choose the appropriate size to fit, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

### **Undesirable side-effects**

This device can cause skin reactions (redness, itching, irritations...). Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application**

Select the insert corresponding to your condition:

- Green insert to be used for the foot with the heel spur,
- Blue insert to be used for the healthy foot.

Slip the selected insert into the cut-out provided for this purpose.

Clean the interior of the shoe.

Place the heelpiece flat in the shoe, pressing the rim of the product against the heel of the shoe.

N.B.: this product is designed for a specific side, indicated by "RIGHT" for your right shoe and "LEFT" for your left shoe.

Repeat this operation with the other heelpiece.

### **Care**

It is recommended to wash the product every day, or after each use. Hand wash. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

### **Storage**

Store at room temperature, preferably in the original packaging.

### **Disposal**

Dispose of in accordance with local regulations.

This medical device is a regulated health product with CE marking

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©Thuasne - 2032302 (2022-07)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186