
THUASNE

Ortel T-Neck


Ortel T-Neck Plus

- de Gebrauchsanweisung - Cervical-Orthese
- en Instructions for use - Cervical orthosis
- fr Consignes d'utilisation - Collier cervical
- it Istruzioni per l'uso - Ortesi cervicali
- es Instrucciones de utilización - Órtesis cervical
- pt Instrukções de uso - Órtese cervical
- nl Gebruiksaanwijzing - Cervicaal orthese
- da Brugsanvisning - Cervikal-ortese
- sv Bruksanvisning - Cervikal-ortos
- fi Käyttöohje - Kaularankatukiside
- el Οδηγίες χρήσης - Αυχενικός κηδεμόνας
- cs Návod k použití - Pevný krční límec
- pl Sposób użytkowania - Orteza szyjna
- et Kasutusjuhend - Tservikaalne ortoos
- lv Lietošanas instrukcija - Cervikālā ortoze
- lt Naudojimo instrukcija - Ortopedinis kaklo įtvaras
- sk Návod na použitie - Pevný krčný golier
- sl Navodila za uporabo - Vratna ortoza
- hu Használati utasítás - Nyaki ortézis
- ro Instrucțiuni de utilizare - Orteză cervicală
- bg Инструкция за употреба - Цервикална ортеза
- hr Upute za uporabu - Cervikalna ortoza

(1)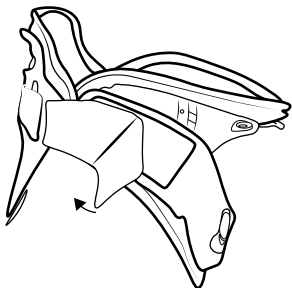


(2)

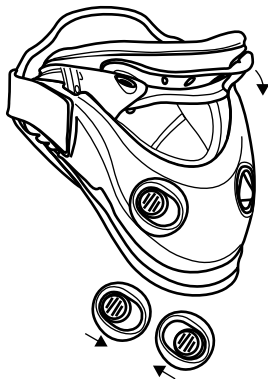(3)


(4)

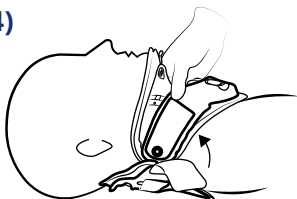
(5)


(6)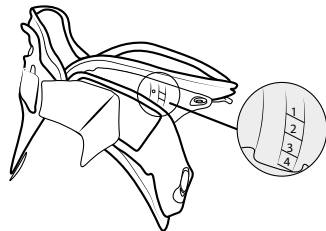


(7)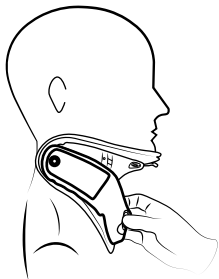


(8)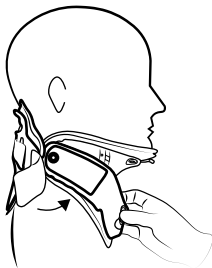

(9)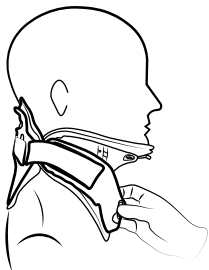

(10)

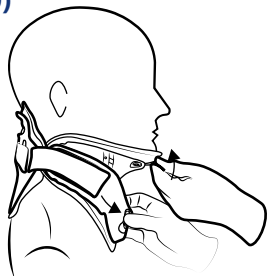Ortel T-Neck (49 100) Ortel T-Neck Plus (49 105)

de

Cervical-Orthese

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Cervical orthosis

Purpose

The Ortel T-Neck and Ortel T-Neck Plus are orthoses for stabilising (T-Neck) and immobilising (T-Neck Plus) the cervical spine. The orthoses may only be used for cervical spine treatment.

Indications

Ortel T-Neck:

- Radicular pain in the cervical spinal region
- Severe whiplash injury
- Moderate to severe cervical syndrome
- Inflammations
- Stable, non-dislocated vertebral body fractures (without neurological symptoms)

Ortel T-Neck Plus:

- Preoperative
- Postoperative
- Posttraumatic
- Inflammatory diseases
- Degeneration
- Severe instabilities of the cervical spine
- Stable, non-dislocated vertebral body fractures (without neurological symptoms)

Contraindications

- Skin injuries or diseases in the body segment receiving care
- Unstable vertebral body fractures combined with neurological phenomena

Side effects

To date, no side effects have been registered during proper use.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Check the skin in the region of the orthosis contact surfaces regularly for tender spots, redness or other changes. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Fitting instructions

Before using the product, carefully read the following instructions completely. The product must be put on properly for it to function as it should.

The Ortel T-Neck and T-Neck Plus must be fitted individually to each patient by qualified experts. At least two people are needed for the first fitting: One to hold the head and neck of the patient in the correct position while the other fits the orthosis.

Putting on in the supine position:

Opening the orthosis

1. Detach the rear head section from the orthosis by loosening the Velcro fasteners on the left- and right-hand sides (Fig. 1).
2. Simultaneously pull the two buttons on the front of the orthosis toward the centre to release the lock, then move the chinrest down to the lowest position (Fig. 2).

Putting on the orthosis

3. The patient is in the supine position with head relaxed. Make sure to keep the patient's head in neutral. Don't extend the front too far as this could put the patient's neck into hyperextension. Slide the rear head section under the neck of the patient (Fig. 3). Use the straps to centre the rear head section between ear and upper shoulder.
4. Slide the front frame into the appropriate position on the patient's neck, hold the front section of the orthosis in place and put the sides of the orthosis close to the patient's neck. Now close the orthosis by alternately tightening the Velcro straps (Fig. 4), making them the same length on each side. (Protruding ends can be shortened using scissors if necessary.)
5. To adjust the chin height, pull the two buttons on the front section of the orthosis toward the centre and then slowly raise the chinrest to the required neutral position (Fig. 5) so that the patient's chin rests on the foam padding.
6. Release the buttons to lock in position once the correct angle is found. Please make sure that the adjusted height is the same on each side (Fig. 6) and locked in position properly.
7. If the orthosis has been fitted properly, it should no longer be possible for the chin to slip.

Putting on in the seated position:

Opening the orthosis

1. Detach the rear head section from the orthosis by loosening the Velcro fasteners on the left- and right-hand sides (Fig. 1).
2. Simultaneously pull the two buttons on the front of the orthosis toward the centre to release the lock, then move the chinrest down to the lowest position (Fig. 2).

Putting on the orthosis

3. Place the patient in a neutral position with the head aligned to the centre.
4. Slide the front frame into the appropriate position on the neck (Fig. 7) and place the neck section of the orthosis centrally on the neck. Then attach the neck section to the left- and right-hand side of the orthosis using the Velcro fasteners (Fig. 8) by alternately tightening the Velcro straps so that they are the same length on both sides. The straps should be between the ear and upper shoulder (Fig. 9). (Protruding ends can be shortened using scissors if necessary.)
5. To adjust the chin height, pull the two buttons on the front section of the ortho-

sis toward the centre and then slowly raise the chinrest to the required neutral position (Fig. 10) so that the patient's chin rests on the foam padding.

6. Release the buttons to lock in position once the correct angle is found. Please make sure that the adjusted height is the same on each side (Fig. 6) and locked in position properly.

7. If the orthosis has been fitted properly, it should no longer be possible for the chin to slip.

Taking off the orthosis:

Loosen one of the Velcro straps on the left- and right-hand sides, then remove the orthosis.

Care

- Do not use any cleaning agents, conditioners or aggressive products (containing chlorine).
- Remove the pads from the orthosis before washing.
- Wash the pads by hand with water and mild soap, then wring out.
- Squeeze the water out of the pads and dry away from any heat sources.
- All plastic parts can be cleaned with a damp cloth.

Material composition

Polyamide, polycarbonate, polypropylene, polyester, EVA foam

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
THUASNE DEUTSCHLAND GmbH